

FEB 0 5 1974

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTERED
ED UNDER NO. 8654-2

MEDICAL CHEMICAL CORP., Santa Monica, California 90404

NOTE

This will not rust or corrode surgical steel, or other metals.

May be used for disinfection of metal, plastic, rubber, or glass instruments and apparatus including catheters, gloves, hypodermic equipment and tubing.

Not to be relied upon to destroy bacterial spores, Mycobacterium tuberculosis or the etiological agent of viral hepatitis.

CAT. NO. 142A
NET CONTENTS
1 GALLON

MEDICAL CHEMICAL

GERMICIDAL SOLUTION

BRAND OF BENZALKONIUM CHLORIDE
AQUEOUS SOLUTION 1:750
FOR DISINFECTION OF DENTAL AND
SURGICAL INSTRUMENTS

ACTIVE INGREDIENTS:

ALKYL (40% C₁₂, 50% C₁₄, 10% C₁₆) DIMETHYL
BENZYL AMMONIUM CHLORIDES 0.133%
SODIUM CARBONATE 0.416%
INERT INGREDIENTS 99.451%

CONTAINS SODIUM NITRITE

Non Rusting Stainless Stable

CAUTION
HARMFUL IF SWALLOWED
KEEP OUT OF REACH OF CHILDREN
E.P.A. REG. NO. 8654-2

DIRECTIONS

The solution should be used full strength. Clean instruments well and rinse. Submerge article in solution for at least 10 minutes. Instrument may be used immediately without further rinsing or drying.

For destruction of spore forming bacteria, steam autoclaving is mandatory, after which instruments or needles may be stored in a sterile covered pan in this solution until used.

**NEW STRENGTH —
REPLACES 1:1000**


Rinse empty container thoroughly with water and discard it.

4 3/4

9"

FEB 0 5 1974

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTRY
EPA UNDER NO. 8654-2

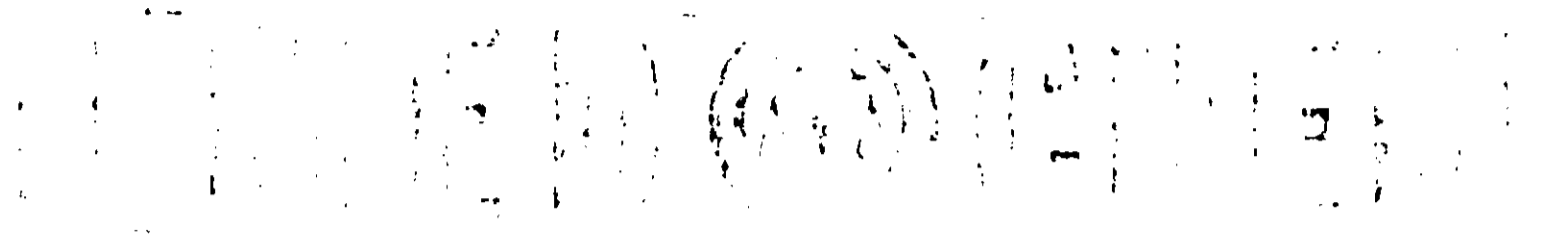

NOTE

This will not rust or corrode surgical steel or other metals.

May be used for disinfection of metal, plastic, rubber, or glass instruments and apparatus including catheters, gloves, hydrostatic equipment and tubing.

Not to be relied upon to destroy bacterial spores, Mycobacterium tuberculosis or the etiologic agent of viral hepatitis.

CAT. NO. 142A
NET CONTENTS
1 GALLON

GERMICIDAL SOLUTION

BRAND OF BENZALKONIUM CHLORIDE
AQUEOUS SOLUTION 1:750
FOR DISINFECTION OF DENTAL AND
SURGICAL INSTRUMENTS

ACTIVE INGREDIENTS:
ALFALAC, INC., 1000 N. 10th Street, Milwaukee, Wis. 53233
BENZYL ALKYL DIMETHYL AMMONIUM CHLORIDE 0.133%
SODIUM CARBONATE 0.416%
INERT INGREDIENTS 99.451%

CONTAINS SODIUM NITRITE
Non Rusting Stainless Stable

CAUTION
HARMFUL IF SWALLOWED
KEEP OUT OF REACH OF CHILDREN
EPA REG. NO. 8654-2

DIRECTIONS

The solution should be used full strength. Clean instruments well and rinse. Submerge article in solution for at least 10 minutes. Instrument may be used immediately without further rinsing or drying.

For destruction of spore forming bacteria, steam autoclaving is mandatory, after which instruments or needles may be stored in a sterile covered pan in this solution until used.

NEW STRENGTH —
REPLACES 1:1000

Rinse empty container thoroughly with water and discard it.

9"