

CAUTION

This germicide must come in contact with all surfaces of the instruments to insure thorough disinfection. This solution should not be relied upon to disinfect instruments which have been exposed to spore bearing and Mycobacterium tuberculosis, or the etiologic agent for viral hepatitis. This germicide should also not be relied upon to disinfect instruments with deep or narrow crevices, hypodermic needles and syringes.

Made in U.S.A.

Graham GERMICIDE CONCENTRATE FOR CHEMICAL DISINFECTION OF DENTAL AND SURGICAL INSTRUMENTS.

Active Ingredients:
N-Alkyl (C14, C18,
dimethyl benzyl-am
chloride 8.33%
Trisodium salt of ethylene
diamine tetracetic
acid 1.67%
Inert ingredients . . . 91.67%
100.00%



2 fl. oz.

CAUTION: Keep out of reach of children. Avoid contact with skin and eyes. In case of contact, flush with plenty of water. If irritation persists, get medical attention. Avoid contamination of food and foodstuffs.

DIRECTIONS

When contents of this bottle are diluted to one gallon of water, there results a solution of 1:1000 Benzalkonium Chloride. No anti-rust need be used. Solution should be changed periodically.

NOTE: All instruments should be washed to remove pus, blood or exudates. All soap should be washed away. Instruments should be immersed for at least fifteen minutes. They may be immersed longer, if desired.

U.S.D.A. REG. NO. 8187-1
GRAHAM CHEMICAL CORP.
Springfield Gardens, N.Y.
Lot No.

ACCEPTED

5-3-67

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTRATION
ED UNDER NO. 8789-1

A O R

1 Fluid Ounce

No. 215

Disinfectant 10% Solution

Active ingredients: alkyl (C₁₂61%,
C₁₄23%, C₁₆11%, C₈, C₁₀,
C₁₈5%) dimethyl benzyl am-
monium chlorides10%

Inert ingredient: water90%

Must be diluted with water as di-
rected. Do not use with soap or
anionic detergents.

For use with Syncro-Mate® ap-
plicators.

CAUTION:

Keep out of the
reach of children.

Avoid contact with
skin and eyes. In case
of contact, wash thor-
oughly with water. If
eye irritation persists,
consult physician
promptly. Avoid con-
tamination of food.
Harmful if swallowed.

Directions for Use

Add 1/2 ounce (1/2 bottle)
to 1/2 gallon of water (800
ppm). After each use wash
the Syncro-Mate applica-
tors in disinfectant solu-
tion and hold for 1 minute
before reuse. See direc-
tions for use of applicator.
U.S.D.A. Reg. No. 8818-1

Distributed by
Animal Health Division

SEARLE

Skokie, Illinois 60076

ACCEPTED


JUL 5 1967

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTERED
UNDER NO. 5818-1 SUBJECT
~~TO ATTACHED COMMENTS.~~

SYNCRO-MATE[®]


APPLICATOR KIT

See other side
for directions

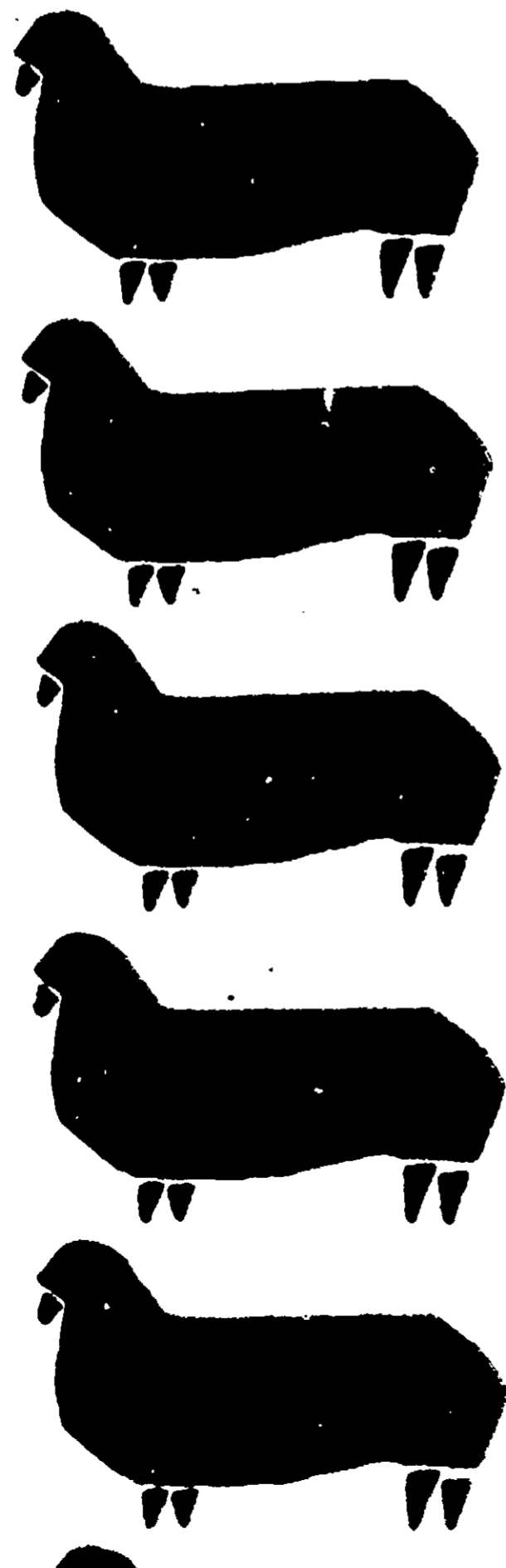

How to prepare and use:

- 1.** Prepare disinfectant solution. Put one-half gallon of water in carton. Add one-half ounce ($\frac{1}{2}$ bottle) of disinfectant concentrate.
- 2.** Wash plastic applicator tubes and rods in disinfectant solution. Allow to stand for 2 or 3 minutes, small ends down.
- 3.** Clean hands, remove one applicator tube and rod from the solution. Handle tube by large end.

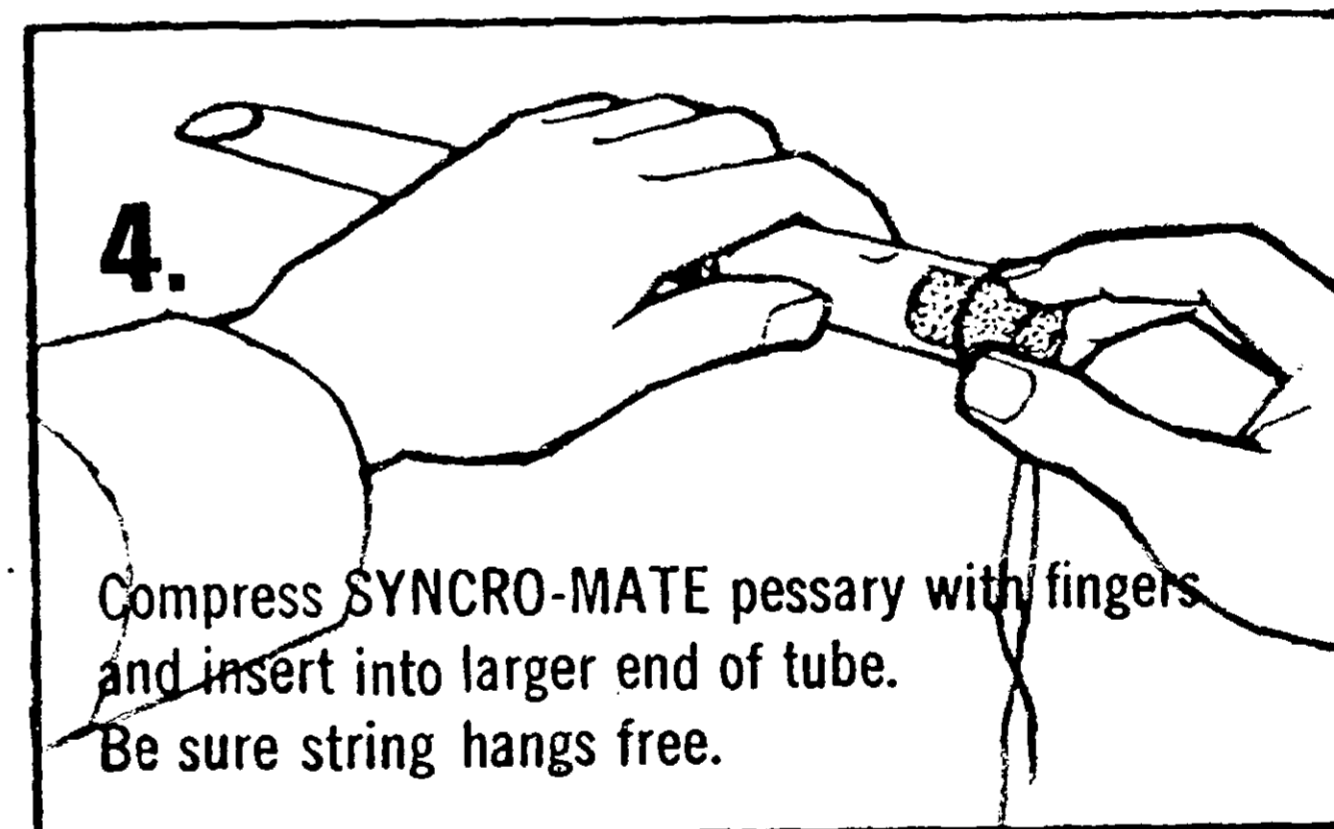

- 5.** Use rod to push pessary through tube toward smaller end until it is just behind beveled opening. Take up slack in string to prevent coiling in tube.

