

EXHIBIT A

PACKAGE LABEL, STERI-AMPS, #400 - 10/22/65

CONTENTS: Each ampul contains:

Active Ingredients:

Methyldodecylbenzyl trimethyl
ammonium chloride 8% ~~w/v~~

Methyldodecylxylylene bis
(trimethyl ammonium chloride) 2% ~~w/v~~

Isopropanol 4.7% ~~w/v~~

Inactive Ingredients: 85.3% ~~w/v~~

CAUTION: Harmful if swallowed. Avoid getting Steri-Amps concentrate in eyes or on skin. If irritation persists, get medical attention. Avoid contamination of food.

Net contents 3 1/3 oz.

Ten 1/3 oz. (10 cc.) Ampuls

No. 400

STERI-AMPS

CONCENTRATED GERMICIDE 10%

INSTRUMENT DISINFECTING ANTI-RUST

GENERAL SANITIZING NON-DULLING

CAUTION: KEEP OUT OF REACH OF

CHILDREN.

CAUTION: NOT FOR INJECTION - FOR EXTERNAL

USE ONLY.

SAVAGE Laboratories, inc.

Houston, Texas

Reg. #3951-1

DIRECTIONS: For instruments, gloves, thermometers, masks, etc., add one 1/3 oz. (10 cc.) Steri-Amps ampul to one quart of water (approx. 1,000 ppm.). Clean and rinse object before putting it into the solution. Allow instruments to remain in the solution at least 10 minutes.

NOTE: Do not introduce soap or detergents into the solution. Use freely on surfaces but not in conjunction with soap or detergents.

Do not store in direct sunlight.

R. 10/65

P. 10/65

