

DANGER (Concentrate):

Keep Out of Reach of Children. Corrosive, the concentrate may cause eye damage or skin irritation. Do not get in eyes, on skin or on clothing. Harmful if swallowed. Avoid contamination of food. Protect eyes and skin when handling concentrate.


Rinse empty container thoroughly with water and discard it.

FIRST AID:

Immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. If swallowed, drink large quantities of water. Call a physician.

NOTE TO PHYSICIAN:

Probable mucosal damage from ingestion of concentrate may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

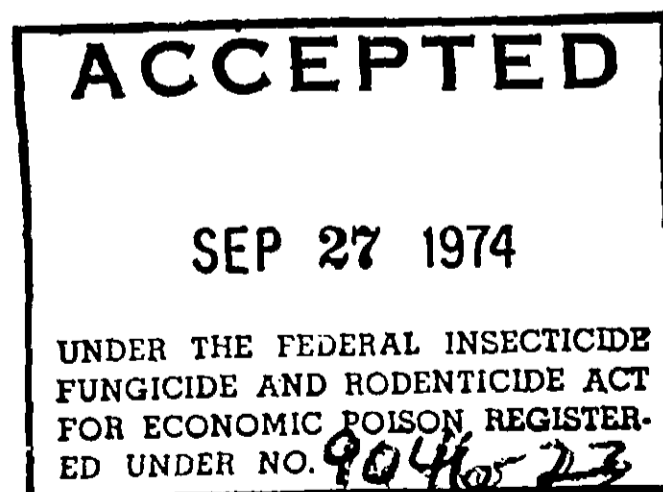

Manufactured by
CONSAN PACIFIC INC., Whittier, CA 90606

CONTROL III™ detergent GERMICIDE

by Sterilaire, Inc.

**Rapid Action — On Inanimate
Hard Surfaces**

ACTIVE INGREDIENTS:

n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈)	
dimethyl benzyl ammonium chlorides	4.5%
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides	4.5%
Tetrasodium ethylenediamine tetraacetate	2.0%
Sodium Carbonate	4.0%

INERT INGREDIENTS:

.....	85.0%
	<hr/>
	100.0%

**ON INANIMATE ENVIRONMENTAL
SURFACES EFFECTIVE AGAINST**

Pseudomonas aeruginosa	Adenovirus type 2
Staphylococcus aureus	Vaccinia viruses
Salmonella choleraesuis	Trichophyton interdigitate
Escherichia coli	Influenza A virus
	Herpes simplex virus

DANGER

KEEP OUT OF REACH OF CHILDREN

This product is a concentrate and must be diluted before use.

**SEE LEFT PANEL FOR ADDITIONAL
PRECAUTIONARY AND FIRST AID
STATEMENTS**

One Quart (32 fluid ounces) EPA Reg. #9046-23

Exclusive International Distributor — **STERILAIRE, INC.**
1807 N. Broadway, Santa Ana, CA 92706

**USED FOR ENVIRONMENTAL
CLEANING AND DISINFECTION:**

Floors, Walls, Equipment
Operating Rooms
Intensive Care Rooms
Patient Care Rooms
Kitchen and Laundry Areas (rinse all food contact surfaces with potable water before re-use).
Heat Sensitive Instruments (before cold disinfection).
Ultrasonic Cleaning

DIRECTIONS FOR USE:

General:

1. Remove heavy soil as required.
2. Apply CONTROL III Detergent as directed to surfaces with cloth, mop or sponge.
3. Will not harm surfaces which are safe in water.

**Environmental Housekeeping Germicide
Cleaner:**

Mix one ounce CONTROL III Detergent in one gallon of water. Resulting mixture contains 700 ppm active ingredients.

Terminal Cleaning in Patient Care Rooms:

Mix one ounce CONTROL III Detergent in one gallon of water. Apply to bed structures and all patient-contact areas.

Heat Sensitive Instrument Cleaning:

Mix one ounce CONTROL III Detergent in one gallon of water. Clean instruments with CONTROL III Detergent prior to cold disinfection. Immerse instruments in CONTROL III Disinfectant solution for 10 minutes or more.