USED FOR:

Disinfection of walls, equipment, heat sensitive instruments and tubing in operating rooms, recovery rooms and intensive care rooms.

DANGER—CORROSIVE (CONCENTRATE):

Keep Out of Reach of Children. The concentrate causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Harmful if swallowed. Avoid contamination of food. Protect eyes and skin when handling concentrate.

Rinse empty container thoroughly with water and discard it.


FIRST AID:

Immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. If swallowed, drink large quantities of water. Call a physician.

NOTE TO PHYSICIAN:

Probable mucosal damage from ingestion of concentrate may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

Manufactured by CONSAN PACIFIC INC., Whittier, CA 90606.

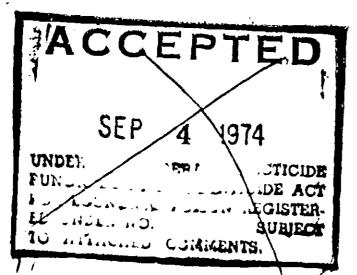

CONTROL III disinfectant

GERMICIDE

by ∫terilaire, Inc.

Rapid Action - Long Life -

ACTIVE INGREDIENTS

INERT INGREDIENTS 80%

LD₅₀ (concentrate) is 3625.

ON INANIMATE ENVIRONMENTAL SURFACES EFFECTIVE AGAINST

Pseudomonas aeruginosa Staphylococcus aureus Salmonella choleraesuis Escherichia coli Adenovirus type 5 Vaccinia viruses Influenza A₂ virus Herpes simplex virus

DANGER—KEEP OUT OF REACH OF CHILDREN

This product is a concentrate and must be diluted before use.

SEE FIRST AID AND OTHER PRECAUTIONS ON LEFT PANEL

One Quart (32 fluid ounces)

EPA Reg. #9046-24

Exclusive International Distributor — STERILAIRE, INC. 1807 N. Broadway, Santa Ana, CA 92706

DIRECTIONS FOR USE:

General: I. Remove heavy soil as required.

- 2. Apply CONTROL III DISINFECT-ANT as directed.
- 3. Will not harm surfaces which are safe in water.

OPERATING ROOM GERMICIDE:

Mix 1/2 ounce CONTROL III DISIN-FECTANT in one gallon of water.

Resulting mixture contains 780 ppm active ingredients.

Apply to surfaces of fixtures and equipment in operating room. Allow to air dry.

COLD DISINFECTANT:

Clean instruments with CONTROL III **Detergent.** Disinfect as follows: Mix one ounce CONTROL III **Disinfectant** in one gallon of water. Immerse instruments for 10 minutes or more. Active concentration is 1560 ppm.

INSTRUMENT STORAGE:

Mix one ounce CONTROL III DISIN-FECTANT in one gallon of water. Add 6 gms. sodium carbonate per gallon of solution. Solution can be retained for one month.

FOOD PROCESSING EQUIPMENT AND DISHES:

Add one ounce CONTROL III DISIN-FECTANT to 8 gallons of water. Apply as spray or flush for final rinse after normal cleaning. At this level **NO POTABLE RINSE** is needed.


LAUNDRY:

Use 1/2 ounce CONTROL per 25 lbs. dry weight of laundry in the final rinse.

ACCEPTED

SEP 4 1974

UNDER THE FEDERAL INSECTICIDE FUNGICIDE AND RODENTICIDE ACT FOR ECONOMIC POISON REGISTERED UNDER NO. 7046-24 SUBJECT TO ATTACHED COMMENTS.

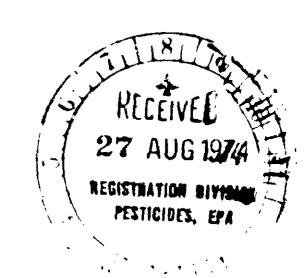

Effective Against:

Pseudomonas aeruginosa (PRD-10 Staphylococcus aureus (ATCC 4574) Salmonelia choleraesuis (ATC 10708)

***Influenza** A₂ virus

*Herpes simplex

*Adenovirus type #5

X Vaccinia viruses

Trichophyton interdigitale

UNDER THE FEDERAL INSECTICIDE FUNGICIDE AND RODENTICIDE ACT

FOR ECONOMIC POISON REGISTER-

Note: Bacteria are killed within ten minutes.

Warning:

Keep Out of Reach of Children. Avoid getting in eyes. Avoid contamination of food. May be harmful if swallowed. Rinse empty container thoroughly with water and discard it.

First Aid:

Immediately flush eyes with plenty of water. If irritation persists, obtain medical attention. If swallowed, drink large quantities of water.

Manufactured by CONSAN PACIFIC INC., Whittier, Ca. 90602

ENTROL III

LABORATORY GERMICIDE

by Sterilaire, Inc.

READY TO USE RAPID ACTION

Bactericidal Fungicidal

Requires No Activators • Contents Are Ready For Use

ACTIVE INGREDIENTS — n-alkyl (60% C_{14} , 30% C_{16} , 5% C_{18} , 5% C_{12}) dimethyl benzyl ammonium chloride0.0781% n-alkyl (68% C_{12} , 32% C_{14}) dimethyl ethyl benzyl ammonium chloride0.0781%

100.000%

* Virucidal

SEE FIRST AID and other precautions on back panel

WARNING — Keep Out Of The Reach Of Children

One Gallon (128 fl. oz.)

EPA Reg. No. 9046-25 E.P.A. Est. No. 9046-CA-1

STERILAIRE, INC. • Santa Ana, California 92706 1807 N. Broadway, Santa Ana, Ca. 92706

DIRECTIONS FOR

General: 1. USE

> 2. Will 3. Befo

of C ed.

tani 4. CO1

doe 5. Solu with

Operating Roon

Apply solution gery to Allow

Surgical Instru

Pre-cle Germic thorough sterile (Fill tra Germic

for ten

Surgical Instru

Place i contair Germic contair tained freque