

ACCEPTED
 April 3, 1975
 THE FEDERAL INSECTICIDE
 AND RODENTICIDE ACT
 ECONOMIC POISON REGISTER-
 DER NO. 9046-25
 minutes.

CONTROL III™

LABORATORY GERMICIDE

by Sterilaire, Inc.

RAPID ACTION — READY TO USE

Bactericidal • Fungicidal • *Virucidal

Requires No Activators • Contents Are Ready For Use

ACTIVE INGREDIENTS — n-alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₈ , 5% C ₁₂)	
dimethyl benzyl ammonium chloride	0.0781%
n-alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethyl	
benzyl ammonium chloride	0.0781%
INERT INGREDIENTS	99.8438%
	100.000%

SEE FIRST AID and other precautions on back panel

WARNING — Keep Out Of The Reach Of Children

One Gallon (128 fl. oz.)

EPA Reg. No. 9046-25
 E.P.A. Est. No. 9046-CA-1

STERILAIRE, INC. • Santa Ana, California 92706
1807 N. Broadway, Santa Ana, Ca. 92706

90602

DIRECTIONS FOR USE:

General:

1. **USE FULL STRENGTH.** Do not dilute.
2. Will not harm surfaces.
3. Before disinfecting, clean off all contaminants. Use of CONTROL III Detergent Germicide is recommended. Rinse well before applying Laboratory Disinfectant/Germicide solution.
4. CONTROL III Laboratory Disinfectant/Germicide does not affect cements.
5. Solution can remain in this container indefinitely without loss of effectiveness.

Operating Room Germicide:

Apply CONTROL III Laboratory Disinfectant/Germicide solution to a clean sponge or cloth. Wipe down surgery table, instrument stands, and other equipment. Allow to dry in air.

Surgical Instruments:

Pre-clean instruments with CONTROL III Detergent Germicide (one ounce per gallon of water). Rinse thoroughly in running water. Place instruments in a sterile auto-clavable plastic tray or other suitable tray. Fill tray with CONTROL III Laboratory Disinfectant/Germicide solution to fully cover all instruments. Soak for ten minutes or longer for disinfection.

Surgical Instrument Storage:

Place instruments in clean storage tray container. Fill container with CONTROL III Laboratory Disinfectant/Germicide solution to fully immerse instruments. Cover container. In normal use storage solution can be retained for thirty days. Replace solution more often if frequent instrument removals and additions are made.