

FOR ECONOMIC POISON REGISTERED UNDER NO. 9156-3

NDC 0009-0753-02

One Pint No. 2332

Pentacresol[®] Instrument Disinfecting Solution

Upjohn

**For Cold Disinfection
of Dental and
Surgical Instruments**

WARNING

Keep out of the reach of children
See additional warnings and
cautions on other panel

Active Ingredients:

Secondary-
amylicresols 0.22% w/w
Isopropyl Alcohol . . . 86.76% w/w
Inert Ingredients 13.02% w/w
(contains sodium nitrite)

The Upjohn Company • Kalamazoo, Michigan 49001, U.S.A.

Warning

Does not destroy bacterial spores

Not a substitute for heat sterilization

Not for medicinal use

Not to be taken internally

Avoid contact with eyes

WARNING - FLAMMABLE!

Keep away from heat or open flame

Discard empty container - Do not reuse

An active disinfecting solution especially prepared for the cold disinfection of dental and surgical instruments which cannot be heat sterilized. Instruments should be thoroughly cleaned, rinsed, and immersed in the undiluted solution for ten to fifteen minutes. The vessel should be kept covered to prevent evaporation.

Caution: After immersion in this solution, surgical instruments, particularly those for ophthalmic or urological use, should be rinsed thoroughly with sterile distilled water. Care should be used in disinfecting rubber and plastic instruments as some types are damaged by the solution.

EPA Reg. No. 9156-3-AA

EPA Est. No. 1023-MI-1

Store at room temperature

Expires Feb. 1, 1961

753A2L

Lot 324E