

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY AND POLLUTION PREVENTION

March 4, 2015

Cristina Griffin Agent for PDI Delta Analytical Corporation 12510 Prosperity Drive, Suite 160 Silver Spring, MD 20904

Subject: PRIA Label Amendment – Revision to the Signal Word and Precautionary

Statement

Product Name: Sani-Cloth Germicidal Wipes

EPA Registration Number: 9480-4 Application Date: October 21, 2014

Decision Number: 496542

Dear Ms. Griffin:

The amended label referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable. This approval does not affect any conditions that were previously imposed on this registration. You continue to be subject to existing conditions on your registration and any deadlines connected with them.

A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. You must submit one copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. "To distribute or sell" is defined under FIFRA section 2(gg) and its implementing regulation at 40 CFR 152.3.

Should you wish to add/retain a reference to the company's website on your label, then please be aware that the website becomes labeling under the Federal Insecticide Fungicide and Rodenticide Act and is subject to review by the Agency. If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) list examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product's label, claims made on the website may not substantially differ from those claims approved through the registration process. Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA's Office of Enforcement and Compliance.

Page 2 of 2 EPA Reg. No. 9480-4 Decision No. 496542

Your release for shipment of the product constitutes acceptance of these conditions. If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6. If you have any questions, please contact Terria Northern by phone at 703-347-0265, or via email at northern.terria@epa.gov.

Sincerely, Elizabeth K Northing for

Elizabeth Watkins

Acting Product Manager 33 Regulatory Management Branch I Antimicrobials Division (7510P) Office of Pesticide Programs

Enclosure: Stamped Label

ACCEPTED

03/04/2015

Under the Federal Insecticide, Fungicide and Rodenticide Act as amended, for the pesticide registered under EPA Reg. No. 2420

9480-4

SANI-CLOTH® GERMICIDAL WIPES GERMICIDAL DISPOSABLE CLOTH

ACTIVE INGREDIENTS:	
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides	. 0.25%
n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5%C ₁₂ , 5%C ₁₈) dimethyl benzyl ammonium chlorides	. 0.25%
Isopropyl Alcohol	55.00%
OTHER INGREDIENTS	44.50%
TOTAL	100.00%
[Does not include the weight of the [cloth] [towelette] [or] [wipe]]	

KEEP OUT OF REACH OF CHILDREN WARNING

See back [side] panel for additional precautionary statements

NET CONTENTS [and/or Net V	/t.]oz (g) [<i>or</i> lbs. (kg)
# OF CLOTHS	[SIZE of each cloth]

EPA Reg. No.: 9480-4 EPA Est. No.: 9480-NY-1

or

EPA Est. No. A=9480-NY-1, B=72956-WI-1, C=72956-AR-1 [or others] Alpha character will precede batch code on product

[Manufactured by [or for]]:
Professional Disposables International, Inc.
Two Nice-Pak Park, Orangeburg, NY 10962-1376
[For information call: [phone number to be inserted]]
[Made in USA]

First Aid:

Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment

- -Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. If on skin:
- -Take off contaminated clothing.
- -Rinse skin immediately with plenty of water for 15-20 minutes.

[For additional information in case of emergency call toll free [phone number to be inserted]].

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 1 of 15

Precautionary Statements

Hazards to Humans and Domestic Animals

Warning. Causes substantial but temporary eye damage. Do not get in eyes or on clothing. Avoid contact with skin. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using restroom. Remove and wash contaminated clothing before reuse.

Physical or chemical hazard

Combustible. Do not use or store near heat or open flame. Do not use on natural marble, windows, unpainted wood, brass, clear plastic or colored grout.

[Use one of the following as applicable to type of container.]

[To Dispense TOWELETTES] [or WIPES] [or TOWELS] [or CLOTHS]:

Remove lid [and discard inner seal] [from container *or* canister.] Find center of wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the [container *or*] canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep [center cap of] lid closed to prevent moisture loss.

TRAVEL PACK:

Peel back label slowly. Pull out wipes as needed and reseal label. Unfold wipe.

BOX:


Open along perforations as indicated. Remove packets as needed.

PACKET/PACKAGE:

Tear open packet [package]. Unfold wipe and use as directed. Discard after single use.

SOLO® PAK:

Open package lid. Completely remove inner seal. Pull out wipe. (The next wipe pops up automatically.) Snap lid closed after use to retain moisture.

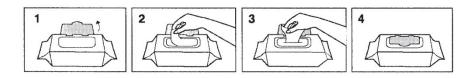

DO NOT FLUSH • STORE AT ROOM TEMPERATURE
DISPOSE OF USED WIPES IN TRASH RECEPTACLE • KEEP SOLO®
LID CLOSED WHEN NOT IN USE

SOLO® is a registered trademark of Nice-Pak Products, Inc.

REFILL PACKET

This refill package is made for use only in a [product name] [container or] canister; do not use this product in a [container or] canister for another product. Do not use the refill wipes in a container that is cracked or broken. Discard the cracked or broken container in accordance with the container disposal instructions on the label.

[Option 1]

Remove [entire] lid from [container *or*] canister. Tear open refill [package] at [notch] [mark] [arrow] and place [wipe] roll in empty [container *or*] canister. Follow information on [container *or*] canister to dispense wipes.

[Option 2]

Remove [entire] lid from [container *or*] canister. Tear open refill [package] at [notch] [mark] [arrow]. Place [entire] refill package of wipes in empty [container *or*] canister. Follow information on [container *or*] canister to dispense wipes.

GENERIC:

Open lid. Pull out wipe. Close lid tightly.

Bucket [or pail or container][numbers optional]

- 1. Remove the lid from the [pail or bucket or container.]
- 2. [Press] open the dispenser at center of the lid.
- 3. Open the refill pack then place in [pail or bucket or container].
- 4. Locate the first wipe at the center of the roll.
- 5. Feed the first wipe though the dispensing orifice.
- 6. Reapply the lid to the [pail or bucket or container.]
- 7. Dispense the wipe.

For the refills:

- 1. Remove the lid from the [pail *or* bucket *or* container.]
- 2. Remove the empty refill pack material from the [pail or bucket or container.]
- 3. [Press] open the dispenser at center of the lid.
- 4. Open the refill pack then place in [pail or bucket or container].
- 5. Locate the first wipe at the center of the roll.
- 6. Feed the first wipe though the dispensing orifice.
- 7. Reapply the lid to the pail.
- 8. Dispense the wipe.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

TO DISINFECT [AND DEODORIZE]: To disinfect nonfood contact surfaces only: Use a wipe to remove heavy soil as needed. Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for a full two (2) minutes. [Use additional wipe(s) if needed to assure continuous two (2) minute wet contact time.] Let air dry. [When TB is included on the label, add the following or use the TB directions below:] These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at 68°F (20°C).

OR

TO DISINFECT [AND DEODORIZE]: To disinfect nonfood contact surfaces only: Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for a full two (2) minutes. Let air dry. For heavily soiled surfaces, use a wipe to pre-clean prior to disinfecting. [When TB is included on the label, add the following or use the TB directions below:] These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at 68°F (20°C).

[The following sets of directions are optional and for hospital/medical facility use only: Tuberculosis (TB) [Outbreaks]: This product is effective against *Mycobacterium bovis* BCG (Tuberculosis) in one minute at 68°F (20°C). Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for one (1) full minute. Let air dry. For heavily soiled surfaces, use a wipe to pre-clean prior to disinfecting Although efficacy at a one (1) minute contact time has been shown to be adequate against TB, this time is not sufficient for other organisms on the label. Therefore a 2 minute wet contact time must be used for all listed non TB organisms. and/or

Ultrasound transducers and probes: For use on external surfaces of ultrasound transducers and probes only.

To disinfect: Pre-clean gel from external surface of ultrasound transducer or probe to be disinfected by using a clean washcloth. Then thoroughly wet the probe with a wipe. Allow treated surface to remain wet for a full two (2) minutes. Let air dry.]

[†]SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS

Personal protection: When using this product, wear disposable protective gloves, protective gowns, masks and eye coverings when handling HIV-1 (AIDS Virus), HBV or HCV infected blood or body fluids.

Cleaning procedure: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal wipe. Open, unfold and use first germicidal wipe to remove heavy soil.

Disposal of infectious materials: Used wipe, blood and other body fluids should be disposed of according to local regulation for infectious waste disposal.

Contact time: Use second germicidal wipe to thoroughly wet surface. Allow to remain wet two (2) minutes, let air dry.

[Use storage/disposal statement A if canister, tub, bucket, pail or refill dispenser directions used. Use storage/disposal statement B if travel pack, Solo® Pak, soft pack, pouch, or [refill or [bulk]] packet/package] directions used.]

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal. [Statement not required for residential or household use products.]

[Statement A:]

Storage: Do not store near heat or open flame. When not in use keep center cap of lid closed to prevent moisture loss. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Dispenser [or Container] [or Package] Disposal:** Nonrefillable container. Do not reuse or refill this container. Offer for recycling. If recycling is not available, put in trash collection. [If refillable dispenser is being used, substitute the following: **Refill Dispenser [or Container] [or Package] Disposal:** Offer for recycling cracked or broken containers. If recycling is not available, discard in trash.]

[Statement B:]

Storage: Do not store near heat or open flame. **Towelette Disposal**: Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Dispenser [or Container] [or Package] Disposal**: Put empty travel pack [or Solo® Pak or [refill] packet [or packaging]] in trash collection.

Areas of Use:

Hospital and Healthcare Settings: Ambulances, Ambulatory Surgical Centers (ASC), Anesthesia, CAT Labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctors Offices, Donation Centers [blood][plasma][semen][milk][apheresis], Emergency Medical Settings, Emergency Rooms, E.R., Emergency Vehicles, Eye Surgical Centers, Healthcare Settings, Health Care Facilities, Home Health Care, Hospices, Hospitals, ICU, Isolation Areas, Laboratory, Laundry rooms, Long Term Care Centers, Neonatal Intensive Care Units (NICU), Newborn nursery, Nursing homes, Operating Rooms (OR), Ophthalmic Offices, Orthopedics, Out-Patient Surgical Centers (OPSC), Patient Care Areas, Pediatrics, Pediatric Intensive Care Units (PICU), Physical therapy, Physician's offices, Radiology, Recovery Rooms, Rehabilitation, Respiratory therapy, Skilled Nursing Facility, Surgery, Surgery Intensive Care Unit (SICU), Surgical Centers, Transport vehicles and X-Ray.

Critical Care Areas: CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU), Surgery and Surgical Intensive Care Unit (SICU).

and

Hospital, Healthcare, and Critical Care Use Sites: May be used on hard non-porous surfaces of: ambulance equipment surfaces; bed railings; bladder scan; blood glucose meters; cabinets; cardiac gym equipment; carts; chairs; counters; defibrillators; dental chairs; dental countertops; dental unit instrument trays; dopplers; electronic thermometers; endodontic equipment such as apex locators; exam tables; gurneys; infant incubators [interior and exterior surfaces of]; infant warmers [interior and exterior surfaces of]; isolettes; IV poles; IV pumps; laboratory equipment and surfaces; loupes; operating room tables and lights; operatory light switches; Ophthalmoscopes; Otoscopes; oxygen hoods; physical therapy (PT) equipment surfaces; patient lift equipment; pulse oximeter; radiology equipment; slit lamps; spine back boards; stethoscopes; stretchers; stools; tables; telemetry equipment; telephones; toilet seats; tympanic thermometers; and hard non-porous outside surfaces of: amalgamators and dental curing lights; anesthesia machines and respiratory therapy equipment; apheresis machines, diagnostic equipment, dialysis machines; patient monitoring equipment, patient support and delivery equipment, pulp testers and motors; toilets; ultrasound monitors; and ultrasound transducers and probes.

Grocery Stores *or* **Shopping Centers** *or* **Supermarkets** *or* **Delicatessens:** Disinfects grocery cart handles and grocery cart child seats. Disinfects shopping cart handles and child seats.

Nursery or Daycare Centers: Bassinets [interior and exterior surfaces of], changing tables, cribs, diaper changing stations, diaper pails, infant/child care equipment surfaces, trash cans, toys and other hard, non-porous surfaces listed on this label.

Bathroom[s] or Public Restrooms: Basins, [bath]tubs, bathroom fixtures, cabinets, countertops, doorknobs, faucet[s], handles, hampers, handrails, shower[s] [stalls], showers, sinks, glazed tiled walls, glazed tiles, toilet [handle] [rim] [seats] [tank] [and other exterior surfaces of], urinals [exterior surfaces of] and vanity tops, and other hard non-porous surfaces listed on this label.

Health club facilities and Gyms: Gym equipment, weight-lifting surfaces and other hard non-porous surfaces listed on this label.

Schools or Classrooms or Churches or Office Buildings or Public Facilities or Manufacturing Facilities or Libraries or Public Telephones: Cabinets, chairs, computers, computer monitors, computer mouses, counters, countertops, desks, desktops, doorknobs, fax machines, filing cabinets, filing doors, filing handles, garbage cans, headsets, keyboards, railings, seats, sinks, tables, tabletops, telephones, toys, trash cans, work stations, and other hard non-porous surfaces listed on this label.

Transport vehicles, school busses, public busses, fire trucks, police cars: Hard non porous surfaces of vehicles including steering wheel, hard plastic or metal flooring, seats, and walls.

Restaurants: Hard non porous, non food contact surfaces only including chairs, display cabinets, garbage cans, booster seats, floor, sinks, bathroom surfaces, glazed tiled walls, telephones, cash registers, and doorknobs.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection

[Symbols including asterisks, bullets or footnotes and similar may be used to designate categories of organisms.]

[ATCC numbers are not required to be listed on production label. Organisms may be listed in a table and in any order.]

Bacteria [(2 Minute Contact)] Contact)]

Bordetella pertussis [ATCC 12743] Burkholderia cepacia [ATCC 25416] ATCC 35743]

Campylobacter jejuni [ATCC 29428] Escherichia coli (E. coli) [ATCC 11229] Escherichia coli O157:H7 [ATCC 35150]

Contact)]

Klebsiella pneumoniae [ATCC 4352] Pseudomonas aeruginosa [ATCC 15442] Salmonella enterica [ATCC 10708] Staphylococcus aureus [ATCC 6538]

TB [Tuberculocidal] [(1 Minute

Mycobacterium bovis BCG (Tuberculosis)

Pathogenic Fungi [(2 Minute

Candida albicans [ATCC 14053]

Multi-Drug Resistant Bacteria [(2 Minute Contact)]

Acinetobacter baumannii [ATCC 19606]

Enterobacter cloacae NDM-1 positive [CDC 1000654]

ESBL Producing Escherichia coli (E. coli) [ATCC BAA-196]

Klebsiella pneumoniae – KPC-2 positive, ST258 (multi-drug and carbapenem resistant) [CDC 2008030][Effective against organism resistant to Ertapenem, Ampicillin, Ampicillin/Sulbactam, Cefazolin, Cefepime, Ceftazidime, Ceftriaxone, Ciprofloxacin, Gentamicin, Imipenem, Levofloxacin,

Piperacillin/Tazo, Tobramycin, Trimethoprim/Sulfa Meropenem and Intermediate resistant to Amikacin.] Methicillin Resistant Staphylococcus aureus (MRSA) [ATCC 33592]

Vancomycin Resistant Enterococcus faecalis (VRE) [ATCC 51299]

Viruses* [(2 Minute Contact)]

- *Adenovirus Type 5 [ATCC VR-5]
- *Influenza A/Hong Kong [ATCC VR-544]
- *Influenza A (H1N1) virus [ATCC VR-98] [Strain A/Malaya/302/54]
- *Herpes Simplex Virus Type 2 [ATCC VR-734]
- *Human Coronavirus [ATCC VR-740] Strain 229E
- *Respiratory Syncytial Virus [RSV]
- *Rhinovirus [ATCC VR-1110]
- *Rotavirus Strain WA
- *Vaccinia virus [ATCC VR-1354]

Bloodborne Pathogens[(2 Minute Contact)]

- *Hepatitis B virus (HBV) Duck HBV[†] [Strain 7/31/07]
- *Hepatitis C virus (Human) (HCV) Bovine Diarrhea Virus[†] [Strain Oregon C24v-genotype 1]
- *HIV-1 (AIDS virus) TStrain HTLV-IIIB

[Effective against all listed organisms in the presence of organic soil (5% blood serum)]

[*][Product name] is virucidal against Adenovirus Type 5, Herpes Simplex Type 2 (causative agent of genital herpes), Influenza A/Hong Kong (representative of the common flu virus), Influenza A (H1N1) virus, Human Coronavirus, Respiratory Syncytial Virus [RSV], Rhinovirus, Rotavirus, and Vaccinia virus in the presence of organic soil (5% blood serum).

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 7 of 15

[Optional: Choose one of the following statements:]

[Respiratory illnesses attributable to Pandemic 2009 H1N1 are caused by influenza A virus. This product is a broad-spectrum hard surface disinfectant that has been shown to be effective against (influenza A virus tested and listed on the label) and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).

This product has demonstrated effectiveness against influenza A virus and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 influenza A virus.

This product has demonstrated effectiveness against (influenza A virus tested and listed on the label) and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).

Kills Pandemic 2009 H1N1 influenza A virus (formerly called swine flu).

Kills Pandemic 2009 H1N1 influenza A virus.]

TKILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON THE PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS AND PROBES PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in two (2) minutes at room temperature (68°- 77°F) or (20°- 25°C) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

[[Product name] is a premoistened, non-woven durable wipe containing a quaternary/alcohol based solution. Recommended for use in hospitals and other critical care areas where the control of the hazards of cross-contamination between treated surfaces is required. [Suitable for use on equipment requiring alcohol-based products.] Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica® and glass.

<u>Optional Claims and Descriptors</u> [Note: Deodorizing directions must be used when deodorizing claims (claims involving odor control) are used]

X" x Y" [wipes]

___% bigger [[quilted] wipes] [vs. X" x Y" wipes]

___# pre-saturated towelettes X" x Y"

© [insert year] [insert company name]

[Absorbs] [Absorbent] [wipe]

After use, just toss wipe(s) away

Always ready to use

An effective method to deodorize and disinfect against odor-causing organisms on hard non-porous surfaces listed on this label.

An effective way to disinfect hard, non-porous environmental surfaces listed on this label.

A non-woven disposable cloth for use in hospitals and other critical care areas where control between treated surfaces is required.

Antibacterial

Antimicrobial

[Antibacterial] [Antimicrobial] health club equipment wipe

A Quaternary [ammonium chloride] solution impregnated in a wiping cloth

Are pre-measured to have the recommended amount of germicidal solution, wipe after wipe.

A Sani-Surface® Brand

Assures proper dilution strength, product effectiveness and standardized technique.

[Bactericidal], [Tuberculocidal], [Fungicidal] and [Virucidal*] in 2 [two] minutes

Bleach-free [formula]

Can help reduce the risk of cross contamination on treated surfaces

Cellulosic non-woven disposable cloth

[Cloth-like texture] [Textured] [strong] [for] [with] scrubbing power

Compatible for use on hard, nonporous surfaces commonly found in healthcare settings listed on this label

Contains [name of active ingredient], the [same] active ingredient used in [Brand-name]

Contains [number] canisters [plus] [and] [number] refills

Convenient

Deodorize hard, non-porous surfaces [Use to]

Deodorizes, and kills common household germs⁺

Diamond-embossed wipe is low-linting [for sensitive equipment surfaces]

Disinfect external surfaces of ultrasound transducers

Disinfect the pre-cleaned surface

Disinfecting towelettes

Disinfects [and deodorizes] in 2 [Two] minutes

Disinfects grocery cart handles *or* shopping cart handles *or* grocery cart child seats *or* shopping cart child seats

Disinfects hard non-porous environmental surfaces listed on this label [Use to]

Disinfects hard, non-porous surface in 2 minutes

[Disinfects] hard surfaces [that people touch]

Disinfects [health club equipment] [treadmill handles][elliptical handles][exercise bike handles][weight benches]

Disinfects [in [just] two [2] minutes]

Disinfects on the go

Dispenses one wipe at a time

Does not contain bleach

Does not produce mist or overspray

Double-sided scrubbing wipes

Durable, low-lint wipe designed for sensitive equipment surfaces

Easy

Easy [and convenient] way to [touch-up] [bathroom] [office] everyday

[Easy] [convenient] [dispensing] lid [closure] [cap] [package]

Easy drop-in refill[s]

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 9 of 15

[Easy] pop-up-tub

Easy start lid [package]

Easy to use [easy to refill]

Easy to use package [canister] [container] keeps [your] wipes moist

[Economical] Refill[s] [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] -pack

[Economical] Refills [now] available

Economy

Effective against bathroom bacteria and viruses* [use at least two bacteria and two viruses from the organism list]

Effective against [the following micro-organisms on hard, non-porous surfaces] [list organisms]

Effective in the presence of 5% blood serum contamination

Eliminates odors

Eliminates potential for overspray

Extra large [disinfecting] [Y" x Z" compared to our A" x B" size] wipes

Fast [Convenient] [Portable] way to disinfect hard non-porous surfaces

For daily use

Gets rid of -or- eliminates odors

[Helps] [kill][s] [eliminate][s] odor-causing bacteria

Helps prevent cross contamination on treated surfaces

Hospital disinfectant [disinfection] [use only when hospital use sites on label]

Includes ___ count refill [pack] [contains] ___ wipes

Just –or- simply wipe exterior toilet surface –or- bathroom germs⁺ away

Just [pull] wipe and toss

Kill germs⁺ and bacteria instead of germy paper towels, sponges and rags [Use this product to]

Kills bacteria and viruses* in the bathroom [use at least two bacteria and two viruses from the organism list:]—or- on the exterior toilet surfaces

Kills bacteria in 2 minutes

[Kills] [Effective against] [MRSA] [(Methicilin Resistant Staphylococcus aureus)]

Kills household –or- exterior toilet surface –or- bathroom germs⁺ [use at least two bacteria and two viruses from the organism list]

Kills odor causing bacteria

[Kills] [Effective against] TB in 1 [One] minute

Leaves [bathroom] [office] disinfected [and smelling fresh]

Less [reduced] packaging [less to throw away]

Moisture lock lid

Multi-pack

Multi-purpose disinfectant [Multi [-refill] pack]

New larger, thicker wipe [with diamond-embossed design]

New thicker wipe with diamond-embossed design

Next wipe pops up automatically [everytime] [!]

No canister needed

No measuring

No mixing

No [pinched fingers] [rips] [tears]

No sponges, no rags, no buckets, no mess

No threading [required]

Non-scratching [wipe]

Non-woven disposable cloth

Not for in-home use

Not for sale in [insert name of state(s)]

Not for sale [or use] after [insert date]

Odor [elimination] [neutralization]

One [wipe] at a time dispensing

On-the-go disinfectant

[Package][insert type of package] made from [x%] recycled materials

Perfect for disinfecting exterior toilet surfaces -or- around the exterior toilet surface

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 10 of 15

Picks up [particles] [dust] [dirt] [hair] [crumbs]

Pop-up [dispensing] [top]

Portable

[Premixed], [premeasured], [premoistened]

Pull n/ Go [pack]

[Quickly] eliminates [lingering] [food] odors on surfaces

Ready to Use

Recyclable [package]

Reduce [risk of cross contamination] [on hard non-porous surfaces]

Reduces [packaging] waste [by x%] [when compared to regular canister or bucket or pail]

[Refill] package contains [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] [wet] [pre-moistened] [wipe] [refill[s]] [roll[s]]

Refill [size] [pack] [value pack]

Refillable [with ____# wipes] [dispenser] refill [pack] [for dispenser]

Requires no mixing or measuring and are ready-to-use

Resealable [stay-moist] pouch [package] [container] inside

Reuse your [empty] canisters [again [and again] [!]

Sample

Scrubbing power of a sponge in a disinfecting wipe

Strong, durable diamond-embossed textured wipe deodorizes

Strong, durable low-lint wipe [designed for sensitive equipment surfaces]

Suitable for use on equipment requiring alcohol-based products

Tear at perforation

Textured diamond-embossed design [deodorizes hard, non-porous surfaces]

The 2-minute germicidal wipe [disinfectant wipe]

Thick, heavy duty wipes are bactericidal, tuberculocidal and virucidal* in 2 minutes

Thick, large wipe disinfects

Thick, textured wipe with diamond-embossed design

This [outer] [selling] [package] [container][canister] contains x% post consumer [material][plastic][resin]

Time to reorder

Trial

Tub refill

Tub with pop-up action

Twin pack

Use by [insert date]

Use on hard, non-porous surfaces only: [stainless steel] [Formica®] [glass] [glass tables] [glazed tile] [Test wipe on small inconspicuous area first.]

Use to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization / high-level disinfection

[Use to] refill[s] [count] [wipe] canisters or tubs

Value

Versatile

Wet wipes

Where control of the hazards of cross contamination between treated surfaces is of prime importance

Wipe [material] made from [x] % [sustainable] [ingredients] [fibers] [May be used only when cellulose wipe material used]

[Wipes out] [Kills] [eliminates] [list organisms] in [just] [2 minutes]

[Wipes out] [Kills] [the] flu virus* (*Influenza A/Hong Kong)

[Wipes out] [Kills] viruses* and bacteria [including the flu virus* (*Influenza A/Hong Kong)]

[Wiping] [soft] side for disinfecting

With a bleach-free formula

X% [more] [wipes] [free] [vs. Z count] [canisters] [packs]

X% reduction in packaging [material]

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 11 of 15

[Note: The following claims can be used only for 6 months]

[Better] [New] [Improved] [Stronger] [Thicker] Wipe [!]

[Better] [new] [&] [improved] [dispensing] lid [closure] [cap] [package] [dispensing] [better [than before] New and improved

[New] [Package] [packaging] uses [x%] less material (plastic)[then] [when compared to] [regular] containers [or canisters]

New thicker and larger diamond-embossed wipe cleans more surface area

[Now] [Reinforced] [With] Quilted [strength]

Now with [%] more towel

Now with [#] more wipes

[Scrubs] (&) [cleans] [better than ever]

[Stronger] more durable [wipe]

Thick[er] [wipe] [with] cloth-like texture

X% [Bigger] [larger] [quilted] wipes

[x%] thicker wipes

Master Label Reader Notes

[A footnote can be used to list organisms shown on page 2. Bacteria and virus can be separated into footnotes 1 and 2 if desired.]

[When organisms are listed next to a claim, the organisms may be listed in a footnote instead of directly next to the claim]

[* All pesticidal "germ" claims must be linked to bacteria (minimum: Salmonella and Staph) and at least one virus. For example: Front panel: "Kills germs*" Back panel: "Kills Salmonella enterica, Staphylococcus aureus, Herpes Simplex Type 2, and Influenza A/Hong Kong"].

[Graphics depicting use sites listed on this label can appear on product label and labeling.]

[The following graphics and statements are optional. They may be placed anywhere on label/container.]







NOT A BABY WIPE
NOT A SKIN WIPE
NOT FOR USE ON SKIN
FOR USE ON HARD NON POROUS SURFACES ONLY

[The following graphics and statements are optional. They may be placed anywhere on label/container.]





This graphic shows use of the wipe dispenser









The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

October 17, 2014 Page 14 of 15

[The following are company logos for use on PDI product labels. The purpose of the logo is for easy recognition of PDI products in the marketplace.]





F/shared/PDI/9480-4/Labels/2014/9480-4 amend label CLEAN 101714.doc