9480-6

2/11/2004

ae 179

# UNITED STATED

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY

WASHINGTON, D.C. 20460

FEB 1 1 2004

OFFICE OF PREVENTION, PESTICIDES AND TOXIC SUBSTANCES

Ms. Christina Griffin Agent for PDI, The Healthcare Division of Nice-Pak Products, Inc. c/o Delta Analytical Corporation 7910 Woodmont Avenue, Suite 1000 Bethesda, MD 20814

Subject: SaniCloth Plus Germicidal Disposal Cloth EPA Registration Number 9480-6 Your Amendment Dated January 29<sup>th</sup>, 2004 EPA Received Date January 29<sup>th</sup>, 2004

The amendment, submitted in connection with registration under the Federal sec. 3(c)(7)(A), to add the following two organisms: "Methicillin-Resistant Staphylococcus Aureus (MRSA)" and "Vancomycin-Resistant Enterococcus Faccalis(VRE)," is acceptable provided that you:

1. Submit and/or cite all data required for registration/reregistration of your product under FIFRA sec. 3(c)(5) and sec.4 when the Agency requires all registrants of similar products to submit such data.

2. Submit two (2) copies of final printed labeling before you release the product for shipment.

Confirmatory efficacy testing conducted according to Agency standards must be submitted for review and approval before the claim for effectiveness as a disinfectant and deodorizer against Hepatitis B virus can appear on the product labeling.

If these conditions are not complied with the registration will be subject to cancellation in accordance with FIFRA sec. 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the labeling is enclosed for your records.

If you have any questions concerning this letter, please contact Karen M. Leavy-Munk at (703)-308-6237.

Sincerely yours,

Marshall Swindell Product Manager 33 Regulatory Management Branch I Antimicrobial Division(7510C)

## SANI-CLOTH® PLUS GERMICIDAL DISPOSABLE CLOTH

#### ACTIVE INGREDIENTS:

n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides	0.125%
n-Alkyl (60% C14, 30% C16, 5%C12, 5%C18) dimethyl benzyl ammonium chlorides	0.125%
Isopropyl Alcohol	14.850%
OTHER INGREDIENTS	<u>84.900%</u>
	00.000%

#### KEEP OUT OF REACH OF CHILDREN

## CAUTION

See back [side] panel for additional precautionary statements

NET WT

#### [NET CONTENTS # OF CLOTHS SIZE]

[The following registration and establishment number information and registrant address information is mandatory, but may appear in any appropriate place on the label:]

EPA Reg. No.: 9480-6

EPA Est. No. : 9480-NY-1 [or EPA Est. No. A=9480-NY-1, B=9480-IN-1 Alpha character will precede batch code on product] ACCEPTED with COMMENTS in EPA Letter Dated:

FEB 1 1 2004

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No.

9480-6

[Manufactured by:] amended PDI, The Healthcare Division of Nice-Pak Products, Inc. Two Nice-Pak Park 944 Orangeburg, NY 10962-1376 [For information: [phone number to be inserted]] [website address]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towelette(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g \labels\pdi\9480-6afinal 020504label doc

February 5, 2004

Page 1 of 1

#### **PRECAUTIONARY STATEMENTS:**

Hazards to humans and domestic animals.

**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling.

#### FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

#### PHYSICAL OR CHEMICAL HAZARD:

Combustible. Do not use or store near heat or open flame.

[All statements below within the brackets (from here through all of page 3) are optional; choose from the following claims as appropriate, location is optional:]

[For use in: CRITICAL CARE AREAS]

[Where control of the hazards of cross contamination between treated surfaces is of prime importance] [Effective against Staphylococcus aureus, Salmonella choleraesuis, and Pseudomonas aeruginosa in (5) minutes.] [Effective in five (5) minutes against Staphylococcus aureus, Salmonella choleraesuis, and Pseudomonas aeruginosa.]

[Virucidal\*] [Kills bacteria and viruses\*] [Tuberculocidal] [and] [Virucidal\*] [In order to include viruses anywhere on the label, the term "virucidal\*" or "viruses\*" must be used and one or more specific viruses must be listed either separately with an asterisk before each virus <u>or</u> as shown in the asterisked phrases below; if HIV is to be listed, use one of the HIV paragraphs on page 3]

[Exhibits effective virucidal\* activity on hard non-porous inanimate surfaces][:] [\*Kills Influenza A2/Hong Kong (representative of the common flu virus) and Herpes Simplex Type 2 (causative agent of genital herpes)]

[\*Kills viruses including Influenza A2/Hong Kong and Herpes Simplex Type 2.]

[\*Also effective against cold and flu viruses Influenza A2/HK (Hong Kong flu).]

[Kills common household germs: Staphylococcus aureus (Staph) and Salmonella choleraesuis ACCEFTED [An effective way to disinfect hard, non-porous environmental surfaces.] with COMMENTS [For use on external surfaces of Ultrasound Transducers only.] in EPA Letter Dated:

[For use on external surfaces of Ultrasound Transducers only.] [Aids in reducing [or the reduction of] cross contamination between treated surfaces.]

[Convenient] [ Premeasured] [ Premoistened] [Kills Mycobacterium bovis BCG (Tuberculosis)][in five (5) minutes at 20°C.]

[Kills Escherichia coli (E. Coli) 0157:H7][ in five (5) minutes.]

[Bactericidal]

[A dual chain Quaternary/Alcohol solution impregnated in a wiping cloth.]

[Assures proper dilution strength, product effectiveness and standardized technique.] 9480-6[Greater quaternary dispersal with no hazard of cross contamination between treated surfaces]

[Quick drying.] [A dual chain Quaternary/Alcohol solution that contains 2,500 parts per million (ppm) of active ingredient/dual chain quaternary ammonium chlorides and 14.85% alcohol.]

[Non-woven disposable cloth]

[Antibacterial] [Antimicrobial]

[Kills household germs]

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towelette(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g \labels\pdi\9480-6afinal 020504label doc

February 5, 2004

Page 2 of 2

FEB 1 1 2004

Funcicide, and Rodenticide Act as

Under the Federal Insecticide,

amended, for the pesticide, registered under EPA Reg. No.

[\*KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in one (1) minute at room temperature (20°-25°C) in health care or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1).] [\*KILLS HIV-1 ON PRE-CLEANED ULTRASOUND PROBES PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in one (1) minute at room temperature (20°-25°C) in health care or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body field SEPTED and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with bomments potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1).] in EPA Letter Dated: [Bath and Kitchen wipes for hard, non-porous surfaces] FEB 1 1 2004 [Cleans and disinfects] [Cleans, disinfects and deodorizes] [Cleans, deodorizes, and kills common household germs] Under the Federal Insecticide, [No mixina] Functicide, and Rodenticide Act as [No measuring] amended, for the pesticide, [Disinfects hard non-porous environmental surfaces] registered under EPA Reg. No. An effective method to clean, deodorize and sanitize against odor-causing organisms and disinfect hard 9486-6 non-porous surfaces.] [To clean: wipe surfaces and let air dry.] [For non-porous surfaces.] [Ready to Use] [Are pre-measured to have the recommended amount of germicidal solution, wipe after wipe.] [Telephone] [cell phone] [cellular phone] [wireless phone] [mobile phone] [digital phone] [disinfectant wipes] [A disinfectant cloth for use on [telephone] [cell phone] [cellular phone] [wireless phone] [mobile phone] [digital phone] surfaces [and other telecommunications equipment surfaces]] [Cleans and disinfects [telephone] [cell phone] [cellular phone] [wireless phone] [mobile phone] [digital phone] surfaces, [telephone] [cell phone] [cellular phone] [wireless phone] [mobile phone] [digital phone] [dictating equipment surfaces, and other telecommunications equipment] [surfaces] [devices]] [Disinfects headsets, telephones, [cell phones] [cellular phones] [wireless phones] [mobile phones] [digital phonesll [Cleans [grocery carts] [shopping carts]] [Disinfects [grocery cart handles] [shopping cart handles] [grocery cart child seats][shopping cart child seats]] [Kills Methicillin Resistant Staphylococcus (MRSA)] [Kills Vancomycin Resistant Enterococcus (VRE)] [Kills Methicillin Resistant Staphylococcus (MRSA) and Vancomycin Resistant Enterococcus (VRE)] [Effective against Methicillin Resistant Staphylococcus (MRSA)] [Effective against Vancomycin Resistant Enterococcus (VRE)]

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towelette(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

Effective against Methicillin Resistant Staphylococcus (MRSA) and Vancomycin Resistant Enterococcus

g \labels\pdi\9480-6afinal 020504label.doc

(VRE)]

[Choose from the following areas of use as appropriate; they may be listed in any order; at least one use site (including corresponding list of surfaces) must be used on the label; "Kitchen" areas of use must be included if "food contact surfaces" directions (see next page) are used.]

**IAREAS OF USE** 

[General Use

[Kitchen:\*\* ACCEPTED with COMMENTS in EPA Letter Pattificom:

Under the Federal Insertiasencluding: Fungicide, and Rodenticide Act as

FEB 1 1 2004

amended, for the pesticide,

registered under EPA Reg. No.

9480-6 [Nursery:

[faucets,] [table tops] [exterior surfaces of [or outside of] refrigerators,] [exterior surfaces of [or outside of] stove] [Toilet seats,] [bathroom fixtures,] [vanity tops,] [shower stalls,] [bathtubs,] [tiles,] [sinks]

[Counter tops,] [[exterior surfaces of [or outside of] appliance surfaces,] [sinks,]

[Doorknobs,] [telephones,] [laundry rooms,] [garbage cans,] [pet areas] [headsets] [cell phone] [cellular phone] [wireless phone] [mobile phone] [digital phone] surfaces, [dictating equipment surfaces] [and other telecommunications equipment] [surfaces] [devices]] [computers [and keyboards]

[Changing tables,] [cribs,] [diaper pails,] [trash cans,] [toys] [and other] [hard, non-porous surfaces]

[Grocery Stores], and/or [Shopping Centers], and/or [Delicatessens]: Disinfects [grocery cart handles,] [shopping cart handles,] [grocery cart child seats,] [shopping cart child seats] [and other hard non-porous surfaces] [Cleans [grocery carts] [shopping carts]

[The following is required if kitchen uses are included: \*\*A thorough rinse with potable water is required for surfaces in direct contact with food.]

[Optional: None, one, or both of the following paragraphs may be used as appropriate; paragraph #"B" may be used before "A" on the label if both are used]

#### [A] [Hospital and Health Care Settings]

[A non-woven disposable cloth for use in hospitals and other critical care areas where control of the hazards of cross-contamination between treated surfaces is required. Use on hard non-porous surfaces and equipment including stainless steel, Formica®, glass, tables, carts, counters, cabinets, telephones. Some organisms are removed from the surface by thoroughly wiping the surface with the towel. Most remaining organisms are killed within five (5) minutes by exposure to the liquid in the towel.] [8]

[B 1] [Areas of Use: Hard non-porous surfaces in: [Hospitals,] [Surgery,] [Recovery,] [Emergency Room,] [E.R.,] [Recovery Room,] [ICU,] [CCU,] [Anesthesia,] [Laboratory,] [Physical therapy,] [Respiratory therapy] [Radiology,] [X-Ray,] [CAT Lab,] [Newborn nursery,] [Pediatrics,] [Central Supply,] [Operating Room,] [Nursing homes,] [Home Health Care], [Physician's offices,] [Dental offices,] [Ambulances,] [Clinics,] [Hospices,] [Healthcare settings] [Doctor's offices]

[Section B 1 must be used if section B2 (below) is on the label]

[B 2] May be used on: Hard non-porous surfaces of: [Ultrasound [transducers or probes]] [Exam tables,] [Gurneys,] [IV poles,] [Bed Railings,] [Outside surfaces of equipment,] [Toilet seats [and] [outside surfaces of toilets,] [Tables,] [Chairs,] [Portable medical equipment,] [Stretchers,] [and other hard non-porous surfaces]

[If paragraph A and/or paragraph B are used, then all of the following paragraph must be used on label.] [This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.]

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towelette(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g \labels\pdi\9480-6afinal 020504label doc

#### DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

### [Use one of the following as applicable to type of container.] [[TOWELETTE] [WIPE] [TOWEL]

#### **[DISPENSER DIRECTIONS:**

registered under EPA Reg. No. To start feed: Remove lid and discard inner seal from canister. Find center of wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep center cap of lid closed to prevent moisture loss.]

#### **TRAVEL PACK:**

Peel back label slowly. Pull out wipes as needed and reseal label. Unfold wipe.]

Use one or more of the following as appropriate; they may be used in any order as long as the entire paragraph is used; food contact directions must be used if "kitchen" is one of the areas of use or listed in the claims section:]

#### TO DISINFECT:

#### NON FOOD CONTACT SURFACES

Use a wipe to remove gross filth (or heavy soil). Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full five (5) minutes. Use additional wipe(s) if needed to assure continuous 5 minute wet contact time. Let air dry.

[Kitchen Areas of Use section (shown on page 4) must be included on label if food contact directions below are used.]

#### FOOD CONTACT SURFACES

Use a wipe to remove gross filth [or heavy soil]. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full five (5) minutes. Use additional wipe(s) if needed to assure continuous 5 minute wet contact time. Let air dry followed by a thorough potable water rinse.

#### ULTRASOUND TRANSDUCERS

For use on external surfaces of Ultrasound Transducers only

To disinfect: Pre-clean ultrasound get from external transducer to be disinfected [by using a clean wash cloth]. Then thoroughly wet the probe with a towelette. Treated surface must remain visibly wet for a full five (5) minutes. Use additional wipe(s) if needed to assure continuous 5 minute contact time. Let air dry.

[At least one of the disinfection directions above must be used on the label in addition to the directions below.] [The following directions are for hospital/medical facility use labels only.]

VIRUS\* OUTBREAKS \*Influenza A2/Hong Kong, \*Herpes Simplex Type 2

[When either of the viruses listed above is the sole concern,] or [For specific outbreaks of either of the viruses listed above, the following directions may be used. Use a wipe to remove gross filth (or heavy soil). Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for one (1) minute. Let air dry. For food contact surfaces, follow one (1) minute contact time with a thorough potable water rinse. Although efficacy at a one (1) minute contact time has been shown to be adequate against the specific viruses listed above, this time is not sufficient for the non-viral organisms listed on the label. Therefore a 5 minute wet contact time must be used for all listed non-viral organisms.

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towelette(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g \labels\pdi\9480-6afinal 020504label.doc

February 5, 2004

Page 5 of 5

FEB 1 1 2004 Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide,

ACCEPTED with COMMENTS in EPA Letter Dated: [The following headings are optional; choose as appropriate:]

ACCEPTED with COMMENTS in EPA Letter Dated:

[Hospital Use] [Nursing Homes] [Doctor's Offices] [Hospices] [Healthcare Settings]

FEB 1 1 2004

[If directions for HIV are to be used, then use the following heading:]Under the Federal Insecticide,<br/>Fungicide, and Rodenticide Act as\*SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINSTHIVaheaded, for the pesticide,<br/>registered under EPA Reg. No.SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS\$9480-6

[000 //, [A]

\*KILLS HIV-1 (AIDS VIRUS) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in one (1) minute at room temperature [68°-77°F] or [20°-25°C] in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

#### [B]

\*KILLS HIV-1 ON THE PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND [TRANSDUCERS] [PROBES] PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in one (1) minute at room temperature [68°-77°F] or [20°-25° C] in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

#### [C]

\*KILLS HIV-1 ON THE PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND [TRANSDUCERS] [PROBES] PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in one (1) minute at room temperature [68°-77°F] or [20°-25° C] in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

#### [If paragraph A, B, or C are used, then all of the language below must be used on label:]

**PERSONAL PROTECTION:** When using this product, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling \*HIV-1 infected blood or body fluids. **CLEANING PROCEDURE:** All blood and other body fluids must be thoroughly cleaned from surface and objects before disinfection by the germicidal cloth. Open, unfold first germicidal cloth to remove gross filth *or* [heavy soil].

**DISPOSAL OF INFECTIOUS MATERIALS:** Dispose of used towelette in accordance with local regulations for infectious waste disposal.

**CONTACT TIME:** Use second germicidal cloth to thoroughly wet surface. Allow to remain wet one (1) minute, let air dry. Although efficacy at a one (1) minute contact time has been shown to be adequate against \*HIV-1 (AIDS VIRUS) this time is not sufficient for the other non-HIV organisms listed on this label. Therefore a 5 minute wet contact time must be used for all listed non-HIV organisms.

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towel(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g:\labels\pdi\9480-6afinai 020504label.doc

[Use storage/disposal statement A if towelette dispenser directions used. Use storage/disposal statement B if travel pack or packet directions used.]

#### [A]

#### STORAGE AND DISPOSAL

Towelette: Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. Dispenser: Do not reuse empty container. Wrap container and put in trash collection.

[B]

#### STORAGE AND DISPOSAL

Towelette: Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. Travel Pack [or Packet]: Put empty travel pack [or packet] in trash collection.

ACCEFTED with COMMENTS in EPA Letter Dated:

FEB 1 1 2004

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. 9 4 8 0 - 6

#### THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towel(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g:\labels\pdi\9480-6afinal 020504label.doc

February 5, 2004

Page 7 of 7

[The following graphics and statements are optional. They may be placed anywhere on label/container.]





#### ACCEPTED with COMMENTS in EPA Letter Dated:

FEB 1 1 2004

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. 9450-6



[This graphic shows use of the wipe dispenser.]

## NOT A BABY WIPE

## NOT FOR USE ON SKIN

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABELING

Statements in brackets [] are optional or instructional. Italicized words in brackets are not included. Sequence of phrases is optional unless specified in 40 CFR 156.10. Capitalization, plurals or singular, bold and italics are all variable unless specified in 40 CFR. The terms "wipe(s)", "towel(s)", towel(s)", and "cloth(s)" may be used interchangeably throughout except for the product primary brand name which can be only changed by notification or amendment. Lists of uses and use sites can be used partially or in their entirety. Any individual statement, claim, restriction or other wording must be used in its entirety, and can not be expanded or shortened by combination, partial deletion, merging, or similar.

g:\labels\pdi\9480-6afinal 020504label.doc

February 5, 2004

Page 8 of 8