

9480-6

9/30/2013

1/16



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY  
WASHINGTON, D.C. 20460

Cristina Griffin  
Professional Disposables International, INC (PDI)  
c/o Delta Analytical Corp.  
12510 Prosperity Drive, Suite 160  
Silver Spring MD 20904

SEP 30 2013

OFFICE OF CHEMICAL SAFETY  
AND POLLUTION PREVENTION

Subject: Sani-Cloth Plus Germicidal Disposable Cloth  
EPA Registration Number 9480-6  
Application Dated: June 8, 2013  
EPA Received Date: June 9, 2013

Dear Ms. Griffin:

The following amendment, submitted in connection with registration under the Federal Insecticide, Fungicide, and Rodenticide Act (FIFRA), as amended, is acceptable.

- To update label by reformatting, adding optional language and rewording Direction for Use.
- To add 5 minutes optional contact time.
- To add "Disinfects in 5 minutes" logo.

Submit and/or cite all data required for registration/reregistration/registration review of your product when the Agency requires all registrants of similar products to submit such data. A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. The next label printing of this product must use this labeling unless subsequent changes have been approved. You must submit one (1) copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. "To distribute or sell" is defined under FIFRA section 2(gg) and it's implementing regulation at 40 CFR 152.3.

**General Comments**

Should you have any questions concerning this letter, please contact Emilia Oiguenblik at (703) 347 0199 or Velma Noble at (703) 308-6233.

Sincerely,

A handwritten signature in black ink, appearing to read "Velma Noble".

Velma Noble

Product Manager (31)  
Regulatory Management Branch I  
Antimicrobials Division (7510P).

# SANI-CLOTH® PLUS

## GERMICIDAL DISPOSABLE CLOTH

**ACTIVE INGREDIENTS:**

n-Alkyl (68% C <sub>12</sub> , 32% C <sub>14</sub> ) dimethyl ethylbenzyl ammonium chlorides.....	0.125%
n-Alkyl (60% C <sub>14</sub> , 30% C <sub>16</sub> , 5% C <sub>12</sub> , 5% C <sub>18</sub> ) dimethyl benzyl ammonium chlorides.....	0.125%
<b>OTHER INGREDIENTS</b> .....	<b>99.750%</b>
<b>TOTAL</b>	<b>100.000%</b>

[Does not include the weight of the [cloth] [towelette] [or] [wipe]]

KEEP OUT OF REACH OF CHILDREN

### CAUTION

See back [side] panel for additional precautionary statements

NET CONTENTS [and/or Net Wt.] \_\_\_\_\_ oz (g) [or lbs. (kg)]  
\_\_\_\_\_ # OF CLOTHS [SIZE of each cloth]


EPA Reg. No.: 9480-6

EPA Est. No. : 9480-NY-1

or

EPA Est. No. A=9480-NY-1, B=72956-WI-1, C=72956-AR-1 [or others]

Alpha character will precede batch code on product

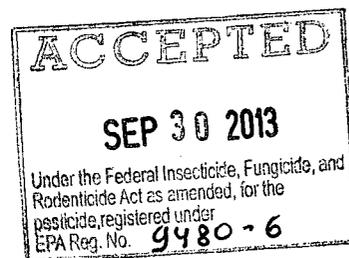

[Manufactured by [or for]:]  
Professional Disposables International, Inc.  
Two Nice-Pak Park  
Orangeburg, NY 10962-1376

[For information call: [phone number to be inserted]]

[Made in USA]

### PRECAUTIONARY STATEMENTS:

#### Hazards to humans and domestic animals.

CAUTION: Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash thoroughly with soap and water after handling, and before eating, drinking, chewing gum, using tobacco, or using toilet.

#### FIRST AID:

**If in eye:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

[For additional information in case of emergency call toll free [phone number to be inserted]].

Have the product container or label with you when calling a poison control center or doctor or going for treatment.

#### PHYSICAL OR CHEMICAL HAZARDS:

Combustible. Do not use or store near heat or open flame. Do not use on natural marble, windows, unpainted wood, brass, clear plastic or colored grout.

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

[Symbols including asterisks, bullets or footnotes and similar may be used to designate categories of organisms.]  
[ATCC numbers are not required to be listed on production label. Organisms may be listed in a table and in any order.] [ATCC # not available unless shown] [Use one or more organisms listed below]

**Bacteria [(3 Minute Contact)] [Time] or [(5 Minute Contact)] [Time]**

- Campylobacter jejuni* [ATCC 29428]
- Escherichia coli* (*E. coli*) [ATCC 11229]
- Escherichia coli* O157:H7 [ATCC 35150]
- Pseudomonas aeruginosa* [ATCC 15442]
- Salmonella enterica* [ATCC 10708]
- Staphylococcus aureus* [ATCC 6538]

**Multi-Drug Resistant Bacteria [(3 Minute Contact)] [Time] or [(5 Minute Contact)] [Time]**

- Methicillin Resistant *Staphylococcus aureus* (MRSA) [ATCC 33592]
- Vancomycin Resistant *Enterococcus faecalis* (VRE) [ATCC 51299] [in 3 minutes [or in 5 minutes]]

**TB [Tuberculocidal] [(5 Minute Contact)]**

- Mycobacterium bovis* BCG (Tuberculosis) [ATCC 35743] ^

**Viruses\* [(3 Minute Contact)] [Time] or [(5 Minute Contact)] [Time]**

- \*Herpes Simplex Type 2 [ATCC VR-734]
- \*Influenza A/Hong Kong [ATCC VR-544]
- \*Influenza A (H1N1) virus [ATCC VR-98] [Strain A/Malaya/302/54]
- \*Respiratory Syncytial Virus [RSV] (Strain Long) [ATCC VR-26]

**Bloodborne Pathogens [(3 Minute Contact)] [Time] or [(5 Minute Contact)] [Time]**

- \*Hepatitis B virus (HBV)<sup>†</sup>
- \*Hepatitis C virus (HCV)<sup>†</sup>
- \*HIV-1 (Aids virus)<sup>†</sup>

A quaternary/alcohol solution impregnated in a wiping cloth. A non woven disposable cloth for use in hospitals and other critical care areas where the control of the hazards of cross-contamination between treated surfaces is required. Suitable for use on equipment requiring alcohol-based products. Use on hard non porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica® and glass. Some organisms are removed from the surface by thoroughly wiping the surface with the wipe. Most remaining organisms are killed within [three (3) [or five (5)] minutes by exposure to the liquid in the wipe [^]

[Reader's note: The symbol and following wording are used only when the 3 minute contact time is used in the paragraph above:] ^ 5 minute contact time for TB

[Optional: Choose one of the following statements:]

- [Respiratory illnesses attributable to Pandemic 2009 H1N1 are caused by influenza A virus. This product is a broad-spectrum hard surface disinfectant that has been shown to be effective against (influenza A virus tested and listed on the label) and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).
- This product has demonstrated effectiveness against influenza A virus and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 influenza A virus.
- This product has demonstrated effectiveness against (influenza A virus tested and listed on the label) and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).
- Kills Pandemic 2009 H1N1 influenza A virus (formerly called swine flu).
- Kills Pandemic 2009 H1N1 influenza A virus.]

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

**Areas of Use:**

**Hospital and Healthcare Settings:** Ambulances, Ambulatory Surgical Centers (ASC), Anesthesia, CAT Labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctors Offices, Donation Centers [blood][plasma][semen][milk][apheresis], Emergency Medical Settings, Emergency Rooms, E.R., Emergency Vehicles, Eye Surgical Centers, Healthcare Settings, Health Care Facilities, Home Health Care, Hospices, Hospitals, ICU, Isolation Areas, Laboratory, Laundry rooms, Neonatal Intensive Care Units (NICU), Newborn nursery, Nursing homes, Operating Rooms (OR), Ophthalmic Offices, Orthopedics, Out-Patient Surgical Centers (OPSC), Patient Care Areas, Pediatrics, Pediatric Intensive Care Units (PICU), Physical therapy, Physician's offices, Radiology, Recovery Rooms, Rehabilitation, Respiratory therapy, Surgery, Surgical Centers, Transport vehicles and X-Ray.

*and/or*

**Critical Care Areas:** CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU) and Surgery.

*and*

**Hospital, Healthcare, and Critical Care Use Sites:** May be used on hard non porous surfaces of: ambulance equipment surfaces; bed railings; cabinets; carts; chairs; counters; dental chairs; dental countertops; dental unit instrument trays; endodontic equipment such as apex locators; exam tables; gurneys; infant incubators [interior and exterior surfaces of]; infant warmers [interior and exterior surfaces of]; isolettes; IV poles; laboratory equipment and surfaces; loupes; operating room tables and lights; operatory light switches; oxygen hoods; physical therapy (PT) equipment surfaces; slit lamps; spine back boards; stethoscopes; stretchers; stools; tables; telephones; toilet seats; and hard non porous outside surfaces of: amalgamators and dental curing lights; anesthesia machines and respiratory therapy equipment; apheresis machines, diagnostic equipment, dialysis machines; patient monitoring equipment, patient support and delivery equipment, pulp testers and motors; toilets; and ultrasound transducers and probes.

**Grocery Stores or Shopping Centers or Supermarkets or Delicatessens:** Disinfects grocery cart handles and grocery cart child seats. Disinfects shopping cart handles and child seats. Cleans grocery carts or shopping carts.

**Nursery or Daycare Centers:** Bassinets [interior and exterior surfaces of], changing tables, cribs, diaper changing stations, diaper pails, infant/child care equipment surfaces, trash cans, toys and other hard, non porous surfaces listed on this label.

**Bathroom[s] or Public Restrooms:** Basins, [bath]tubs, bathroom fixtures, cabinets, countertops, doorknobs, faucet[s], handles, hampers, handrails, shower[s] [stalls], showers, sinks, glazed tiled walls, glazed tiles, toilet [handle] [rim] [seats] [tank] [and other exterior surfaces of], urinals [exterior surfaces of] and vanity tops, and other hard non porous surfaces listed on this label.

**Health club facilities and Gyms:** Gym equipment, weight-lifting surfaces and other hard non porous surfaces listed on this label.

**Schools or Classrooms or Office Buildings or Public Facilities or Manufacturing Facilities or Libraries or Public Telephones:** Cabinets, chairs, computers, computer monitors, computer mouses, counters, countertops, desks, desktops, doorknobs, fax machines, filing cabinets, filing doors, filing handles, garbage cans, headsets, keyboards, railings, seats, sinks, tables, tabletops, telephones, toys, trash cans, work stations, and other hard non porous surfaces listed on this label.

**Transport vehicles, school busses, public busses, fire trucks, police cars:** Hard non porous surfaces of vehicles including steering wheel, hard plastic or metal flooring, seats, and walls.

**Restaurants:** Hard non porous, non food contact surfaces only including chairs, display cabinets, garbage cans, booster seats, floor, sinks, bathroom surfaces, tiled walls, telephones, cash registers, and doorknobs.

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

[Use one of the following as applicable to type of container.]

**[To Dispense TOWELETTES] [or WIPES] [or TOWELS] [or CLOTHS]:**

Remove lid [and discard inner seal] [from container or canister] Find center of wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the [container or] canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep [center cap of] lid closed to prevent moisture loss.

**TRAVEL PACK:**

Peel back label slowly. Pull out wipes as needed and reseal label. Unfold wipe.

**BOX DIRECTIONS:**

Open along perforations as indicated. Remove packets as needed.


**PACKET/PACKAGE:**

Tear open packet. Unfold wipe and use as directed. Discard after single use.

**To Dispense [or To use] [SOLO® PAK] [or soft pack] [or product]:**

Open package lid. Completely remove inner seal. Pull out wipe [at an angle] [towards you through orifice or opening]. [The next wipe pops up automatically.] Snap lid closed after use to retain moisture.

**SOLO® PAK:**

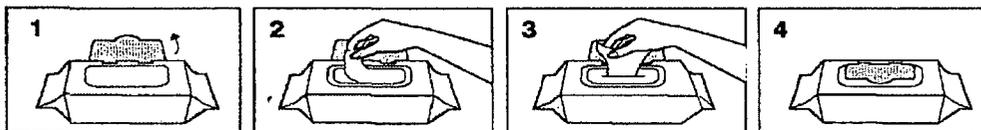

DO NOT FLUSH • STORE AT ROOM TEMPERATURE  
DISPOSE OF USED WIPES IN TRASH RECEPTACLE • KEEP SOLO® LID CLOSED WHEN NOT IN USE  
SOLO® is a registered trademark of Nice-Pak Products, Inc.

**REFILL PACKET [or PACKAGE] [or POUCH]:**

This refill package is made for use only in a [product name] container; do not use this product in a container for another product. Do not use the refill wipes in a container that is cracked or broken. Discard the cracked or broken container in accordance with the container disposal instructions on the label.

**[Option 1]**

Remove [entire] lid from container. Tear open refill [package] at [notch] [mark] [arrow] and place [wipe] roll in empty container. Follow information on container to dispense wipes.

**[Option 2]**

Remove [entire] lid from container. Tear open refill [package] at [notch] [mark] [arrow]. Place [entire] refill package of wipes in empty container. Follow information on container to dispense wipes.

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

**GENERIC:**

Open lid. Pull out wipe. Close lid tightly.

Bucket [or pail or container] *[[numbers optional]*

1. Remove the lid from the [pail or bucket or container.]
2. Press open the dispenser at center of the lid.
3. Open the refill pack.
4. Locate the first wipe at the center of the roll.
5. Feed the first wipe though the dispensing orifice.
6. Reapply the lid to the [pail or bucket or container.]
7. Dispense the wipe.

- For the refills:
1. Remove the lid from the [pail or bucket or container.]
  2. Remove the empty refill pack material from the [pail or bucket or container.]
  3. Press open the dispenser at center of the lid.
  4. Open the refill pack.
  5. Locate the first wipe at the center of the roll.
  6. Feed the first wipe though the dispensing orifice.
  7. Reapply the lid to the pail.
  8. Dispense the wipe.

**DIRECTIONS FOR USE**

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

**DEODORIZING AND DISINFECTING:** To disinfect non food contact surfaces only: Use a wipe to remove heavy soil as needed. Unfold a clean wipe and thoroughly wet surface. Allow surface to remain wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous three (3) minute wet contact time. Let air dry. *[When TB is included on the label, add the following:]* These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at a five (5) minute contact time at 68°F (20°C).

*[or the following alternate directions may be used]:*

**DEODORIZING AND DISINFECTING:** To disinfect non food contact surfaces only: Use a wipe to remove heavy soil as needed. Unfold a clean wipe and thoroughly wet surface. Allow surface to remain wet for a full five (5) minutes. Use additional wipe(s) if needed to assure continuous five (5) minute wet contact time. Let air dry. *[When TB is included on the label, add the following:]* These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at 68°F (20°C).

*[optional directions:]* Ultrasound transducers and probes:

For use on external surfaces of ultrasound transducers and probes only.

To disinfect: Pre-clean gel from external surface of ultrasound transducer or probe to be disinfected by using a clean washcloth. Then thoroughly wet the probe with a towelette. Allow surface to remain wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous three (3) minute wet contact time. Let air dry. *[When TB is included on the label, add the following:]* These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at a five (5) minute contact time at 68°F (20°C).

*[or the following alternate directions may be used]*

Ultrasound transducers and probes:

For use on external surfaces of ultrasound transducers and probes only.

To disinfect: Pre-clean gel from external surface of ultrasound transducer or probe to be disinfected by using a clean washcloth. Then thoroughly wet the probe with a towelette. Allow surface to remain wet for a full five (5) minutes. Use additional wipe(s) if needed to assure continuous five (5) minute wet contact time. Let air dry. *[When TB is included on the label, add the following:]* These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at 68°F (20°C).

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

†KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON THE PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS AND PROBES PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in two (2) minutes [or in five (5) minutes] at room temperature (68°- 77°F) or (20°- 25°C) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

†SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS

**Personal protection:** When using this product, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 (AIDS Virus), HBV or HCV infected blood or body fluids.

**Cleaning procedure:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal wipe. Open, unfold first germicidal cloth to remove heavy soil.

**Disposal of infectious materials:** Dispose of used towelette in accordance with local regulations for infectious waste disposal.

**Contact time:** Use second germicidal cloth to thoroughly wet surface. Allow to remain wet two (2) minutes, let air dry. Although efficacy at a two (2) minute contact time has been shown to be adequate against HIV-1, HBV and HCV this time is not sufficient for the other organisms listed on this label. Therefore a three (3) minute wet contact time must be used for other listed organisms. [When TB is included on the label, add the following:] five (5) minutes for TB.

[for the following alternate contact time wording may be substituted for the entire contact time paragraph above]:

**Contact time:** Use second germicidal cloth to thoroughly wet surface. Allow to remain wet five (5) minutes, let air dry.

[Use storage/disposal statement A if canister, tub, bucket, pail or refill dispenser directions used. Use storage/disposal statement B if travel pack, Solo® Pak, soft pack, pouch, or [refill or [bulk]] packet/package] directions used.]

**STORAGE AND DISPOSAL**

Do not contaminate water, food, or feed by storage and disposal. [Statement not required for residential or household use products.]

[Statement A:]

**Storage:** Do not store near heat or open flame. When not in use keep center cap of lid closed to prevent moisture loss. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Dispenser [or Container] [or Package] Disposal:** Nonrefillable container. Do not reuse or refill this container. Triple rinse and offer for recycling. If recycling is not available, put in trash collection.

[If refillable dispenser is being used, substitute the following: **Refill Dispenser [or Container] [or Package]**

**Disposal:** Triple rinse cracked or broken containers and offer for recycling. If recycling is not available, discard in trash.]

[Statement B:]

**Storage:** Do not store near heat or open flame. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Dispenser [or Container] [or Package] Disposal:** Put empty travel pack [or Solo® Pak or [refill] packet [or packaging]] in trash collection.

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

8/16

**Optional Claims and Descriptors** [Note: Deodorizing directions must be used when deodorizing claims (claims involving odor control) are used]

- X" x Y" [wipes]
- \_\_\_% bigger [[quilted] wipes] [vs. X" x Y" wipes]
- \_\_\_# pre-saturated towelettes X" x Y"
- © [insert year] [insert company name]
- [A][An] [Easy] One-step disinfectant [or disinfecting] (except precleaning is required for heavy soil, blood and body fluids)
- A bleach-free (formula) [a phenolic free (formula)]
- [Absorbs] [Absorbent] [wipe]
- After use, just toss wipe(s) away
- Always ready to use
- An effective method to deodorize and disinfect against odor-causing organisms on hard non porous surfaces listed on this label.
- An effective way to disinfect hard, non porous environmental surfaces listed on this label.
- A non woven disposable cloth for use in hospitals and other critical care areas where control of the hazards of cross contamination between treated surfaces is required.
- [Antibacterial] [Antimicrobial] health club equipment wipe
- A Quaternary [ammonium chloride] solution impregnated in a wiping cloth
- A Sani-Surface® Brand
- Are pre-measured to have the recommended amount of germicidal solution, wipe after wipe.
- Assures proper dilution strength, product effectiveness and standardized technique.
- [Bactericidal]
- Bleach-free [formula]
- Bonus
- Can help reduce the risk of cross contamination on treated surfaces
- Cellulosic non woven disposable cloth
- [Cloth-like texture] [Textured] [strong] [for] [with] scrubbing power
- Compatible for use on hard, nonporous surfaces commonly found in healthcare settings listed on this label
- Contains [name of active ingredient], the [same] active ingredient used in [Brand-name]
- Contains [number] canisters [plus] [and] [number] refills
- Convenient
- [Convenient] [Portable] [way to disinfect hard non porous surfaces]
- Deodorize hard, non porous surfaces [Use to]
- Deodorizes, and kills common household germs\*
- Diamond-embossed wipe is low-linting
- Disinfect [and for touch-up clean]
- Disinfect and scrub clean
- Disinfect external surfaces of ultrasound transducers
- Disinfect the precleaned surface
- Disinfecting towelettes
- Disinfects and deodorizes
- Disinfects grocery cart handles or shopping cart handles or grocery cart child seats or shopping cart child seats
- Disinfect[s] hard non porous environmental surfaces listed on this label [Use to]
- Disinfects hard non porous surfaces [that people touch]
- Disinfects hard non porous [surfaces:] [health club equipment] [treadmill handles] [elliptical handles] [exercise bike handles] [weight benches]
- Disinfects on the go
- Dispenses one wipe at a time
- Does not contain bleach
- Does not produce mist or overspray
- Double-sided scrubbing wipes
- Durable, low-lint wipe
- Easy

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

9/16

Easy [and convenient] way to disinfect bathroom [or] exterior toilet surfaces  
 Easy [and convenient] way to [touch-up] [bathroom] [office] everyday  
 [Easy] [convenient] [dispensing] lid [closure] [cap] [package]  
 Easy drop-in refill[s]  
 [Easy] pop-up-tub  
 Easy start lid [package]  
 Easy to use [easy to refill]  
 Easy to use package [canister] [container] keeps [your] wipes moist  
 Easy way to disinfect hard, non porous surfaces  
 [Economical] Refill[s] [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] –pack  
 [Economical] Refills [now] available  
 Economy  
 Effective against bathroom bacteria and viruses\* *[use at least two bacteria and two viruses from the organism list]*  
 Effective against [the following micro-organisms on hard, non porous surfaces] *[list organisms]*  
 Eliminates odors  
 Eliminates potential for overspray  
 Extra large [disinfecting] [Y" x Z" compared to our A" x B" size] wipes  
 For daily use  
 Gets rid of [or] eliminates odors  
 [Helps] [kill][s] [eliminate][s] odor-causing bacteria  
 Helps prevent [bacterial] [viral] cross contamination [between] [hard non porous] [on treated] [surfaces]  
 Hospital disinfectant [disinfection] *[use only when hospital use sites on label]*  
 Includes \_\_\_ count refill [pack] [contains] \_\_\_ wipes  
 Just [or] simply wipe exterior toilet surface [or] bathroom germs\* away  
 Just [pull] wipe and toss  
 Kills bacteria and viruses\* in the bathroom *[use at least two bacteria and two viruses from the organism list:]* –or-  
 on the exterior toilet surfaces  
 [Kills] [Effective against] [MRSA] [(Methicilin Resistant Staphylococcus aureus)]  
 Kill germs\* and bacteria instead of germ paper towels, sponges and rags [Use this product to]  
 Kills germs\* and bacteria and viruses in the bathroom [or] on the exterior toilet surfaces  
 Kills household [or] exterior toilet surface [or] bathroom germs\* + *[use at least two bacteria and two viruses from the organism list]*  
 Kills odor causing [bacteria] [germs\*]  
 Leaves [bathroom] [office] disinfected [and smelling fresh]  
 Less [reduced] packaging [less to throw away]  
 Moisture lock lid  
 Multi-pack  
 Multi-purpose disinfectant [Multi [-refill] pack]  
 Next wipe pops up automatically [everytime] [!]  
 No canister needed  
 No measuring  
 No mixing  
 Non woven disposable cloth  
 No [pinched fingers] [rips] [tears]  
 No sponges, no rags, no buckets, no mess  
 No threading [required]  
 Non-scratching [wipe]  
 Not for sale in *[insert name of state(s)]*  
 Not for in-home use  
 Not for sale [or use] after *[insert date]*  
 Odor [elimination] [neutralization]  
 One [wipe] at a time dispensing  
 On-the-go disinfectant  
 [Package] *[insert type of package]* made from [x%] recycled materials

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

10/16

Perfect for disinfecting exterior toilet surfaces –or- around the exterior toilet surface  
Picks up [particles] [dust] [dirt] [hair] [crumbs]  
Pop-up dispensing  
Pop-up [top]  
Portable  
Premeasured  
Premixed  
Premixed, premeasured, premoistened  
Premoistened  
[[Product name] is virucidal against:] [use at least two bacteria and two viruses from the organism list:]  
[[Product name] kills bacteria listed on this label:] [use at least two bacteria and two viruses from the organism list:]  
Pull n/ Go [pack]  
Ready to Use [on the following hard non porous surfaces]  
Recyclable [package]  
Reduce [risk of cross contamination] [on hard non-porous surfaces]  
Reduces [packaging] waste [by x%] [when compared to regular canister or bucket or pail]  
[Refill] package contains [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] [wet] [pre-moistened] [wipe] [refill[s]] [roll[s]]  
Refill [size] [pack] [value pack]  
Refillable [with \_\_\_# wipes] [dispenser] refill [pack] [for dispenser]  
Requires no mixing or measuring and are ready-to-use  
Resealable [stay-moist] pouch [package] [container] inside  
Reuse your [empty] canisters [again [and again] [!]  
Sample  
Scrubbing power of a sponge in a disinfecting wipe  
Strong, durable diamond-embossed textured wipe deodorizes  
Strong, durable low-lint wipe [designed for sensitive non porous equipment surfaces]  
Suitable for use on equipment requiring alcohol-based products  
Tear at perforation  
Textured diamond-embossed design [deodorizes hard, non porous surfaces]  
Thick, large wipe disinfects  
Thick, textured wipe with diamond-embossed design  
This [outer] [selling] [package] [container][canister] contains x% post consumer [material][plastic][resin]  
Time to reorder  
Trial  
[Tuberculocidal]  
Tub refill  
Tub with pop-up action  
Twin pack  
Use by [insert date]  
Use on hard, non-porous surfaces only: [stainless steel] [Formica®] [glass] [glass tables] [glazed tile]  
[Test wipe on small inconspicuous area first.]  
Use to preclean or decontaminate critical or semi-critical medical devices prior to sterilization / high-level disinfection  
[Use to] refill[s] [count] [wipe] canisters or tubs  
Value  
Versatile  
[Virucidal\*]  
Virucide\* including HIV-1, HBV and HCV applications and effective against Staphylococcus aureus  
Wet wipes  
Where control of the hazards of cross contamination between treated surfaces is of prime importance  
Wipe [material] made from [x] % [sustainable] [ingredients] [fibers] [May be used only when cellulose wipe

The terms “wipe(s)”, “towel(s)”, “towelette(s)”, and “cloth(s)” and the product name may be used interchangeably except for within the product primary brand name.

*material used]*

[Wipes out] [Kills] [common] [household] [bacteria] [viruses] [germs\*]  
[Wipes out] [Kills] [the] flu virus, (Influenza A/Hong Kong)  
[Wipes out] [Kills] viruses\* and bacteria [including the flu virus, (Influenza A/Hong Kong)]  
[Wiping] [soft] side for disinfecting  
X% [more] [wipes] [free] [vs. Z count] [canisters] [packs]  
X% reduction in packaging [material]

**[Note: The following claims can be used only for 6 months]**

[Better] [New] [Improved] [Stronger] [Thicker] Wipe [!]  
[Better] [new] [&] [improved] [dispensing] lid [closure] [cap] [package] [dispensing] [better [than before]  
[New] [Package] [packaging] uses [x%] less material (plastic) [then] [when compared to] [regular]  
containers [or canisters]  
New and improved  
New larger, thicker wipe [with diamond-embossed design]  
New thicker and [or with] larger diamond-embossed [wipe cleans more surface area]  
[Now] [Reinforced] [With] Quilted [strength]  
Now with [%] more towel [use this for 6 months after converting to larger towel]  
Now with [#] more wipes [use this for 6 months after converting to a larger number of towels]  
[Scrubs] (&) [cleans] [better than ever]  
[Stronger] more durable [wipe]  
Thick[er] [wipe] [with] cloth-like texture  
X% [Bigger] [larger] [quilted] wipes  
[x%] thicker wipes

**3 minute claims:** [if TB is included on the label add footnote for TB as shown, otherwise leave off footnote and mark]

Disinfects and deodorizes [in 3 minutes^]  
Disinfects hard, non porous surfaces [in 3 minutes^]  
Disinfects [in [just] three [3] minutes^]  
Kills bacteria in 3 minutes ^  
Kills [list one or more organisms][in 3 minutes^]  
The 3-minute^ germicidal wipe [disinfectant wipe]  
Thick, heavy duty wipes are bactericidal and virucidal in 3 minutes^  
[Wipes out] [Kills] [eliminates] [list organisms] in [just] [3 minutes ^]

^ 5 minutes for TB

**5 minute claims:**

Disinfects and deodorizes [in 5 minutes]  
Disinfects hard, non porous surfaces [in 5 minutes]  
Disinfects [in [just] five [5] minutes]  
Kills bacteria in 5 minutes  
Kills [list one or more organisms][in 5 minutes]  
Kills TB in 5 minutes  
The 5-minute germicidal wipe [disinfectant wipe]  
Thick, heavy duty wipes are bactericidal and virucidal in 5 minutes  
[Wipes out] [Kills] [eliminates] [list organisms] in [just] [5 minutes]

1 [A footnote can be used to list organisms shown on page

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

2. Bacteria and virus can be separated into footnotes 1 and 2 if desired.]

\* All "germ" claims must be linked to bacteria (minimum: Salmonella and Staph) and at least one virus with an asterisk or footnote. For example: Front panel: "Effective against household germs\*" Back panel: "\*\*Kills Salmonella enterica, Staphylococcus aureus, Herpes Simplex Type 2, and Influenza A/Hong Kong.

**CLEANING:** [cleaning directions and claims have been grouped on this page (they were previously incorporated into other parts of the label)]

**DIRECTIONS FOR USE:**

**CLEANING:** Wipe surface with towelette until clean. Let air dry.

**CLAIMS:**

An effective method to clean against odor-causing organisms on hard non porous surfaces listed on this label.

Cleans

Cleans and disinfects

Cleans, deodorizes, and kills common household germs\*

Cleans, disinfects and deodorizes

Cleans grocery carts and shopping carts

Diamond-embossed pattern [texture] provides effective cleaning

Leaves [bathroom] [office] clean [and disinfected] [and smelling fresh]

Leaves bathroom fresh and clean in [just] minutes

Multi-purpose cleaner [Multi [-refill] pack]

New thicker and larger diamond-embossed wipe cleans more surface area [use this for 6 months after converting to larger towel]

Perfect for cleaning [and disinfecting] around the toilet

Premoistened [wet] household cleaning wipe

Premoistened with specially formulated cleaners to handle daily clean-ups without bottles, sponges or buckets

[This product is]

Easy [and convenient] way to clean [and disinfect]

Easy way to clean [and disinfect] bathroom –or- exterior toilet surfaces

Easy way to clean [and disinfect] hard, non porous surfaces

[So] surfaces are more than [just] [clean] [they're] [disinfected]

[Soft] side for [cleaning] dirt away

Strong, durable diamond-embossed textured wipe cleans [and deodorizes]

Textured wipe for effective cleaning

Textured diamond-embossed design cleans hard, non porous surfaces

Textured diamond-embossed design helps remove soil from surfaces

[Textured] [scrubby] [rough] side for [scrubbing] [cleaning] [removing] [wiping] [tough] [stains] [soils] [messes]

Thick, large wipe cleans

Wipe away odors

[Wipes out] [Kills] germs\* that cause odors

[Wipes out] [Kills] germs\* [while it cleans]

[Wiping] [soft] side for [wiping] dirt away [and disinfecting]

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

[The following graphics and statements are optional.  
They may be placed anywhere on label/container.]




**NOT A BABY WIPE**

**NOT FOR USE ON SKIN**

*This graphic shows use of the wipe dispenser.*

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.

[The following graphics and statements are optional. They may be placed anywhere on label/container.]

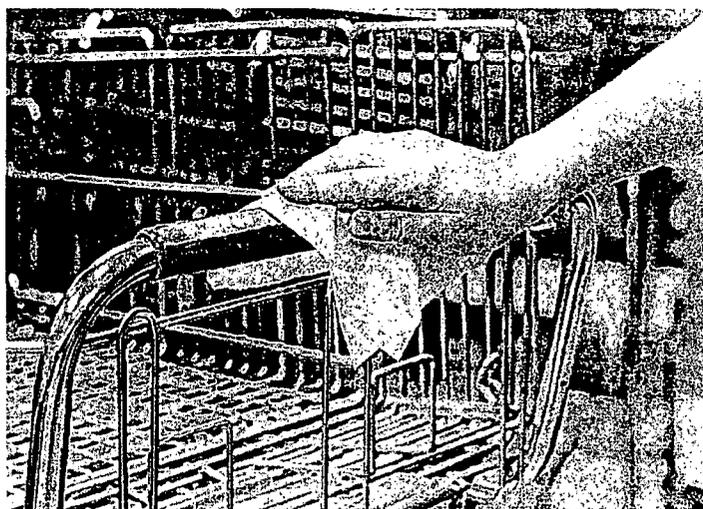

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.



Not a Skin Wipe.  
For Use on Hard Surfaces Only.



Not a Skin Wipe.  
For Use on Hard Surfaces Only.

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.


[The following graphics and statements are optional.  
They may be placed anywhere on label/container.]



[One side scrubs]  
[One side wipes]



[The following are company logos for use on PDI product labels. The purpose of the logo is for easy recognition of PDI products in the marketplace.]

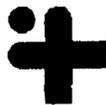

The terms "wipe(s)", "towel(s)", "towelette(s)", and "cloth(s)" and the product name may be used interchangeably except for within the product primary brand name.