

Premier
STA-KLEEN

**X-Ray
Fungistatic Solution**

STA-KLEEN controls and prevents fungi and slime in the X-ray Developer Solution and Rinse Tanks.

STA-KLEEN controls odors and extends the life of the Developer solution resulting in obtaining a clear-sharp picture for a longer period of time.

Fungi and slime digest and grow on the organic chemicals in the X-ray solution—thus reducing the life of this costly compound.

WARNING

KEEP OUT OF REACH OF CHILDREN

See reverse panel for other precautions and directions.

Exclusive Distributor
PREMIER DENTAL PRODUCTS COMPANY
Philadelphia, Pennsylvania

U.S.D.A. REG. No. 9678 Z

This container is equipped with a white two ounce measuring cap for your use and convenience.

DIRECTIONS

TO CONTROL FUNGI, SLIME AND ODOR IN THE DEVELOPER SOLUTION TANK

- Start with a clean one gallon tank
- Mix Developer Solution and water, then
- Add 2 ounces of Sta-Kleen

TO CONTROL FUNGI, SLIME, AND ODOR IN THE RINSE TANK

- Start with a clean one gallon tank
- Fill rinse tank with water, then
- Add 2 ounces of Sta-Kleen

In larger or smaller tanks, use Sta-Kleen at the rate of 2 ounces per gallon.

In rinse tanks where water is added it may be necessary to add a small amount of Sta-Kleen as a shock treatment if growth of fungi is noted.

ACTIVE INGREDIENT	
Sodium Pentachlorophosphate	1.21%
Sodium Salt of Other Chlorophosphate	0.19%
TOTAL ACTIVE INGREDIENT	1.40%
INERT INGREDIENTS	98.60%

WARNING: Injurious to eyes. Absorbed through skin from strong solutions. Causes skin irritation. Harmful if swallowed.

Do Not Get in Eyes, on Skin or Clothing
Do Not Take Internally

In case of contact, flush eyes immediately with plenty of water for at least 15 minutes and get medical attention; for skin, wash with soap and plenty of water. Wash clothing before reuse. Note to physician: This product is a metabolic stimulant. Causes hyperthermia. Treat symptomatically.

Manufactured by
SCHWARTZ CHEMICAL COMPANY
Orlando, Florida

CONTAINS 12 FLUID OUNCES
02 00200