

FRONT PANEL

SONIC PLUS
SANITIZING DENTURE CLEANER

tablets, not liquid

SANITIZER

DEODORIZER


For the removal of Crown and Bridge Cement, Tobacco Stains, Calcium Deposits, Plaster, Stone, and other foreign matter from Dentures, Crowns and Bridges in minutes.

Mixes to half-ounce of aqueous solution

DANGER: Keep out of reach of Children
See other warning statements on right panel

USEPA EPC. No. 9678-5

NET CONTENTS ONE POUND

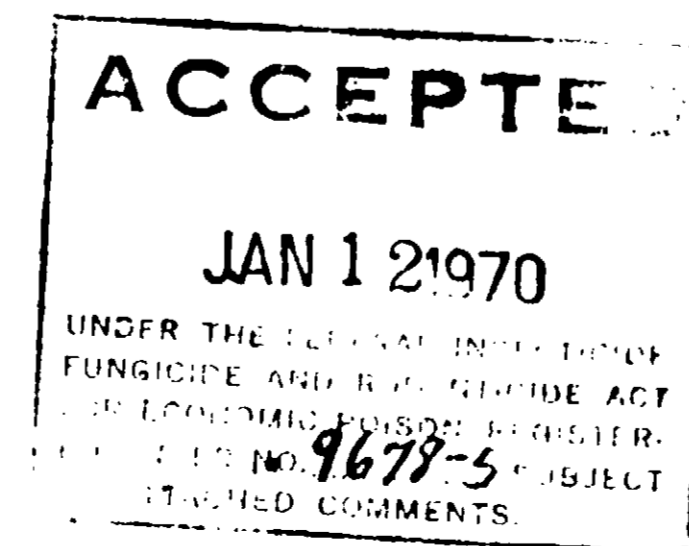

LEFT PANEL

Enclosed in this container is a one ounce measuring scoop for your convenience.

Product may be used with or without the aid of an ultrasonic cleaning machine.

Product contains a built-in inhibitor system and may be used on acrylics, enamels, and plastics normally used in dentures, crowns, and bridges when used as directed.

Product may be used as a polishing agent for gold.

Directions: Add 1/2 ounce (15 ml) of product to 1/2 cup (125 ml) of water in a glass beaker. If used with an ultrasonic cleaning machine will put product into solution in 10 seconds.

Directions for use on dentures: PLUS solution in ultrasonic cleaning machine. Immerse dentures, crowns or bridges in PLUS solution in glass beaker. Run through 5 to 10 minute cycle for cleaning. Allow to remain in solution for 15 minutes for complete chemical sanitation. Rinse and wash and rinse under flowing water, thoroughly.

Caution: Use limit for cleaning should be doubled when using ultrasonic cleaning machine without the aid of an ultrasonic cleaning machine.

RIGHT PANEL

Warning: Damaging to the eyes - may irritate nose, throat and skin - do not get in eyes - avoid contact with skin and clothing - Avoid breathing dust - avoid contact with food - may be harmful or fatal if swallowed - Do not use on gold, silver, platinum, or jewelry - Do not use on softened soap, or raw egg whites - Call physician at once - in case of contact, flush skin with water - for eyes flush with plenty of water for 15 minutes and get medical attention.

ACTIVE INGREDIENTS

Sulfamic Acid

Dibutylphenoxycetylalcohol

benzyl ammonium chloride

OTHER INGREDIENTS

For Professional Use Only
Exclusive Distributor
DENTAL PRODUCTS COMPANY
Philadelphia, Pennsylvania
Manufactured By
LUCASITE CHEMICAL COMPANY
Orlando, Florida

ACCREDITED
1-12-70
9678-5