

WAVICIDE-01

Concentrate

MIXING DIRECTIONS:

FOR STERILIZATION OF HOSPITAL AND DENTAL INSTRUMENTS

To prepare a 2% sterilizing solution, pour contents of both Wavicide-01 Concentrate and Corrosion Inhibitor into a 128 fl. oz. (1 U.S. gallon) container and fill with distilled water.

FOR DISINFECTION OF HOSPITAL AND DENTAL INSTRUMENTS

To prepare a 0.5% disinfecting solution, pour 1 gallon of Wavicide-01 2% sterilizing solution into a tray and add 3 gallons of potable water.

To prepare a 1% solution for disinfecting scopes, pour 1 gallon of 2% Wavicide-01 sterilizing solution into a tray and add 1 gallon of potable water or pour half the contents of 1 gallon 2% Wavicide-01 sterilizing solution into another gallon bottle and fill with potable or distilled water.

FOR CLEANING IN ULTRASONIC TANKS

The sterilizing (2%) or disinfecting (0.5%) solutions can also be used for precleaning of instruments in ultrasonic tanks. A 10 minute exposure in a high intensity cavitation field should be satisfactory for precleaning of most instruments. Solutions used for ultrasonic cleaning should be discarded after each use.

BEST AVAILABLE COPY

PROXEL PLTM

Industrial Microbiostat

For industrial use only as a bacterial slime control agent in the pulp and paper manufacturing process.

Active Ingredient—30%
Inert Ingredients—70%

1, 2-Benzisothiazolin-3-one

E.P.A. Reg. No. 10182-4
Est. Reg. No. 33909-EN-3

Contents: 440 lbs. net minimum (45 gal.)

See T. I. Bulletin 322 for directions for use.

DANGER:

KEEP OUT OF REACH OF CHILDREN. CORROSIVE. CAUSES EYE AND SKIN BURNS. HARMFUL OR FATAL IF SWALLOWED. MAY CAUSE ALLERGIC SKIN REACTION. Do not get in eyes, on skin, or on clothing. Wear goggles or face shield and rubber gloves when handling. Avoid breathing mist. Use with forced ventilation. Wash thoroughly after handling. Avoid contamination of food or feed.

FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes while removing contaminated clothing and shoes. Call a physician. Wash clothing and decontaminate shoes before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution, or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN:

If ingested, probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression, and convulsion may be needed.

ENVIRON- MENTAL HAZARDS:

This product is toxic to fish. Treated effluent should not be discharged where it will drain into lakes, streams, ponds, or public water.

In case of Spill or leak - Flush away with water to a self-contained sewer or soak up with sand, earth, or synthetic absorbent and remove to chemical waste area where the material can be disposed of as a solid waste in accordance with state and local pollution regulations.

Container Disposal - Do not reuse container. Puncture, crush, and bury in a sanitary landfill in accordance with state and local pollution regulations.

SOLD BY



ICI United States Inc.

Wilmington, Delaware 19897

Contents Manufactured in Great Britain

NET WEIGHT

LBS

KILOS

LOT NUMBER

Am 92h New 1 77