

RIGHT PANEL - Continued

DIRECTIONS FOR USE

When using MANTEK MEREX Germicidal Cleaner on resilient surfaces, such as asphalt tile, vinyl tile, linoleum, rubber, plastic or on sealed wood floors, add 2 ounces MANTEK MEREX Germicidal Cleaner to one gallon of water. If the floor is extremely dirty, a stronger cleaning solution should be used - 4 to 6 ounces per gallon of water.

After the cleaner has loosened the dirt and soil, pick up the remainder with a mop or squeegee. If desired, the floor may be rinsed with clear water.

AS A DISINFECTANT: MANTEK MEREX Germicidal Cleaner has a phenol coefficient of 13.0 against Staphylococcus Aureus and a phenol coefficient of 10.0 against Salmonella Typhosa. A dilution of 2 ounces per gallon of water will destroy or control many bacteria and fungi thereby reducing the danger of cross infection. For infectious diseases, repeated rinsing of all surfaces is recommended.

CAUTION: Avoid Contamination of Foods. Avoid contact with skin and eyes. In case of contact, wash thoroughly with water. If irritation persists, get medical attention.

USDA Reg. No. _____

DISTRIBUTED BY
AMERICAN HOSPITAL SUPPLY
DIVISION OF AMERICAN HOSPITAL SUPPLY
CORPORATION, EVANSTON, ILLINOIS 60201


**TOMAC.
SENTRY™
THERMOMETER
KIT**

**SUGGESTED
PROCEDURE:**

- Remove contents from box
- Pull off plastic cover
- Remove thermometer
- Snap off center post on cap
- Insert thermometer
- Replace plastic cover
- Clean thermometer after each use

PATIENT NAME _____


ROOM NO. _____

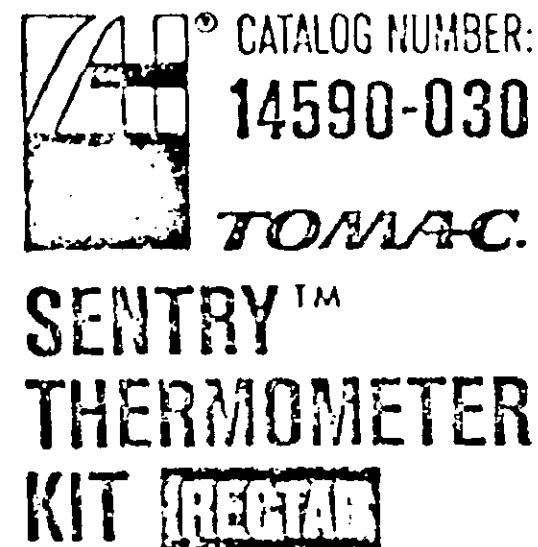

CONTENTS:

- Thermometer
- Prefilled Thermometer Vial*
- Holder and Cover
- Packet of Lubricant

* 1:750 Aqueous Benzalkonium Chloride

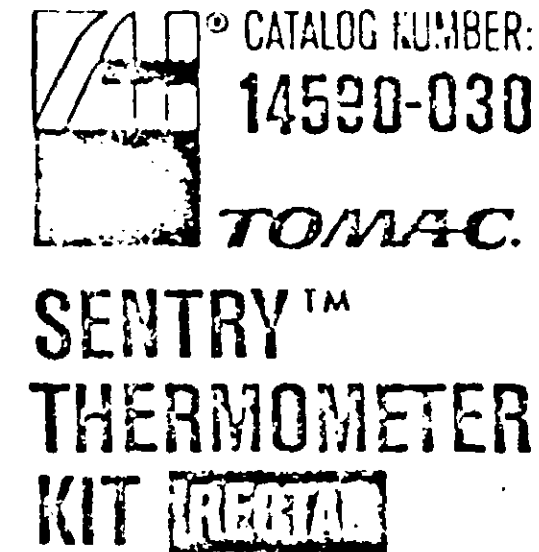

FOR INDIVIDUAL PATIENT USE ONLY
Rinse vial with water before releasing to patient.

Thermometer solution contains:

1:750 Aqueous Benzalkonium Chloride
solution for disinfecting clinical
thermometer

ACTIVE INGREDIENTS:

n Alkyl (C₁₂-50% , C₁₂-40% , C₁₂-10%)

-Dimethylbenzylammonium

Chloride 0.13%


INERT INGREDIENT 99.87%

EPA Reg. No. 10227-1 EPA Est. 33595 IL-1

WARNING - KEEP OUT OF REACH
OF CHILDREN

Net Contents 0.67 Fl. Oz.

DISTRIBUTED BY
AMERICAN HOSPITAL SUPPLY
DIVISION OF AMERICAN HOSPITAL SUPPLY
CORPORATION • MCGRAW PARK • ILLINOIS 60085

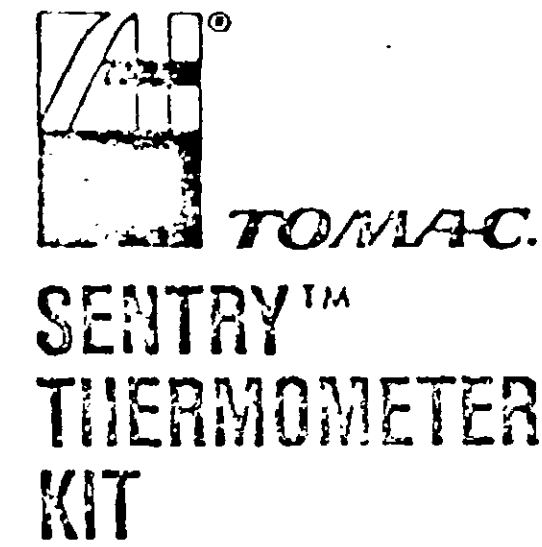

This thermometer conforms
to all the requirements
established in Voluntary
Product Standard PS 39-70
Published by the National
Bureau of Standards

These are one dimension reproductions of
actual boxes for outer containers measuring
1 7/8" x 1 7/8" x 4 3/4", black print on white
cardboard backgrounds.