

# Vacudent

INSTRUCTIONS

INTERNAL PARTS

CONTAINER DISPOSAL:

**DANGER** Keep out of reach of children. Corrosive. Causes eye damage and skin irritation. Do not get in eyes, on skin, or on clothing. Protect eyes and skin when handling. Harmful if swallowed. Avoid contamination of food.

**FIRST AID:** In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Call a physician immediately.

**NOTE TO PHYSICIAN:** Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression, and convulsion may be needed.

**NOTICE:** Manufacturer or seller is not liable for any injury or damage caused by this product due to misuse or mis-handling, or for any application other than specifically recommended on this label.

# Vacudent

## cleaner & deodorizer CONCENTRATE

### CAUTION

DO NOT USE IN THIS CONCENTRATED FORM. MIX 1 PART CONCENTRATE WITH 100 PARTS OF WATER. COMPLETE DIRECTIONS ON BACK.

**ACTIVE INGREDIENTS:**  
 n-Alkyl (60% C<sub>12</sub>, 30% C<sub>14</sub>, 5% C<sub>16</sub>, 5% C<sub>18</sub>)  
 dimethyl benzyl ammonium chloride 5%  
 n-Alkyl (50% C<sub>12</sub>, 30% C<sub>14</sub>, 17% C<sub>16</sub>, 3% C<sub>18</sub>)  
 dimethyl ethyl benzyl ammonium chloride 5%  
**INERT INGREDIENTS** 90%  
**TOTAL** 100%

### DANGER

KEEP OUT OF REACH OF CHILDREN  
HARMFUL IF SWALLOWED. SEE BACK PANEL FOR DETAILS  
See First Aid and other precautions on panel

EPA REG. NO. 100-225-1

## CONTENTS: ONE QUART



ACCEPTED

9-8-69

UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE ACT  
FOR ECONOMIC POISON REGISTER-  
ED NUMBER 10327-1



Eaton Allen Phone Spray is a proven formula that sanitizes and deodorizes telephone mouthpieces and ear pieces. It provides a bacterio-static residual that inhibits the growth of bacteria and provides odor control.

**DIRECTIONS**

1. Hold the can 6 to 8 inches from telephone.
2. Aim the valve at area to be sprayed. Press valve down. Use circular motion while spraying. One short spray is sufficient.

**CAUTION:** Contents under pressure. Do not puncture or incinerate. Do not use near heat or store at temperatures above 120°F. Do not spray near eyes. In case of contact, flush eyes with plenty of water for 15 minutes. Get medical attention if required.

**ACTIVE INGREDIENTS:** Isopropyl Alcohol 1.92% Para-diisobutylphenoxyethoxyethyl benzyl ammonium chloride 0.08%.

**INERT INGREDIENTS:** 98.00% — Propellant: 97.5%  
Essential Oils: .5%

(Part No. 15A)

**EATON ALLEN CORP.**

67 Kent Avenue, Brooklyn, New York 11211  
Manufacturers of KO-REC-TYPE, KO-REC-COPY, Carbons,  
Ribbons, and Duplicating Supplies.

U.S.D.A. Reg.No. 10327-1



Contents  
5 oz.  
Net. Wt.



**Sanitizes  
and  
Deodorizes  
TELEPHONES**

**CAUTION:**  
**KEEP OUT OF REACH  
OF CHILDREN**  
See back panel  
for other cautions.

9-8-69  
10327-1



Eaton Allen Phone Spray is a proven formula that sanitizes and deodorizes telephone mouthpieces and ear pieces. It provides a bacterio-static residual that inhibits the growth of bacteria and provides odor control.

**DIRECTIONS**

1. Hold the can 6 to 8 inches from telephone.
2. Aim the valve at area to be sprayed. Press valve down. Use circular motion while spraying. One short spray is sufficient.

**CAUTION:** Contents under pressure. Do not puncture or incinerate. Do not use near heat or store at temperatures above 120-F. Do not spray near eyes. In case of contact, flush eyes with plenty of water for 15 minutes. Get medical attention if required.

**ACTIVE INGREDIENTS:** Isopropyl Alcohol 1.92% Para diisobutylphenoxyethoxyethyl benzyl ammonium chloride 0.08%

**INERT INGREDIENTS:** 98.00% - Propellant 97.5%  
Essential Oils: 5%

(Part No. 15A)

**EATON ALLEN CORP.**

67 Kent Avenue, Brooklyn, New York 11211

Manufacturers of KO REC-TYPE, KO-REC COPY, Carbon Ribbons, and Duplicating Supplies