



IV. (UCARCIDE HS Activator Salt Label - Front Panel)

UCARCIDE HS
Sterilizing And Disinfecting Solution

ACTIVATOR VIAL
For Use With UCARCIDE HS Solution

UNION CARBIDE CORPORATION • SPECIALTY CHEMICALS DIVISION
Danbury, Connecticut 06817 • Offices in principal cities
UCARCIDE is a registered trademark of Union Carbide Corporation

ACCEPTED
COMMENTS
NOV 03 1983
10550 29

**UNION
CARRIDE** **BUSINESS
CONFIDENTIAL**

(UCARCIDE HS Draft Label - Left Panel)

**Precautionary Statements
Hazards to Humans**

DANGER

Keep out of Reach of Children

Causes eye and skin irritation. Avoid contact with eyes, skin and clothing. Wash thoroughly after handling. Avoid contamination of food.

Storage and Disposal

Store this container and any of its unused contents in a cool area designated for the storage of disinfecting or sterilizing products. Keep out of reach of children and away from drugs, medicines, or food.

Do not re-use empty container.

Rinse empty container thoroughly and discard.

WARRANTY:

1. The manufacturer guarantees and warrants (a) that the active ingredient content and the total net weight are as stated within lawful limits and (b) that the directions, warnings and other statements on this label are based upon responsible experts' evaluation of reasonable tests of effectiveness and of toxicity to laboratory animals.
2. The manufacturer further warrants that the material herein is reasonably fit for use under normal conditions as directed hereon. The manufacturer DOES NOT MAKE AND DOES NOT AUTHORIZE ANY AGENT OR REPRESENTATIVE TO MAKE ANY OTHER WARRANTIES OF FITNESS OR OF MERCHANTABILITY, GUARANTEE OR REPRESENTATION, EXPRESS OR IMPLIED, CONCERNING THIS MATERIAL. This product is sold only on the basis that buyer assumes all risks of use, handling, or storage which result in loss or damage and which are beyond manufacturer's control. No claim of any kind, and whether or not based on negligence, shall be greater in amount than the purchase price of the material in respect of which such claim is made. In no event shall manufacturer or seller be liable for special, indirect or consequential damages resulting from the use, handling or storage of this material.
3. No modification of this warranty and disclaimer is authorized, except by specific reference to them in writing by an authorized employee of the manufacturer.

ACCEPTED
with COMMENTS
in EPA Letter Dated:

NOV 03 1983

Product of Union Carbide
Foliar Disinfectant
Registered under the Pesticide
Act of 1970
EPA Reg. No. 105.2.27

UNION
CARBIDE

BUSINESS
CONFIDENTIAL

(UCARCIDE HS Draft Label - Right Panel)

DIRECTIONS FOR USE

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

DIRECTIONS FOR ACTIVATING

THE ATTACHED ACTIVATOR VIAL MUST BE ADDED TO THE SOLUTION BEFORE THE PRODUCT IS EFFECTIVE. Add entire contents of attached activator package to this container of UCARCIDE HS solution. Mix well. Do not dilute. UCARCIDE HS solution is now green and ready for use. The shelf life of the activated, unused stock solution is 28 days.

DIRECTIONS FOR USE

For disinfecting or sterilizing medical instruments and equipment, thoroughly clean, rinse, and rough dry items before immersing in FULL Strength Activated UCARCIDE HS Solution. Flush and fill cleansed lumen of any hollow instrument. Surfactant action will facilitate penetration into hard-to-reach areas.

DISINFECTION: Immerse item completely for a minimum of 10 minutes at room temperature to destroy vegetative pathogens including: *Mycobacterium tuberculosis*, *Pseudomonas aeruginosa* and viruses* ((Coxsackie virus Type B-1, Herpes simplex Type 1, Influenza Type A-2 (Hong Kong), Poliovirus Type 1)) on inanimate surfaces.

STERILIZATION: Immerse item completely for a minimum of 10 hours at room temperature to destroy resistant pathogen spores including: *Clostridium sporogenes* and *Bacillus subtilis*.

Remove items from UCARCIDE HS Solution using aseptic technique and rinse thoroughly with sterile water. Discard the used portion of the solution after each day's usage.

Date of Activating

+ 28 Days →

Date to Discard

ACCEPTED
with COMMENTS
in EPA Letter Dated:

NOV 03 1963

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, this pesticide
registered under EPA Reg. No.

105029

III. (UCARCIDE HS Draft Label - Center Panel)

UCARCIDE HS

Sterilizing and Disinfecting Solution

A rapid-acting solution for disinfection and sterilization of medical, surgical, and dental instruments, respiratory therapy and anesthesia equipment.

BROAD SPECTRUM OF ACTIVITY

Sporicidal	Bactericidal
Tuberculocidal	Pseudomonocidal
Virucidal*	Fungicidal

Active Ingredients:	Glutaraldehyde.....	2.0%
	Phenol.....	0.0% B.S.
Inert Ingredients:	97.0%
		100.0%

ACCEPTED
with COMMENTS
in EPA Label Draft

NOV 03 1983

10352-09

Keep Out of Reach of Children

DANGER

Statement of Practical Treatment

In case of contact: for eyes, immediately flush with plenty of water for at least 15 minutes. Call a physician. For skin, flush with water. Wash clothing before reuse.

UNION CARBIDE CORPORATION . SPECIALTY CHEMICALS DIVISION
 Danbury, CT 06817 . Offices in principal cities
 EPA Reg. No. EPA Est. No.
 UCARCIDE is a Registered Trademark: UCC No.
 of Union Carbide Corporation Made in USA

Net Volume One Gallon (128 fl. oz.) *See directions for use





(UCARCIDE HS Activator Salt Label - Back Panel)

Directions: Add entire contents of this vial to one gallon of UCARCIDE HS solution. Mix well. UCARCIDE HS solution is now green and ready for use.

Danger: In case of contact: for eyes, immediately flush with plenty of water. Call a physician. For skin, flush with water.

ACCEPTED
NOV 03 1963
10352-29

0000
0000

U.S. ENVIRONMENTAL PROTECTION AGENCY
OFFICE OF PESTICIDES PROGRAMS
REGISTRATION DIVISION (HH-567)
WASHINGTON, D.C. 20460

EPA REGISTRATION NO.

DATE OF ISSUANCE

TERM OF ISSUANCE

NAME OF PESTICIDE PRODUCT

03 NOV 1983

NOTICE OF PESTICIDE: REGISTRATION
 REREGISTRATION
(Under the Federal Insecticide, Fungicide
and Rodenticide Act, as amended)

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

[Faint, illegible text in the registrant name and address field]

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.

Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL

DATE

5. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e) our release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

Sincerely yours,

John H. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (TS-767C)