



Ucarcide[®] 602

Sterilizing and
Disinfecting Solution

ACCEPTED
FEB 18 1986
Under the Fungicide, Insecticide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 10352-29

ACTIVATOR VIAL

For use with
UCARCIDE[®] 602
Solution

**UNION CARBIDE CORPORATION
SPECIALTY CHEMICALS DIVISION**

Danbury, Connecticut 06817
Offices in principal cities

UCARCIDE is a registered trademark of
Union Carbide Corporation

NET WT. 34 GRAMS

DIRECTIONS

Add entire contents of
this vial to one gallon
of UCARCIDE[®] 602
solution. Mix well.
UCARCIDE[®] 602
solution is now green
and ready for use.

DANGER

In case of contact:
For eyes, immediately
flush with plenty of
water. Call a physician.
For skin, flush with
water.

PRECAUTIONARY STATEMENTS
HAZARDS TO HUMANS

DANGER

KEEP OUT OF REACH OF CHILDREN

Corrosive - Causes irreversible eye damage. May cause skin sensitization.
Do not get in eyes, on skin or on clothing. Wear goggles, protective clothing, and rubber gloves.
Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash before reuse.

ACCEPTED

FEB 18 1986

Under the Fungicide, Insecticide and Rodenticide Act as amended, for the pesticide registered under EPA Reg. No. 10352-29

ENVIRONMENTAL HAZARDS

Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

STORAGE AND DISPOSAL

DO NOT REUSE EMPTY CONTAINER (BOTTLES, CANS, BUCKETS).
WRAP CONTAINER AND PLACE IN TRASH.



Ucarbide[®] 6

Sterilizing and Disinfecting Solution

A RAPID-ACTING SOLUTION FOR DISINFECTION AND STERILIZATION OF MEDICAL, SURGICAL, AND DENTAL INSTRUMENTS, RESPIRATORY THERAPY AND ANESTHESIA EQUIPMENT.

Broad Spectrum Antimicrobial

Sporicidal • Pseudomonacidal • Virucidal •
Bactericidal • Tuberculocidal • Fungicidal

Active Ingredient: Glutaraldehyde
Inert Ingredients

20%
98.0%
100.0%

KEEP OUT OF REACH OF CHILDREN

DANGER

STATEMENT OF PRACTICAL TREATMENT

- If in eyes: Immediately flush with plenty of water for at least 15 minutes. Get immediate medical attention.
- If on skin: Wash with plenty of soap and water. Get medical attention.
- If swallowed: **DO NOT INDUCE VOMITING.** Give at least two glasses of water. Seek medical advice with or without emesis.
- If inhaled: Remove to fresh air. If breathing is difficult, administer oxygen. If symptoms persist, call a physician.
- Note to Physician:** Probable mucosal damage may contraindicate the use of contact lenses.

UNION CARBIDE CORPORATION • SPECIALTY CHEMICALS

Danbury, CT 06817 • Offices in Principal Cities
EPA Reg. No. 10352-29
UCARCIDE is a registered trade mark
of Union Carbide Corporation

NET VOLUME: One Gallon (128 fl. oz.)



Ucarcide® 602

Sterilizing and Disinfecting Solution

A RAPID ACTING SOLUTION FOR DISINFECTION AND STERILIZATION OF MEDICAL, SURGICAL, AND DENTAL INSTRUMENTS, RESPIRATORY THERAPY AND ANESTHESIA EQUIPMENT

Broad Spectrum Antimicrobial
Sporicidal • Pseudomonacidal • Virucidal*
Bactericidal • Tuberculocidal • Fungicidal

Active Ingredient: Glutaraldehyde 2.0%
Inert Ingredients 98.0%
100.0%

KEEP OUT OF REACH OF CHILDREN

DANGER

STATEMENT OF PRACTICAL TREATMENT

- In eyes:** Immediately flush with plenty of water for at least 15 minutes. Get immediate medical attention.
 - On skin:** Wash with plenty of soap and water. Get medical attention.
 - Swallowed:** **DO NOT INDUCE VOMITING.** Drink at least two glasses of water. Seek medical advice with urgency.
 - Inhaled:** Move to fresh air. If breathing is difficult, administer oxygen. If symptoms persist, call a physician.
- Note to Physician:** Probable mucosal damage may contraindicate the use of gastric lavage

UNION CARBIDE CORPORATION • SPECIALTY CHEMICALS DIVISION

Way, CT 06817 • Offices in Principal Cities
Reg. No. 10352-29
UCARCID is a registered trade mark
of Union Carbide Corporation

EPA Est. No. 10352-29-WV-2
UCC No. 3987-1F
Made in U.S.A.

* See Recommendations for Use

NET VOLUME: One Gallon (128 fl. oz.)

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

DIRECTIONS FOR ACTIVATING

THE ATTACHED ACTIVATOR VIAL MUST BE ADDED TO THE SOLUTION BEFORE THE PRODUCT IS EFFECTIVE.

Add entire contents of attached activator package to this container of UCARCID® 602 solution. Mix well. Do not dilute. UCARCID® 602 solution is now green and ready for use. The shelf life of the activated, unused stock solution is efficacious for 28 days.

For disinfecting or sterilizing medical instruments and equipment, thoroughly clean, rinse and rough dry items before immersing in full strength activated UCARCID® 602 solution. Flush and thoroughly cleanse lumen of any hollow instrument. Surfactant action will facilitate penetration into hard-to-reach areas.

Disinfection: Immerse item completely for a minimum of 10 minutes at room temperature to destroy vegetative pathogens including *Mycobacterium tuberculosis*, *Pseudomonas aeruginosa* and *viruses* (Coxsackievirus Type B-1, Herpes Simplex Type 1, Influenza Type A-2 (Hong Kong), Poliovirus Type 1) on inanimate surfaces.

Sterilization: Immerse item completely for a minimum of 10 hours at room temperature to destroy resistant pathogen spores including *Clostridium sporogenes* and *Bacillus subtilis*.

Remove items from UCARCID® 602 solution using aseptic technique and rinse thoroughly with sterile water. Discard the used portion of the solution after each day's usage.

LIMITED WARRANTY AND DISCLAIMER

The manufacturer warrants (a) that this product conforms to the chemical description on the label, (b) that this product is reasonably fit for the purposes set forth in the directions for use when it is used in accordance with such directions, and (c) that the directions, warnings and other statements on this label are based upon responsible experts' evaluation of reasonable tests of effectiveness and of toxicity to laboratory animals. Tests have not been made on all varieties or in all states or under all conditions. THE MANUFACTURER NEITHER MAKES NOR INTENDS, NOR DOES IT AUTHORIZE ANY AGENT OR REPRESENTATIVE TO MAKE, ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, AND IT EXPRESSLY EXCLUDES AND DISCLAIMS ALL IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. This warranty does not extend to, and the Buyer shall be solely responsible for, any and all loss or damage which results from the use of this product in any manner which is inconsistent with the label directions, warnings or cautions.

BUYER'S EXCLUSIVE REMEDY AND MANUFACTURER'S OR SELLER'S EXCLUSIVE LIABILITY FOR ANY AND ALL CLAIMS, LOSSES, DAMAGES, OR INJURIES RESULTING FROM THE USE OR HANDLING OF THIS PRODUCT, WHETHER OR NOT SUCH LIABILITY IS BASED IN CONTRACT, NEGLIGENCE, STRICT LIABILITY IN TORT OR OTHERWISE, SHALL BE LIMITED, AT THE MANUFACTURER'S OPTION, TO REPLACEMENT OF, OR THE REPAYMENT OF THE PURCHASE PRICE FOR, THE QUANTITY OF PRODUCT WITH RESPECT TO WHICH DAMAGES ARE CLAIMED. IN NO EVENT SHALL MANUFACTURER OR SELLER BE LIABLE FOR SPECIAL, INDIRECT OR CONSEQUENTIAL DAMAGES RESULTING FROM THE USE OR HANDLING OF THIS PRODUCT.

Date of Activating

After 28 Days

Date to Discard