

DIRECTIONS:

To prepare a one-quart solution pour 10 ml (a 10-30 ml measuring cup is included) into 1 quart of tap or distilled water. To prepare a one-gallon solution pour 40 ml into a gallon of tap or distilled water. In known hard water areas always use distilled water. Be sure to wash instruments removing all blood, debris and soap. Immerse instruments for at least 15 minutes and remove ready to use. Chemical disinfection should not be relied upon to disinfect hinged instruments, those with deep narrow crevices, hypodermic needles and syringes. However, to store such instruments that have been heat sterilized, a 1.750 solution (40 ml of CREDO GERMICIDAL CONCENTRATE to 1 gallon of water) which has not been previously used may be employed. Precautions should be observed to prevent contamination of these solutions with resistant organisms. Quaternary ammonium type disinfectants will not eliminate spore bearing organisms, mycobacterium tuberculosis or viral hepatitis and any instruments thought to be contaminated by such organisms should be sterilized by autoclaving.

Do not use CREDO GERMICIDAL CONCENTRATE for disinfection of optical instruments with cemented parts.

CREDO GERMICIDAL CONCENTRATE is a formula consisting of very efficient germicidal concentrate with an anti-rust factor included and has been shown to be effective in chemical disinfection of professional and hospital instruments and equipment. In the recommended dilutions it is odorless, non-staining and will inhibit rust formation. Its extreme wetting action and low surface tension give it a distinct advantage for disinfection of hospital and dental equipment.

DANGER:

Corrosive. Do not get in eyes, skin or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed. Avoid contamination of food.

FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatine solution or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN:

Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

Rinse empty container thoroughly with water and discard it.

Contains Sodium Nitrite.

CREDO

**GERMICIDAL
CONCENTRATE**

MUST BE DILUTED

ACTIVE INGREDIENTS:

n-Alkyl (C₁₂50%, C₁₄30%,
C₁₀17%, C₈-C₁₆3%) Dimethyl
Benzyl Ammonium Chloride 12.9%
Isopropanol 10.0%

INERT INGREDIENTS:

(including 19.2% Sodium
Nitrite as a rust inhibitor) 77.1%
100.0%

DANGER

Keep out of reach of children

(See First Aid and other
precautions on back panel)

NET CONTENTS: ONE QUART

EPA Reg. No. 10451-3

CREDO COMPANY
1708 CHASE DRIVE, FENTON, MO. 63026