

IODINE TYPE DETERGENT DISINFECTANT

CAUTION!

KEEP OUT OF REACH OF CHILDREN!

"HARMFUL IF SWALLOWED. Causes skin irritation. Avoid contact with skin and eyes. In case of contact flush skin or eyes with plenty of water for at least 15 minutes. For eyes get medical attention. Avoid contamination of food."

NET CONTENTS 1 GALLON

Manufactured by

CLARKSON LABORATORIES, INC.

San Francisco, California 94124

U.S. DEPARTMENT OF AGRICULTURE

REGISTRATION #

DIRECTIONS FOR USE

FOR BATHROOM AND KITCHEN — dilute 3 ounces of compound to 5 gallons with water. Remove heavy dirt and grease. Wash with rag, sponge or brush. Can be used on walls, floors and fixtures.

FOR COLD DISINFECTION OF MEDICAL AND SURGICAL INSTRUMENTS — dilute 5 ounces of compound to 5 gallons of cold water. Clean instruments to remove all adhering blood and serous exudates before immersion in solution.

FOR DISHES — use 1 ounce to 5 gallons of water.

Not effective when used with alkaline type cleaners such as ammonia and strong soaps.

ACTIVE INGREDIENTS

Percent by Weight

Nonylphenoxy poly (ethyleneoxy) ethanol iodine complex (providing 1.75 titratable iodine)	8.75%
Phosphoric Acid	6.50%
Nonylphenoxy (ethyleneoxy) ethanol	2.50%

INERT INGREDIENTS

Water	82.25%
-------------	--------

