

(FRONT PANEL)

ISOTEX DISINFECTING TOWELETTES*

A dual chain quaternary ammonium/high level alcohol solution impregnated
non-woven disposable towelette

TUBERCULOCIDAL VIRUCIDAL BACTERICIDAL FUNGICIDAL

ACTIVE INGREDIENTS

n-alkyl (60% C ₁₄ 30% C ₁₆ 5% C ₁₂ 5% C ₁₈)dimethyl benzyl ammonium chloride	0.12%
n-alkyl (68% C ₁₂ 32% C ₁₄)dimethyl ethyl benzyl ammonium chloride	0.12%
Isopropyl alcohol	63.25%

INERT INGREDIENTS

TOTAL INGREDIENTS

36.51%
100.00%

Inert Ingredients do not include towelette

For use in: CRITICAL CARE AREAS such as hospitals, laboratories, clinics, nursing homes,
dental suites, intensive care, emergency units and veterinary offices.

Kills HIV-1 (AIDS Virus) on precleaned environmental surfaces or objects previously soiled with
blood or body fluids in 1 minute at room temperature (20-25°C) in health care settings or other
settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood
or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be
associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-
1)(associated with AIDS).

Isotex 70 is tuberculocidal against Mycobacterium bovis BCG by the Tuberculocidal Activity test
within 1 minute at 20°C; virucidal against Herpes Simplex Virus type 2 and Adenovirus type 2 by
an EPA approved virucidal test in 1 minute at 20°C; fungicidal against Trichophyton
mentagrophytes by the AOAC Fungicidal Activity test in 4 minutes at 20°C and bactericidal
against Staphylococcus aureus, Pseudomonas aeruginosa, and Salmonella choleraesuis by the
AOAC Use Dilution test in 6 minutes at 20°C.

WARNING

KEEP OUT OF REACH OF CHILDREN

See back panel for additional precautionary statements


NET CONTENTS: 16 FLUID OZ.**

160/5.75 X 8.0 IN. HERBAL SCENTED TOWELETTES**

EPA REG. NO. 10492-4

EPA EST. NO. 56952-AR-01

- * Additional name is DisCide Disinfecting Towelettes
- ** Alternate net weight is 25 oz; Alternate towelette count and size is 160/6 X 6.75 in.
Alternate net weight is 25.5 oz; Alternate towelette count and size is 60/10.5 X 10.5 in.
Alternate net weight is 30.5 oz; Alternate towelette count and size is 60/11 X 12 in.

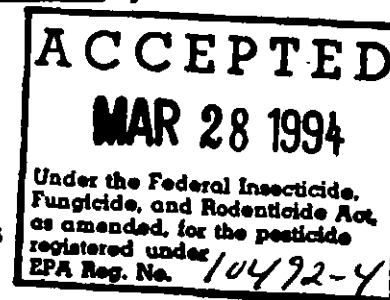

BEST AVAILABLE COPY

292

(BACK PANEL)

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

TO START FEED: Remove large cover from container. From center of towelette roll, pull up a towelette corner, twist into a point and thread through hole in container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep small center cap closed to prevent moisture loss.

TO DISINFECT: Completely preclean surfaces to be disinfected. Then, thoroughly wet the surfaces with the towelette and allow to remain wet for 6 minutes.

For disinfection of inanimate, precleaned, hard, non-porous surfaces such as chairs, counter tops, trays, drawer pulls, carts, baskets, tables in operating rooms, intensive care and emergency units as well as in dental or veterinary offices. Use on stainless steel, glass, formica and other similar surfaces. Discard towelette after use.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS) FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

PERSONAL PROTECTION: When using the towelettes, wear disposable latex gloves, protective gown, face mask and eye covering, as appropriate, when handling HIV-1 infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant.

CONTACT TIME: Thoroughly wet surface. Allow to air dry. Although the efficacy of a 1 minute contact time has been shown to be adequate against HIV-1 (AIDS virus), this time would not be sufficient for control of other organisms.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS


Hazards to humans and domestic animals: WARNING. Avoid contact with eyes. In case of eye contact, flush with plenty of water for at least 15 minutes. If irritation persists, call a physician.

Physical or chemical hazards: Flammable. Keep away from heat or open flame.

DISPOSAL: Do not reuse towelette or empty container. Wrap towelette container and discard in trash.

PALMERO SALES COMPANY, INC., 120 GOODWIN PLACE, STRATFORD, CT 06497
For technical information or in case of emergency, call 1-800-344-6424.

PATENT 4,464,293

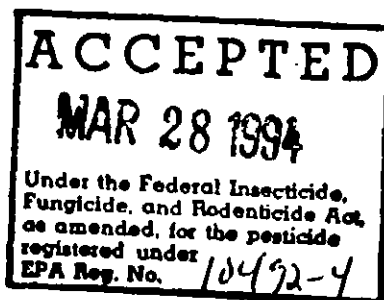

BEST AVAILABLE COPY