

10492-4

11-20-2003

1/2

Isotex 70 Disinfecting Towelettes

Kills A Wide Range of Pathogens In One Minute*

Bactericidal ♦ Tuberculocidal ♦ Virucidal ♦ Fungicidal ♦ Herpeicidal**

Isotex 70 Disinfecting Towelettes are herbal scented, non-woven disposable towelettes impregnated with a dual chain quaternary ammonium/high level alcohol solution.

Use in Health Care & Alternate Care Settings such as hospitals, intensive care units, emergency medical settings, central supply, laboratories, clinics, nursing homes and dental suites.

Isotex 70 Disinfecting Towelettes are effective against the following pathogens on hard non-porous, contact surfaces when used as directed:

- * ^{added} HIV-1** (AIDS virus) (1 min.) ^{added}
- * Adenovirus type 2** (1 min.)
- * Herpes Simplex Virus type 2** (genital herpes virus) (1 min.)
- * Vancomycin Resistant Enterococcus faecalis (VRE) (1 min.)
- * Methicillin Resistant Staphylococcus aureus (MRSA) (1 min.)
- * Staphylococcus aureus (Staph) (1 min.)
- * Pseudomonas aeruginosa (1 min.)
- * Salmonella choleraesuis (1 min.)
- * Escherichia coli (E. coli) (1 min.)
- * Trichophyton mentagrophytes (Athlete's Foot fungus) (1 min.)
- * Mycobacterium bovis (Tuberculosis bacteria) (5-minute contact time required)*

ACTIVE INGREDIENTS

n-alkyl (80% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride..... 0.12%

n-alkyl (68% C12, 32% C14) dimethyl ethyl benzyl ammonium chloride..... 0.12%

Isopropyl alcohol 63.25%

OTHER INGREDIENTS..... **36.51%**

TOTAL INGREDIENTS..... **100.00%**

WARNING

**KEEP OUT OF REACH
OF CHILDREN**

See back panel for additional
precautionary statements

ACCEPTED

NOV 20 2003

Under the Federal Insecticide, Fungicide, and
Rodenticide Act as amended, for the
pesticide, registered under
EPA Reg. No. 10492-4


EPA Reg. No. 10492-4

EPA Est. No. 56952-WI-001

Isotex 70 Disinfecting Towelettes

meet the disinfection requirements of
OSHA's Bloodborne Pathogen Standard (29 CFR §1910.1030).

NET CONTENTS:

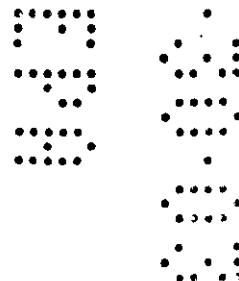

added 2/2

DIRECTIONS FOR USE:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

General Precautions and Restrictions

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contact intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

[dispenser] **TO START FEED:** Remove large cover from container and discard seal. From center of towelette roll, pull up a towelette corner, twist into a point and thread through hole in container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep small center cap closed to prevent moisture loss.

TO DISINFECT: Completely preclean surfaces to be disinfected. An Isotex 70 Disinfecting Towelette can be used for this purpose. Then, thoroughly wet the surface with another towelette and allow to remain wet for 1 minute. When used as a tuberculocidal disinfectant, allow the surface to remain visibly wet for 5 full minutes at 20° C.

For Disinfection of: inanimate, precleaned, hard, non-porous surfaces such as chairs, counter tops, drawer pulls, carts, baskets, tables, cabinets, telephones and other related hard surfaces in hospitals, operating rooms, intensive care and emergency units, dental or veterinary offices, surgery, recovery, anesthesia, X-Ray, Cat. Lab, orthopedics, newborn nursery, respiratory therapy, radiology, central supply, emergency medical vehicles, police and fire vehicles, correctional facilities, schools and school medical facilities, school buses, and day care. Use on stainless steel, glass, formica and other similar surfaces. Use care when using on varnished surfaces, plexi-glass or leather. **DO NOT USE AS A BABY WIPE.** Discard towelette after use.

deleted "food-contact" and "Note"

Kills HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY

IDS in health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS) FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

PERSONAL PROTECTION: When using this disinfecting towelette, wear disposable latex gloves, protective gown, face mask and eye covering, as appropriate, when handling HIV-1 infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant. An Isotex 70 Disinfecting Towelette can be used for this purpose.

CONTACT TIME: Thoroughly wet surface with a new Isotex 70 Disinfecting Towelette. Allow to air dry. The efficacy of a 1 minute contact time has been shown to be adequate against HIV-1 (AIDS virus).

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS:

Hazards to Humans and Domestic Animals: WARNING. Causes substantial but temporary eye injury. Do not get in eyes or on skin. Wear protective eyewear. Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash clothing before reuse.

Physical or Chemical Hazards: Flammable. Keep away from heat or open flame.

ACCEPTED
added
NOV 20 2003

FIRST AID**If in eyes:**

- Hold eye open and rinse slowly and gently with water for 15-20 minutes.
- Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
- Call a poison control center or doctor for treatment advice.

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. 10492-4

HOT LINE NUMBER

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact Chem•Tel Inc. at 1-800-255-3924.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

PESTICIDE STORAGE: Store away from heat.

CONTAINER DISPOSAL: [dispenser] Do not reuse towelette or empty container. Wrap towelette container and discard in trash. [foil packages] Do not reuse towelette or empty packet. Dispose of used towelette and packet in trash.

For technical information on DisCide Ultra Spray, Towelettes, Aerosols, and DisAseptic Antibacterial Hand Asepsis Towelettes, call 1-800-344-6424 or fax 203-377-8988.

modified