

DisCide ULTRA Disinfecting Spray

[A One-Step*] Disinfectant - [Cleaner - Decontaminant]

Kills All Listed Pathogens In One Minute

Bactericidal ♦ Tuberculocidal ♦ Virucidal ♦ Fungicidal*** ♦ Herpecidal**

DisCide ULTRA Disinfecting Spray is herbal scented, dual chain quaternary ammonium/high level alcohol solution.

[Healthcare] Use in Health Care & Alternate Care Settings such as hospitals, intensive care units, emergency medical settings, central supply, laboratories, clinics, nursing homes, and dental suites where control of the cross-contamination hazards between treated surfaces is required.

[Consumer] Use on hard, non-food-contact surfaces in kitchens, food service areas and bathrooms.

[Consumer] Use on surfaces such as shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, door knobs and handles, other related hard, non-food-contact surfaces.

DisCide ULTRA Disinfecting Spray is effective against the following pathogens on hard non-porous, contact surfaces when used as directed:

- Hepatitis B Virus** (HBV) (1 min.)^a
- Hepatitis C Virus** (HCV) (1 min.)^a
- Human Coronavirus** (1 min.)^a
- HIV-1** (AIDS virus) (1 min.)^a
- Adenovirus type 2** (1 min.)^a
- Herpes Simplex Virus type 2** (genital herpes virus) (1 min.)^{a, b}
- Vancomycin Resistant Enterococcus faecalis (VRE) (1 min.)^a
- Methicillin Resistant Staphylococcus aureus (MRSA) (1 min.)^a
- Staphylococcus aureus (Staph) (1 min.)^{a, b}
- Pseudomonas aeruginosa (1 min.)^{a, b}
- Salmonella choleraesuis (1 min.)^{a, b}
- Escherichia coli (E. coli) (1 min.)^{a, b}
- Mycobacterium bovis (Tuberculosis bacteria) (1 min. at 20° C)^a
- Trichophyton mentagrophytes (Athlete's Foot fungus) (1 min.)^{a, b}

[^b Consumer-related claim; footnote will not be included on final printed label.]

ACTIVE INGREDIENTS

n-alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride	0.12%
--	-------

n-alkyl (68% C12, 32% C14) dimethyl ethyl benzyl ammonium chloride	0.12%
--	-------

Isopropyl alcohol	63.25%
-------------------------	--------

OTHER INGREDIENTS	36.51%
--------------------------------	---------------

TOTAL INGREDIENTS	100.00%
--------------------------	----------------

**KEEP OUT OF REACH OF CHILDREN
WARNING**

See back panel for additional precautionary statements

*A precleaning step is required for HIV-1, HBV, and HCV.

***A precleaning step is required.

[^a Healthcare-related claim; footnote will not be included on final printed label.]

EPA Reg. No. 10492-5

EPA Est. No. 8155-OH-001

DisCide ULTRA Disinfecting Spray
meets the disinfection requirements of
OSHA's Bloodborne Pathogen Standard (29 CFR §1910.1030)

ACCEPTED

OCT 4 2005

Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide, registered under EPA Reg. No. 10492-5

NET CONTENTS:

DIRECTIONS FOR USE:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

[Entire paragraph must be used for healthcare-related claims]

General Precautions and Restrictions

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contact intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

TO CLEAN, DECONTAMINATE, AND DISINFECT: Thoroughly wet the surface with DisCide ULTRA Disinfecting Spray and allow to remain wet for 1 minute. **[The following statement will be included when making claims against Trichophyton mentagrophytes.]** When used as a fungicidal disinfectant, preclean the surfaces to be disinfected. DisCide ULTRA Disinfecting Spray can be used for this purpose. Then, thoroughly wet the surface with DisCide ULTRA Disinfecting Spray and allow to remain wet for 1 minute.

[Healthcare] For Disinfection of: inanimate, hard, non-porous surfaces such as chairs, stools, counter tops, drawer pulls, door knobs and handles, carts, baskets, tables, cabinets, lighting fixtures, bassinets, bed rails, incubators, physical therapy equipment, shower stalls, toilet exteriors and toilet seats, telephones and other related hard surfaces in hospitals, operating and surgical rooms, intensive and critical care units, emergency units, dental and veterinary offices, surgery, recovery, anesthesia, X-Ray, Cat. Lab, blood collection and donation centers, isolation units, orthopedics, newborn nursery, respiratory therapy, radiology, pediatrics, physical therapy, rehabilitation, ophthalmology, optometry, hospices, central supply, emergency medical vehicles, police and fire vehicles, correctional facilities, funeral homes, morgues, spas, health clubs, salons, schools and school medical facilities, school buses, day care and non-food-contact surfaces.

[Consumer] For Disinfection of: inanimate, hard, non-porous kitchen/food service and bathroom surfaces such as counter tops, drawer pulls, carts, baskets, tables, cabinets, door knobs and handles, appliance exteriors, shower stalls, toilet exteriors and toilet seats, bathroom and lighting fixtures, sinks, flooring, walls, vanities, diaper pails, trash cans, telephones, and other related hard, non-food-contact surfaces.

[Consumer] For Disinfection of: inanimate, hard, non-porous surfaces such as shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, door knobs and handles, and other related hard, non-food-contact surfaces.

Use on stainless steel, glass, formica and other similar surfaces. Use care when using on varnished surfaces, plexi-glass or other.

[If directions for HIV, HBV or HCV are included, the following two paragraphs must be used.]

Kills HIV-1, HBV, and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV), and Hepatitis C Virus (HCV).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS), HBV, and HCV FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

PERSONAL PROTECTION: Wear disposable latex gloves, protective gown, face mask and eye covering, as appropriate, when handling HIV-1, HBV, and HCV infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant. DisCide ULTRA Disinfecting Spray can be used for this purpose.


CONTACT TIME: Thoroughly wet surface with DisCide ULTRA Disinfecting Spray. Allow to air dry. The efficacy of a 1 minute contact time has been shown to be adequate against HIV-1 (AIDS virus), HBV, and HCV.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS:

Hazards to Humans and Domestic Animals: WARNING. Causes substantial but temporary eye injury. Do not get in eyes or on clothing. Wear protective eyewear. Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash clothing before reuse.

Physical or Chemical Hazards: Flammable. Keep away from heat or open flame.

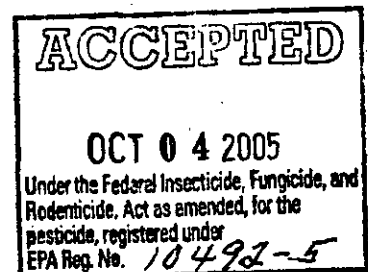

3 8 3

FIRST AID

If in eyes:

- Hold eye open and rinse slowly and gently with water for 15-20 minutes.
- Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
- Call a poison control center or doctor for treatment advice.

HOT LINE NUMBER

Keep the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact Chem•Tel Inc. at 1-800-255-3924.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

PESTICIDE STORAGE: Store away from heat.

CONTAINER DISPOSAL: Do not reuse empty container. Discard in trash.

For technical information on DisCide ULTRA Spray, Towelettes, Aerosols, and DisAseptic Antibacterial Hand Asepsis Towelettes, call 1-800-344-6424 or fax 203-377-8988.
PALMERO HEALTH CARE, 120 GOODWIN PLACE, STRATFORD, CT 06615
www.palmerohealth.com

ACCEPTED
OCT 04 2005
Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. 10492-5