


DIRECTIONS FOR ASSEMBLY


1. FILL OUT THERMOMETER I.D. CARD AND PLACE ON STEM

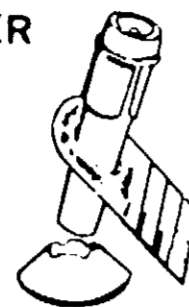


2. PLACE STEM IN BASE.

3. POUR BENZALKONIUM CHLORIDE SOLUTION PACKET U.S.P. INTO HOLDER.

4. PRESS INSERT DISC INTO TOP OF STEM.

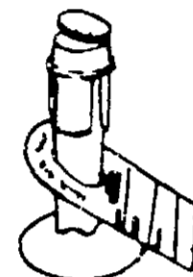


5. INSERT THERMOMETER THROUGH HOLE IN DISC.

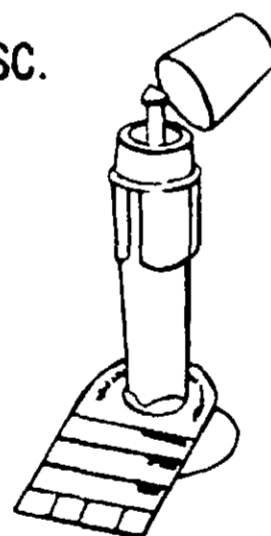

6. PLACE DUST COVER OVER THERMOMETER.

7. FOR WALL MOUNTING, DISCARD BASE AND SLIP SLOTTED COLLAR ONTO MOUNTING BRACKET.

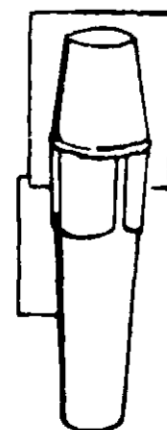

U.S.D.A. REG. NO. 10586-3

U.S. PAT. D212-043

DEAR FRIEND,

Kenwood temp pak
THIS IS YOUR PERSONAL THERMOMETER SET FOR YOUR USE. PLEASE FEEL FREE TO TAKE IT HOME WITH YOU, ALONG WITH OUR SINCERE WISHES FOR A HEALTHY AND SPEEDY RECOVERY.

ONE

REORDER
93-1431

Kenwood®

temp pak

RECTAL

THERMOMETER SET

- STUBBY TIP, RED TRIANGLE TOP RECTAL THERMOMETER
- THERMOMETER HOLDER
- LUBRICATING JELLY PACKET
- PATIENT THERMOMETER I.D. CARD
- BENZALKONIUM CHLORIDE SOLUTION PACKET U.S.P.

ACTIVE INGREDIENT —

Alkyl* dimethyl benzyl ammonium chlorides (benzalkonium chloride U.S.P.) 0.13%

INERT INGREDIENT —

membrane filtered water 99.87%

* (C₁₂ 67%, C₁₄ 25%, C₁₆ 7%, C₁₂C₁₄C₁₆ 1%) dimethyl benzyl ammonium chloride.

CAUTION

Keep Out of Reach of Children

MFD BY  **LABORATORIES, INC.**
JACKSON, WISCONSIN 53037

Directions:

1. Fill out patient thermometer I.D. card with patient's name and room number. Slide patient identification card toward top of holder and place in base, or fold card at scoremark and slip into wall mounting bracket. Spaces at bottom of card may be used to indicate dates the Benzalkonium Chloride Solution Packet is changed.
2. Open Benzalkonium Chloride Solution Packet and pour solution into thermometer holder.
3. Place thermometer holder disc on top of holder and insert thermometer through hole.
4. Cover thermometer with protective dust cover.
5. Insert thermometer into lubricating jelly packet to lubricate. Packet may be re-used for same patient by folding over end of packet after each use.

ONE

REORDER
93-1431