

ORAL
PATIENT NAME

ROOM DATE

DIRECTIONS FOR ASSEMBLY

1. FILL OUT THERMOMETER I.D. CARD AND PLACE ON STEM
2. PLACE STEM IN BASE
3. POUR BENZALKONIUM CHLORIDE SOLUTION PACKET U.S.P. INTO HOLDER.
4. PRESS INSERT DISC INTO TOP OF STEM
5. INSERT THERMOMETER THROUGH HOLE IN DISC
6. PLACE DUST COVER OVER THERMOMETER
7. FOR WALL MOUNTING, DISCARD BASE AND SLIP SLOTTED COLLAR ONTO MOUNTING BRACKET

U.S.D.A. REG. NO. 10586-4
U.S. PAT. D212 043

DEAR FRIEND:
THIS IS YOUR PERSONAL **Kenwood temp pak** THERMOMETER SET FOR YOUR USE. PLEASE FEEL FREE TO TAKE IT HOME WITH YOU, ALONG WITH OUR SINCERE WISHES FOR A HEALTHY AND SPEEDY RECOVERY.

ONE REORDER
93-1430

Kenwood
temp pak

ORAL
THERMOMETER SET

- STUBBY TIP, TRIANGLE TOP ORAL THERMOMETER
- THERMOMETER HOLDER
- PATIENT THERMOMETER I.D. CARD
- BENZALKONIUM CHLORIDE SOLUTION PACKET U.S.P.

ACTIVE INGREDIENT —
Alkyl dimethyl benzyl ammonium chlorides (benzalkonium chloride U.S.P.) 0.13%

INERT INGREDIENT —
membrane filtered water 99.87%

(C. 67%, C. 25%, C. 7%, C.C.C. 1% dimethyl benzyl ammonium chloride)

CAUTION
Keep Out of Reach of Children

MADE BY
BADGER LABORATORIES, INC.
JACKSON WISCONSIN 53021

Directions:

1. Fill out patient thermometer I.D. card with patient's name and room number. Slide patient identification card toward top of holder and place in base, or fold card at scoremark and slip into wall mounting bracket. Spaces at bottom of card may be used to indicate date.
2. Open Benzalkonium Chloride Solution Packet and pour solution into thermometer holder.
3. Place thermometer holder disc on top of holder and insert thermometer through hole.
4. Cover thermometer with protective dust cover.

ACCEPTED
Jan 4-1971

UNDER THE PROVISIONS OF THE FEDERAL FOOD, DRUG AND COSMETIC ACT, THIS PRODUCT IS REGISTERED UNDER NO. 10586-4

ONE REORDER
93-1430

Kenwood
temp pak

ORAL
THERMOMETER SET

MADE BY
BADGER LABORATORIES, INC.
JACKSON WISCONSIN 53021