

TEAR HERE

**bafor.**

**BENZALKONIUM  
CHLORIDE**

U.S.P.

**AQUEOUS  
SOLUTION 1:750**

**CAUTION:** mixing benzalkonium chloride solution with ordinary soaps and with anionic detergents may decrease or destroy the bactericidal activity of the solution.

**.67 FL. OZ.**

**ACTIVE INGREDIENTS**

Alky<sup>®</sup> dimethyl benzyl Ammonium chlorides (benzalkonium chloride USP) . . . . . 0.13%

**INERT INGREDIENT**

membrane filtered water 99.87%

\*C<sub>12</sub> 67% C<sub>14</sub> 25% C<sub>16</sub> 7% C<sub>8</sub> C<sub>10</sub> C<sub>18</sub>  
1% dimethyl benzyl ammonium chloride

MFD. BY

**BAGGER**

**LABORATORIES, INC.  
JACKSON, WISCONSIN 53037**

**balfor**<sup>®</sup> REORDER  
22-9960

**oral  
THERMOMETER  
KIT**

ORAL THERMOMETER,  
STUBBY TIP, TRIANGLE TOP  
THERMOMETER HOLDER  
IDENTIFICATION CARD  
BENZALKONIUM CHLORIDE  
SOLUTION

Active Ingredient—Alkyl\* dimethyl  
benzyl Ammonium chlorides (ben-  
zalkonium chloride USP) . . . 0.13%

Inert Ingredient — membrane fil-  
tered water . . . . . 99.87%

\* (C<sub>12</sub> 67%, C<sub>14</sub> 25%, C<sub>16</sub> 7%,  
C<sub>8</sub> C<sub>10</sub> C<sub>18</sub> 1%) dimethyl benzyl  
ammonium chloride.

U.S.D.A. REG. NO. 10586-6

MFD BY  **LABORATORIES, INC.**  
JACKSON, WISCONSIN 53037

**balfor**<sup>®</sup>

PATIENT NAME: \_\_\_\_\_

PATIENT ACCOUNT NO. \_\_\_\_\_

PATIENT ROOM NO. \_\_\_\_\_

MFD BY  **LABORATORIES, INC.**  
JACKSON, WISCONSIN 53037

**balfor**<sup>®</sup> REORDER  
22-9960

ACCEPTED


12-21-70

UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE  
FOR TOXIC CHEMICALS REGISTERED  
ED UNDER NO. 10586-6

**oral  
THERMOMETER  
KIT**

**CAUTION**

Keep Out of Reach of Children

MFD BY  **LABORATORIES, INC.**  
JACKSON, WISCONSIN 53037

**Directions:**

- 1) Open Benzalkonium Chloride Solution Packet and pour solution into thermometer holder.
- 2) Fill out patient identification card with patient's name and room number. Spaces at bottom of card may be used to indicate dates the Benzalkonium Chloride Solution Packet is changed.
- 3) Slide patient identification card down over barrel of holder to base.
- 4) Place cap on top of holder.
- 5) Insert thermometer through hole in cap into thermometer holder.