

PM31
10648-1
Pg. 1 of 2

Harvey®

VAPO-STERIL SOLUTION

Vapo-Steril is a sterilizing compound for use in Harvey Checkmate
containers (Models E, 4000, 4200, 5000, 6000, 7000, 8000, 900,
1200) for sterilizing dental endodontic instruments.

ACTIVE INGREDIENTS:	Ethanol	72.38%
	Formaldehyde	9.23%
INERT INGREDIENTS		27.38%
TOTAL		100.00%

Contains Methanol

KEEP OUT OF REACH OF CHILDREN

DANGER

See other precautionary statements on back panel

NET CONTENTS

300 ML. (1.0 FL. OZ.)

EPA Reg. No. 10040-1 EPA Reg. No. 10040-CA-1



Technology for Life

Harvey®

Manufactured by

NEO Biologic Company, 1777 East Henrietta Road, P.O. Box 23077, Rochester, New York 14602, U.S.A.

ACCEPTED

JUN 08 1980

Under the Federal Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 10648-1

BEST AVAILABLE COPY

PM31
10648-1
Pg. 20, 2

PRECAUTIONARY STATEMENT

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

DANGER

Causes eye damage and may cause skin irritation. Do not get in eyes, or on skin or on clothing. Wear goggles or face shield when handling. Harmful or fatal if swallowed. Avoid contamination of food.

STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Get medical attention if irritation persists.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

PHYSICAL AND CHEMICAL HAZARDS

FLAMMABLE Keep away from heat or open flame. **VOLATILE SOLVENTS** Do not use Vapo-Steril in steam sterilizers.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Use quantity of undiluted Vapo-Steril sufficient to fill reservoir tank. Refer to operating instructions accompanying each Harvey Chemistone.

STORAGE AND DISPOSAL


Store in an upright position in a cool, dry location. Do not reuse Vapo-Steril. Used Vapo-Steril may be flushed down drain with plenty of water in most areas. If in doubt, contact your local authorities.

CONTAINER DISPOSAL

Do not reuse container. Triple-rinse use empty container. Then offer container for recycling or remanufacturing, or purchase and disposal of it in a nearby landfill, or by incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

PESTICIDE DISPOSAL

Pesticide wastes are toxic. Improper disposal of empty pesticide or waste is a violation of Federal Law. If these wastes cannot be disposed of by use according to the other instructions, contact your State Pesticide or Environmental Control Agency or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

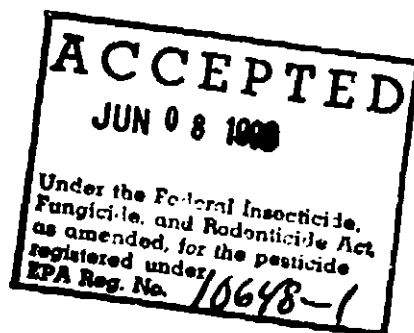

BEST AVAILABLE COPY

Harvey®

VAPO-STERIL[®] SOLUTION

Vapo-Steril is a sterilizing compound for use in Harvey Chemiclave sterilizers (Model E, 4000, 5000, 6000, 6800, 7000, 8000, 912, 1212) for sterilizing dental and surgical instruments.

ACTIVE INGREDIENTS: Ethanol	72.38%
Formaldehyde	0.23%
INERT INGREDIENTS	27.39%
TOTAL	<u>100.00%</u>

Contains Methanol

KEEP OUT OF REACH OF CHILDREN

DANGER

See other precautionary statements on back panel

NET CONTENTS

33.8 FL. OZ. (1.0 liter) U.S. MEASURE

EPA Reg. No. 10648-1 EPA Est. No. 10648-CA-1



MDI

Technology for Life

Harvey®

Manufactured by

MDI Biologic Company, 1777 East Hennetta Road, PO Box 23077 Rochester, New York 14602, U.S.A.

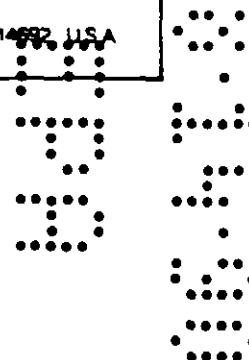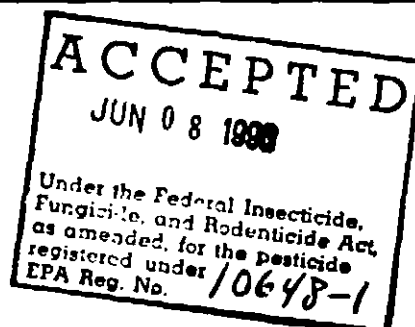

PRECAUTIONARY STATEMENTS**HAZARDS TO HUMANS AND DOMESTIC ANIMALS
DANGER**

Causes eye damage and may cause skin irritation. Do not get in eyes, or on skin or on clothing. Wear goggles or face shield when handling. Harmful or fatal if swallowed. Avoid contamination of food.

STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Get medical attention if irritation persists.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

PHYSICAL AND CHEMICAL HAZARDS

FLAMMABLE Keep away from heat or open flame. **VOLATILE SOLVENTS** Do not use Vapo-Steril in steam sterilizers.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Use quantity of undiluted Vapo-Steril sufficient to fill reservoir tank. Refer to operating instructions accompanying each Harvey Chemicleve.

STORAGE AND DISPOSAL



Store in an upright position in a cool, dry location. Do not reuse Vapo-Steril. Used Vapo-Steril may be flushed down drain with plenty of water in most areas. If in doubt, contact your local authorities.

CONTAINER DISPOSAL

Do not reuse container. Triple rinse (or equivalent) container. Then offer container for recycling or reconditioning, or puncture and dispose of it in a sanitary landfill, or by incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

PESTICIDE DISPOSAL

Pesticide wastes are toxic. Improper disposal of excess pesticide or mixture is a violation of Federal Law. If these wastes cannot be disposed of by use according to the label instructions, contact your State Pesticide or Environmental Control Agency or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

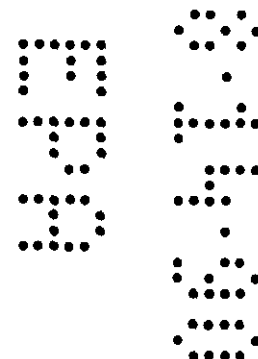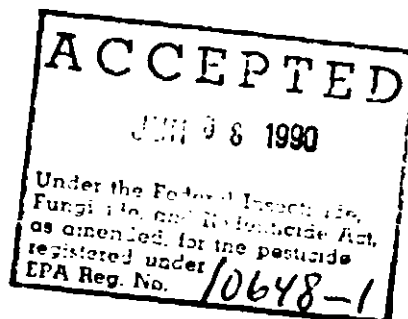



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, D.C. 20460

OFFICE OF
PESTICIDES AND TOXIC
SUBSTANCES

MDI Corporation
1777 E. Henrietta Road
Rochester, NY 14623

JUN 08 1990

Attn: Raymond E. Bergan

Subject: Product Name: Vapo-Steril Solution
EPA Reg. No.: 10648-1
Submission Dated: 3/20/90
EPA Received Date: 3/21/90


The amendment referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended is acceptable. A stamped copy of the label is enclosed for your record.

If you have any questions concerning this letter, please contact Martha DeLaney at (703) 557-6982.

Sincerely yours,

A handwritten signature in dark ink, appearing to read "John H. Lee".

John H. Lee
Product Manager (31)
Antimicrobial Program Branch
Registration Division (H7505C)

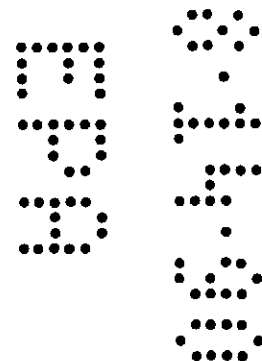

Harvey®

VAPO-STERIL[®] SOLUTION

Vapo-Steril is a sterilizing compound for use in Harvey Chemclave sterilizers (Model E, 4000, 5000, 5500, 6000, 7000, 8000, 912, 1212) for sterilizing dental and surgical instruments.

ACTIVE INGREDIENTS: Ethanol	72.38%
Formaldehyde	0.23%
INERT INGREDIENTS	27.39%
TOTAL	100.00%

Contains Methanol

KEEP OUT OF REACH OF CHILDREN

DANGER

See other precautionary statements on back panel

NET CONTENTS

33.8 FL. OZ. (1.9 liter) U.S. MEASURE

EPA Reg. No. 10648-1 EPA Est. No. 10648-CA-1



MDI

Technology for Life

Harvey®

Manufactured by

MDI Biologic Company, 1777 East Henrietta Road, PO Box 23077 Rochester, New York 14602, U.S.A.

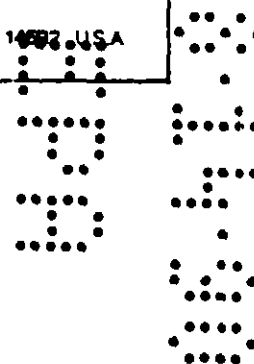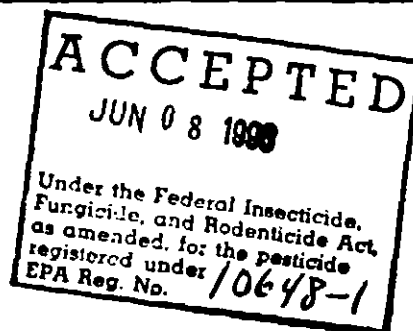

PRECAUTIONARY STATEMENTS

**HAZARDS TO HUMANS AND DOMESTIC ANIMALS
DANGER**

Causes eye damage and may cause skin irritation. Do not get in eyes, or on skin or on clothing. Wear goggles or face shield when handling. Harmful or fatal if swallowed. Avoid contamination of food.

STATEMENT OF PRACTICAL TREATMENT

In case of contact immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Get medical attention if irritation persists.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

PHYSICAL AND CHEMICAL HAZARDS

FLAMMABLE Keep away from heat or open flame. **VOLATILE SOLVENTS** Do not use Vapo-Steril in steam sterilizers.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Use quantity of undiluted Vapo-Steril sufficient to fill reservoir tank. Refer to operating instructions accompanying each Harvey Chemicleave.

STORAGE AND DISPOSAL



Store in an upright position in a cool, dry location. Do not reuse Vapo-Steril. Used Vapo-Steril may be flushed down drain with plenty of water in most areas. If in doubt, contact your local authorities.

CONTAINER DISPOSAL

Do not reuse container. Triple rinse (or equivalent) container. Then offer container for recycling or reconditioning, or puncture and dispose of it in a sanitary landfill, or by incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

PESTICIDE DISPOSAL

Pesticide wastes are toxic. Improper disposal of excess pesticide or mixture is a violation of Federal Law. If these wastes cannot be disposed of by use according to the label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

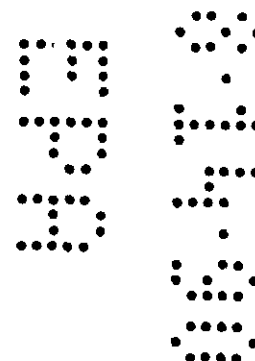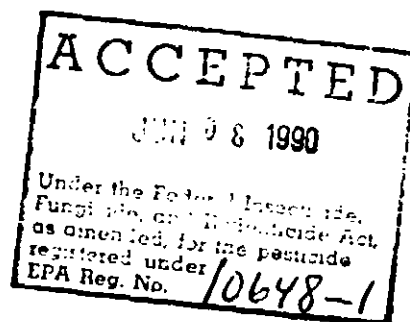



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, D.C. 20460

OFFICE OF
PESTICIDES AND TOXIC
SUBSTANCES

MDI Corporation
1777 E. Henrietta Road
Rochester, NY 14623

JUN 08 1990

Attn: Raymond E. Bergan

Subject: Product Name: Vapo-Steril Solution
EPA Reg. No.: 10648-1
Submission Dated: 3/20/90
EPA Received Date: 3/21/90

The amendment referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended is acceptable. A stamped copy of the label is enclosed for your record.

If you have any questions concerning this letter, please contact Martha DeLaney at (703) 557-6982.

Sincerely yours,

A handwritten signature in cursive script, appearing to read "John H. Lee".

John H. Lee
Product Manager (31)
Antimicrobial Program Branch
Registration Division (H7505C)

