

# Harvey®

## VAPO-STERIL® SOLUTION

Vapo-Steril is a sterilizing compound  
for use in Harvey Chemiclave sterilizers  
(Model E, 4000, 5000, 5500, 6000, 7000, 8000, 912, 1212)  
for sterilizing dental and surgical instruments.

<b>ACTIVE INGREDIENTS:</b>	
Ethanol	72.38%
Formaldehyde	0.23%
<b>INERT INGREDIENTS</b>	<b>27.39%</b>
<b>TOTAL</b>	<b>100.00%</b>

Contains Methanol  
KEEP OUT OF REACH OF CHILDREN

### DANGER

See other precautionary statements on back panel

### NET CONTENTS

33.8 FL. OZ. (1.0 liter) U.S. MEASURE  
EPA Reg. No. 10648-1 EPA Est. No. 10648-CA-1

**ACCEPTED**  
**MAR 29 1994**

Under the Federal Insecticide,  
Fungicide, and Rodenticide Act,  
as amended, for the pesticide  
registered under  
EPA Reg. No. 10648-1



Technology for Life

Harvey®

Manufactured for  
MDT Biologic Company, 1777 East Henrietta Road, P.O. Box 23077, Rochester, New York 14692, U.S.A.

### PRECAUTIONARY STATEMENTS

#### HAZARDS TO HUMANS AND DOMESTIC ANIMALS

### DANGER

Causes eye damage and may cause skin irritation. Do not get in eyes, or on skin or on clothing. Wear goggles or face shield when handling. Harmful or fatal if swallowed. Avoid contamination of food.

### STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Get medical attention if irritation persists.

### NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

### PHYSICAL AND CHEMICAL HAZARDS

FLAMMABLE. Keep away from heat or open flame. VOLATILE SOLVENTS. Do not use Vapo-Steril in steam sterilizers.

### DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Use quantity of undiluted Vapo-Steril sufficient to fill reservoir tank. Refer to operating instructions accompanying each Harvey Chemiclave.

### STORAGE AND DISPOSAL

**PROHIBITIONS:** Do not contaminate water, food, or feed by storage or disposal. Open dumping is prohibited. Store in an upright position in a cool, dry location. Do not reuse Vapo-Steril.

**PESTICIDE DISPOSAL:** Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

**CONTAINER DISPOSAL:** Do not reuse container. Triple rinse (or equivalent). Then offer for recycling or reconditioning, or puncture and disposal of in a sanitary landfill, or by other procedures approved by state and local authorities, by burning. If burned, stay out of smoke.

**BEST AVAILABLE COPY**