

ACCEPTED
MAY 24 1972
UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTER-
ED UNDER NO. 11563-1

**BENZALKONIUM
CHLORIDE
U.S.P.**

AQUEOUS SOLUTION 1:750

CAUTION

This product will not disinfect *pseudomonas aeruginosa* when used as directed.

FOR USE IN HOSPITALS

Thermometer Kits: To disinfect thermometer, use undiluted. Allow thermometer to remain immersed at least ten minutes. Rinse thermometer with potable water before use. **Surgical Instruments:** Preclean to remove all adhering blood and serous exudates prior to immersion in undiluted solution.

CAUTION

Mixing benzalkonium chloride solution with ordinary soaps and with anionic detergents may decrease or destroy the bactericidal activity of the solution.

Discard used package in a safe place.

MANUFACTURED BY
Liquipak Corporation
ALMA, MICHIGAN 48801

TEAR HERE

**BENZALKONIUM
CHLORIDE
U.S.P.**

AQUEOUS SOLUTION 1:750

CAUTION

KEEP OUT OF REACH OF CHILDREN. SEE OTHER PRECAUTIONS ON THE BACK PANEL.

2/3 Fluid Oz.

ACTIVE INGREDIENTS:

ALKYL (50% C₁₂, 30% C₁₄, 17% C₁₆, 3% C₁₈) DIMETHYL BENZYL AMMONIUM CHLORIDE 0.13%
INERT INGREDIENT: 99.87%

EPA REG. NO.
11563-1

MANUFACTURED BY
Liquipak Corporation
ALMA, MICHIGAN 48801