

USE DIRECTIONS

For Disinfecting portable or non-portable dental chairs, dental X-ray Therapy Equipment, and X-ray Equipment, etc.

For Disinfecting Instruments: Pre clean instrument and immerse in EQUIPMENT DISINFECTANT for at least ten (10) minutes.

For Disinfecting Larger Equipment: Wash equipment with water and scrub counters thoroughly with EQUIPMENT DISINFECTANT. Rinse and repeat if heavily soiled.

This product at its ready to use level is effective against Pseudomonas aeruginosa PRD 10. This has been substantiated by the AOAC Use Dilution Confirmation Test.

Fungicidal Properties


This product will pass the AOAC fungicidal test.

Conductivity

Conforms to NFPA Standards.

EPA REG. NO. 11662-2 KEEP FROM FREEZING

CLEAR-UNCOLORED



Insta Soak
BRAND
READY TO USE • NO MIXING REQUIRED

EQUIPMENT DISINFECTANT

Active ingredients:	
n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈)	
dimethyl benzyl ammonium chlorides	0.0625%
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides	0.0625%
Inert Ingredients:	99.8750%
CONTAINS SODIUM NITRITE	00.0000%

CAUTION: Keep out of reach of children.
Avoid contact with eyes. In case of eye irritation, flush with water.
Rinse container thoroughly with water and discard.

CONTENTS: 1 GALLON PRINTED IN U.S.A.

EQ
Gern
Labo

Ha

USE DIRECTIONS

For Disinfecting Operating Room and Laboratory Equipment, Inhalation Therapy Equipment, Anesthesia Equipment, etc.

For Disinfecting Instruments: Pre-clean instrument and immerse in EQUIPMENT DISINFECTANT for at least ten (10) minutes.

For Disinfecting Larger Equipment: Wash equipment such as tables, sinks, and counters thoroughly with EQUIPMENT DISINFECTANT. Rinse and repeat if heavily soiled.

This product at its ready to use level is effective against *Pseudomonas aeruginosa* PRD-10. This has been substantiated by the AOAC Use Dilution Confirmation Test.

Fungicidal Properties

This product will pass the AOAC fungicidal test.

Conductivity

Conforms to NFPA Standards.

EPA REG. NO. 11662-2

KEEP FROM FREEZING

CLEAR-UNCOLORED

INSTA SOAK

BRAND

READY TO USE • NO MIXING REQUIRED

EQUIPMENT DISINFECTANT

Active ingredients:

n-Alkyl (60% C₁₄, 30% C₁₆, 5% C₁₂, 5% C₁₈) dimethyl benzyl ammonium chlorides 0.0625%

n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl ethylbenzyl ammonium chlorides 0.0625%

Inert Ingredients: 99.8750%

CONTAINS SODIUM NITRITE 00.0000%

CAUTION:

Keep out of reach of children.
Avoid contact with eyes. In case of eye irritation, flush with water.
Rinse container thoroughly with water and discard.

ENVIRACHEM, INC. • 14861 MEYERS RD. • DETROIT, MI 48227

CONTENTS: 1 GALLON

PRINTED IN U.S.A.

EPA REG. NO. 11662-2

KEEP FROM FREEZING

CLEAR-UNCOLORED

INSTA SOAK
BRAND

READY TO USE • NO MIXING REQUIRED

EQUIPMENT DISINFECTANT

Active ingredients:

n-Alkyl (60% C₁₄, 30% C₁₆, 5% C₁₂, 5% C₁₈)
dimethyl benzyl ammonium chlorides 0.0625%

n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl ethylbenzyl
ammonium chlorides 0.0625%

Inert Ingredients: 99.8750%

CONTAINS SODIUM NITRITE 00.0000%


CAUTION: Keep out of reach of children.
Avoid contact with eyes. In case of eye irritation, flush with water.
Rinse container thoroughly with water and discard.

ENVIRACHEM, INC. • 11861 MEYERS RD. • DETROIT, MI 48227

CONTENTS: 1 GALLON

PRINTED IN U.S.A.

EQUIPMENT DISINFECTANT is a specially compounded Disinfectant-Germicide, ready to use for the cleaning and disinfecting of Operating Room, Laboratory, Inhalation Therapy, Anesthesia, and other such equipment.

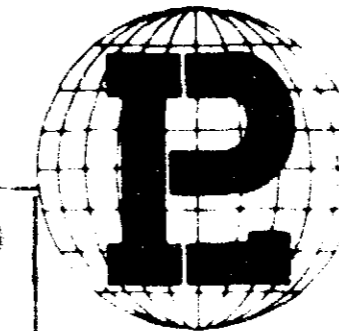

ACCEPTED

4-30-74

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND ROACHING ACT
FOR ECONOMIC POISON REGISTRY
EPA REG. NO. 11662-2

Harmless to metal-rubber-plastics in use solution.