

11724-27

ACCEPTED
9-13-67
UNDEL...
FOR...
ED... 3696-45



DECITEX

DECITEX Disinfectant is a non-specific multi-purpose germicide. It is effective against antibiotic resistant Staphylococcus strains in dilutions up to 1:180. DECITEX Disinfectant will confer a use dilution of 1:180 against Salmonella typhosa and Micrococcus pyogenes var. aureus. Staphylococcus aureus is the official A.O.A.C. Use Dilution Confirmation Test. DECITEX Disinfectant will kill Trichophyton mentagrophytes generally associated with Athlete's Foot at dilutions up to 1:180. DECITEX Disinfectant contains a balanced blend of chlorinated phenolics scientifically combined with other chemicals to produce a highly effective germicide.



8308

DECITEX

(Formerly Tenco)

ACTIVE INGREDIENTS

Potassium Soap	11.1%
4 and 6-Chloro 2 phenylphenol	5.8%
Isopropyl Alcohol	3.2%
2-Chloro-4-phenylphenol	1.8%
Ortho-benzyl para-chlorophenol	1.7%
Tetrasodium ethylenediaminetetraacetate	0.5%
Total Active Ingredients	24.1%

INERT INGREDIENTS: Including water, sodium tetraphosphate and sodium chromate: 75.9%

TOTAL 100.0%

A.O.A.C. Use Dilution 1:180 Phenol coefficient 10, A.O.A.C. Method, 20-13

CAUTION: KEEP OUT OF REACH OF CHILDREN

"SEE SIDE PANEL FOR ADDITIONAL CAUTIONS"

TEXIZE CHEMICALS, INC.

Greenville, S. C., Cleveland, Ohio, Palestine, Texas

DIRECTIONS FOR USE

Do not add other disinfectants or cleaning agents to DECITEX Disinfectant.

FOR GENERAL DISINFECTION: Surgical instruments, rubber products, thermometers, linens, bedpans, laboratories, toilets, urinals, floors. Use a 0.6% solution DECITEX Disinfectant.

FOR DISINFECTION IN THE PRESENCE OF BLOOD AND OTHER ORGANIC MATTER: Use a 1% solution DECITEX Disinfectant.

DILUTION TABLE

	oz. qt.	oz. gal.	oz. 5 gal.
0.6% solution 1:180	5.0	1.0	0.2
1% solution 1:100	10.0	2.0	0.4

CAUTION: DECITEX is irritating to the skin. Getting this material in eyes or on skin. If irritation persists get medical attention. Harmful if swallowed. If taken internally, call physician. Contamination of food. **KEEP OUT OF REACH OF CHILDREN**

1167 U.S. PAT. OFF. No. 2,995,411
Copyright © 1967 Texize Chemicals, Inc.

ACCEPTED WITH COMMENTS

11724-2A

ACCEPTED

9-13-67

UNDER THE FEDERAL REGISTER ACT

REGISTERED TRADEMARK

REGISTERED TRADEMARK

REGISTERED TRADEMARK



8308

DECITEX

(Formerly Tenco)

ACTIVE INGREDIENTS

Potassium Soap	11.1%
4 and 6-Chloro-2 phenylphenol	5.8%
Isopropyl Alcohol	3.2%
2-Chloro-4-phenylphenol	1.8%
Ortho-benzyl-para-chlorophenol	1.7%
Tetrasodium ethylenediaminetetraacetate	0.5%
Total Active Ingredients	24.1%

INERT INGREDIENTS: Including water, sodium tetraphosphate and sodium chromate 75.9%

TOTAL 100.0%

A.O.A.C. Use Dilution 1:180 Phenol coefficient 10, A.O.A.C. Method, 20°C

CAUTION: KEEP OUT OF REACH OF CHILDREN

"SEE SIDE PANEL FOR ADDITIONAL CAUTIONS"

TEXIZE CHEMICALS, INC.

Greenville, S. C., Cleveland, Ohio, Palestine, Texas

DIRECTIONS FOR USE

Do not add other disinfectants or cleaning agents to DECITEX Disinfectant

FOR GENERAL DISINFECTION: Surgical instruments, rubber products, thermometers, linens, bedpans, lavatories, toilets, urinals, floors. Use a 0.6% solution of DECITEX Disinfectant.

FOR DISINFECTION IN THE PRESENCE OF BLOOD OR OTHER ORGANIC MATTER: Use a 1% solution of DECITEX Disinfectant.

DILUTION TABLE:	oz. qt.	oz. gal.	oz. 3 gal.
0.6% solution 1:180	5 1/2	1 1/4	2 1/4
1% solution 1:100	10	1 1/2	4

CAUTION: DECITEX is irritating to the skin. Avoid getting this material in eyes or on skin. In case of contact, wash thoroughly with water. If irritation persists get medical attention. Harmful if swallowed. If taken internally, call physician. Avoid contamination of food. **KEEP OUT OF REACH OF CHILDREN**

1167 U.S. Pat. Reg. No. 1626497 Made in U.S.A.
Copyright © 1957, Texize Chemicals, Inc. 42050