

# POLY-PHEN

Polyphenolic Germicidal Detergent Concentrate

BACTERIOCIDAL, PSEUDOMONOCIDAL, STAPHYLOCIDAL,  
TUBERCULOCIDAL, FUNGICIDAL (PATHOGENIC FUNGI)

ACTIVE INGREDIENTS: o-phenylphenol, 5.0%; 4-chloro-2-cyclopentylphenol, 3.0%; p-tertiary  
amylphenol, 1.0%; tetrasodium ethylene diamine tetraacetate, 1.2%; isopropyl alcohol, 10.0%  
INERT INGREDIENTS: 79.8%

EPA Registration No. 11853-2

## DANGER:

KEEP OUT OF REACH OF CHILDREN

Corrosive to eyes, skin, and mucous membranes. Do not get in eyes or on skin or clothing. Pro-  
tect eyes and skin when handling concentrated product. Handle carefully. Avoid contact with  
food and feed.

## FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eye,  
call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of water. If inhaled, get fresh air immediately and  
available medical attention or water. Avoid taking food or physical exertion.



NET CONTENTS 30 GALLONS

**BISCAYNE CHEMICAL**

LABORATORIES, INC.

1215 N.W. 7th AVE.

PHONE (305) 377-1421

MIAMI, FLORIDA 33152

ACCEPTED

JUN 22 1973

UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE ACT  
FOR CHRONIC POISON REGISTERED  
ED UNDER NO 11853-2

11853-2

## DIRECTIONS:

Poly-Phen is a concentrated liquid detergent with a low surface tension which allows penetration of  
the product into cracks and crevices for removal of dirt.

## METHOD OF APPLICATION:

Remove dirt, grease, and debris and prior to application. Apply with sponge or brush in  
a manner to thoroughly wet all surfaces.

Poly-Phen is a professionally formulated polyphenolic germicidal detergent concentrate  
designed to improve sanitation and bacterial control and maintain an aesthetic environ-  
ment, especially in hospital, health care facilities, and institutions.

Poly-Phen is used primarily for the following: keepers, stills, and medical instruments; for  
laboratory equipment; for health care facilities, restrooms, and hospital fixtures; in  
operating rooms and kitchens.

Poly-Phen is used also for disinfecting and cleaning of athletic equipment, swimming  
pools, and tubs.

Poly-Phen is used also for cleaning of floors, walls, and ceilings in health care facilities and  
institutions.

Remove residue from surfaces by rinsing with water.

**Biscayne Chemical Laboratories, Inc.**

MIAMI, FLORIDA 33152