FOSTRACIDE EX is a germicide with an irganic soil. tolerance. Its disinfricting action has been verified in the AOAC Use Dilution Test against Staphylococcus aureus. Salmonella choleraesuis, and Pseudomona, aeruginusa in the presence of 5% blood serum as a fillurce of or game matter. This means that one step disintection and cleaning can be achieved when used as directed

Cross-contamination is of major housekeeping concern not only in hospitals, but in schools, institutions, and in dustry FOSTRACIDE EX is formulated for this problem area. It cleans disinfects, and is virucidal when used as directed. Its hard surface disinfecting action will reduce the hazard of cross contamination

FOSTRACIDE EX is virucidal against Herpes Simplex (a member of the Herbes viruses that cause chicken box severe brain infections, and infectious mononucleosis). Vaccinia (representative of the pox viruses) and Influenza As (the Hong Kong flu virus) on manimate environmental surfaces

Precautionary Statements Hazards to Humans and domestic <u>animais</u>

DANGER

They out of reach of children. Corrosive. Causes severe eye and skin damage. Do not get in eyes, on skin, or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed or absorbed through the skin. Avoid contamination of food.

STORAGE AND DISPOSAL

DO NOT CONTAMINATE WATER, FOOD, OR FEED BY STORAGE OR DISPOSAL

-OPEN DUMPING IS PROHIBITED

-DO NOT REUSE EMPTY CONTAINER

PESTICIDE DISPOSAL

PESTICIDE THAT CANNOT BE USED, OR CHEMICALLY REF POCESSED SHOULD BE DISPOSED OF IN A LAND FILL APPROVED FOR PESTICIDES OR BURIED IN A SAFE PLACE AWAY FROM WATER SUPPLIES

CONTAINER DISPOSAL

DISPOSE OF IN AN INCINERATOR OR LANDFILL AP PROVED FOR RESTIGIDE CONTAINERS BURY IN A SAFE PLACE OF RETURN TO DRUM RECONDITIONER

GENERAL STATE OR LOCAL DISPOSAL 4 14 FIGES FOR APPROVED ALTERNATIVE PRO HIS SO HIAS LIMITED OPEN BURNING

FOSTRACIDE EX

Heavy Duty Germicide with Organic Soil Tolerance

Cleaner-Disinfectant-Deodorizer-Fungloide-Virucide*-Sanitizer for Hospital and Institutional Use

Active Ingredients

Octyl Decyl Dimethyl Ammonium Chloride Dioctyl Dimethyl Ammonium Chloride Didecyl Dimethyl Ammonium Chloride Tetrasodium Ethylenediamine Tetraacetate

2.25° 2.25% 2.28¢5

4.50%

Inert Ingredients

88 72° c 100.00%

KEEP OUT OF REACH OF CHILDREN ONLY FOR SALE TO, USE AND STORAGE BY SERVICE PERSONNEL"

DANGER

Statement of Practical Treatment

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution; or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, as well as oxygen and measures to support breathing manually or mechanically may be needed. If persistent, convulsions may be controlled by the cautious intravenous injection of a short-acting barbituate drug.

SEE LEFT PANEL FOR

ADDITIONAL PRECAUTIONARY STATEMENTS

EPA Registration No. EPA Establishment No. 12179-7

Net Contents

12179-MI-1

Directions for Use

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

FOSTRACIDE EX is a proven one step cleaner disinfectant samitizer fung cide virucide in the presence of moderate amounts of organic soil when used as directed

App., FOSTRACIDE EX to walls floors and other hard unanimate) non-porous surfaces such as tables chairs countertops, sinks and beatrames with a cloth, modific mechanical scray device so as to thoroughly well surfaces. For heard, sweed areas, a preliminary cleaning is required Prepare a fresh solution daily or when use solution be sure- sistly diffy

Disinfection — for routine disinfection of hard, non-porous surfaces add 1 oz per gallon of water. Treated surfaces ast remain well for 10 minutes. At this use level, FOSTRACIDE EX is also virucidal and fungicidal. When encountering organic soil in hospitals, nursing homes schools institutional and industrial uses, add 2 oz per gatton of water

Sanitizing - To sanitize polous or non-porous, non-tood contact surfaces, add 34 oz per gallon of water Treated surfaces must remain wet for 60 seconds.

For hospitals, nursing homes, schools, institutions and industry, add 1 oz. per gallon of water. The broad spectrum activity of FOSTRACIDE EX has been evaluated in the presence of 5° serum and found to be effective against the following organisms by the AOAC Use Dilution test

Pseudomonas aeruginos... Staphylococcus aureus Salmonella choleraesuis

AOAC Fungicidal Test

FOSTRACIDE EX is an effective fungicide against Trichophyton mentagrophytes (the athlete's foot fungus) when used on surfaces in areas such as locker rooms, pressing rooms, shower and bath areas, exercise facilities,

Virucidal Performance

'At 1 oz Le granuse riel FOSTRACIDE EX MAS evaluated in the presence of 10% serum and found to be effective against the following viruses Herpes Simplex (a member of the crus family that causes infectious recovery, ecolor Vaucinia irecresentative of the poxing on an influency Avia incresented by the strain remoting and the Hora King Fluids inanimate analica harital surfaces

Sanitizing Non-Food Contact Surfaces (such as walls, floors, lables, etc.). At 1 oz garlon use level FOSTRACIDE EX is an effect vision fizer in waters of up 420 ppm hardness against Staphylopocous aureus and Kittle a pneumoniae in the presente of 5% serum on hard curs is and on hipprous environmental surfaces.

FOSTER CH'EMICALS INCOFPORATED

DETROIT MICHIGAN 48238

TEL (313) 867-5444