

**DIRECTIONS:**

1. Thermometer included inside holder. Holder contains Zephiran Chloride disinfecting solution. No other fluid need be added.
2. Before each use, shake thermometer to remove excess disinfecting solution prior to inserting into patient's mouth.
3. After use, return thermometer to disinfecting solution in holder and replace cover.
4. This is a single patient use thermometer kit.
5. Discard disinfecting solution and rinse holder with water upon dismissal of patient.

**CERTIFICATE OF EXAMINATION**

We hereby certify that the enclosed Fever Thermometer was tested and compared with standards verified by U. S. Government Bureau of Standards.

This Thermometer met or surpassed all specifications for Clinical Thermometers. This Thermometer is guaranteed to be of absolute accuracy and will be replaced if found unsatisfactory for any cause except breakage.

**ACCEPTED**  
NOV 27 1972  
UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE ACT  
FOR ECONOMIC POISON REGISTERED  
UNDER NO. 4811-1 SUBJECT  
TO ATTACHED COMMENTS.