

UNDER THE FEDERAL INSECTICIDE  
 FUNGICIDE AND RODENTICIDE ACT  
 FOR FEDERAL POISON REGISTER-  
 SUBJECT NO. 15035-1  
 COMMENTS:  
 JAN 4 1973  
 ACCEPTED

IMPORTANT! Must be diluted before using.

**DANGER:** Keep Out of Reach of Children. Corrosive. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Protect eyes and skin when handling concentrate. Harmful or fatal if swallowed. Avoid contamination of food.

**FIRST AID:** In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician.

"If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Call a physician immediately."

**TO PREPARE 1 QT. OF USE SOLUTION  
 ADD 1/4 OZ. TO 1 QT. WATER**

**DIRECTIONS FOR USE**

Cleanse and rinse instruments thoroughly. Immerse in solution for at least 15 minutes. Chemical disinfectants should not be relied upon to disinfect hinged instruments, those with deep narrow crevices, hypodermic needles and syringes nor to destroy spores and M. tuberculosis or the etiologic agent for viral hepatitis. Such instruments should be sterilized by heat. To maintain disinfection they may be immersed in a fresh solution of germicide. Protect solution from contamination with resistant organisms.

When empty rinse container thoroughly before discarding.

**PERFECTO**

**germicide**

**ACTIVE INGREDIENTS:**  
 n-Alkyl (60% C<sub>14</sub>, 30% C<sub>16</sub>, 5% C<sub>12</sub>, 5% C<sub>18</sub>)  
 dimethyl benzyl ammonium chlorides .....7%  
 n-Alkyl (68% C<sub>12</sub>, 32% C<sub>14</sub>) dimethyl  
 ethylbenzyl ammonium chlorides .....7%  
 Ethyl Alcohol .....3.5%  
**INERT INGREDIENTS** .....82.5%  
 Including Sodium Nitrite as a rust inhibitor.

**concentrated  
 dental and surgical  
 instrument disinfectant**

**DANGER:**  
**Keep Out Of The Reach Of Children!**

See Antidote Statement And Other Precautions On Back Panel.

**NET: 32 FL. OZ. (1 QT.)**  
**Makes 128 quarts of diluted solution**

E.P.A. Reg. No. 15035-1

Manufactured by  
**Perfecto Products Mfg., Inc. / Atlanta, Georgia 30313**