

# SONACIDE

## STERILIZING AND DISINFECTING SOLUTION Rapid acting and stable—Minimum Use Dilution 1:10

Use with water for sterilizing and disinfecting instruments and surfaces.

- As a preservative
- For disinfecting dental instruments
- For disinfecting surgical instruments
- For disinfecting laboratory glassware
- For disinfecting hospital floors and walls
- For disinfecting hospital linens
- For disinfecting hospital furniture
- For disinfecting hospital equipment

**DANGER. MAY BE FA  
IF SWALL**

EPA Registered No. 10136-C. See EPA Label for Precautions and First Aid.

**WAVE ENERGY SYSTEMS INC.**  
NEWTOWN Pa. U.S.A.

15736-2  
 REGISTERED

Recommended for use in Hospitals, Dental, Medical Offices and Veterinary Hospitals.

**DIRECTIONS FOR USE: Mix one part by weight or volume of this concentrate to nine parts of distilled water for the correct strength Sonacide.**

### STERILIZATION or DISINFECTION

Instruments, tools, wet cloths and needles should be thoroughly cleaned, rinsed or flushed and drained before immersing in SONACIDE solution.

In metal objects to be sterilized in SONACIDE solution, immerse in solution maintained at 60°C for **60 minutes**. Rinse and flush with sterile water. This is a complete sterilization procedure including the destruction of viruses and tubercle bacilli.

In metal objects in SONACIDE solution, maintain at 60°C for **5 minutes** to destroy vegetative organisms including *Pseudomonas* species and *Epithelium*.

For vegetative organisms, including *Pseudomonas* species and *Epithelium*, immerse in solution maintained at 60°C for **10 minutes**.

For spore-forming organisms, including *Bacillus* and *Clostridium* species, immerse in solution maintained at 60°C for **20 minutes**.

For spore-forming organisms, including *Bacillus* and *Clostridium* species, immerse in solution maintained at 60°C for **20 minutes**.

Causes eye, ear and skin irritation. May be absorbed through the skin. Do not get in eyes, or on face or clothing. Protect eyes and skin when handling. Avoid contact with mucous membranes. Use in covered containers.

For eye irritation, flush with water immediately. For skin irritation, wash with water. Report any adverse reactions to the manufacturer. For first aid, see label. For more information, contact the manufacturer.

Wave Energy Systems Inc., Newtown, Pa., U.S.A.