

GERMICIDAL CONCENTRATE

For Instrument Disinfection

ACTIVE INGREDIENTS:

Cetyl Dimethyl Ethyl Ammonium
Bromide 6.5%
Alkyl (50% C₁₂, 30% C₁₄, 17%
C₁₆, 3% C₁₈) dimethyl benzyl
ammonium chloride* 6.5%

Isopropyl Alcohol 13.0%
INERT INGREDIENTS: 74.0%

Formulation Contains Sodium Nitrite
Total ingredients 100.0%

*Benzalkonium Chloride

DANGER:

Keep out of reach of children.

Do not use, pour, spill or
store near heat or open flame.

*See back panel for antidote
statement and other precautions.*

NET CONTENTS: 32 FL. OZ. (1 QT.)

FOR PROFESSIONAL USE ONLY

Manufactured in the U.S.A. by
SULTAN CHEMISTS, INC.

Hackensack, N.J. 07601

DIRECTIONS

Add 10cc of Healthco Germicide to one quart (32 fl. oz.) or one ounce to one gallon of tap water. In known hard water areas, use distilled water. Clean instruments thoroughly — rinse free of all soap before immersion and allow them to remain in Healthco Germicide solution 15 minutes. Germicides containing a Quaternary Ammonium derivative should not be relied on to destroy spore bearing organisms of Mycobacterium tuberculosis, or the etiologic agent of viral hepatitis. Thus, instruments suspected of such contamination should be sterilized by heat. Needles, hypodermic syringes, corroded instruments or those with deep narrow crevices, as well as hinged or defective plated instruments should be sterilized by heat — to maintain disinfection they may be kept emmersed in Healthco Germicide solution. Change solution periodically.

Rinse empty container thoroughly with water and discard it.

Healthco Germicidal Concentrate when diluted as directed, is a potent germicidal solution combining high bactericidal and disinfecting action against most kinds of pathogenic bacteria. In both concentrated and diluted form, Healthco Germicide is stable. When diluted for use as directed, Healthco Germicide makes a RUST-PROOF, ODORLESS, COLORLESS highly efficient germicidal solution for the chemical disinfection of SURGICAL AND DENTAL INSTRUMENTS.

DANGER:

Keep Out of Reach of Children. Corrosive. Causes eye damage and severe skin irritation. Do not get in eyes, on skin or on clothing. Protect eyes and skin when handling concentrate. Harmful or fatal if swallowed. Avoid contamination of food.

FIRST AID: In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

E.P.A. #18184-1

18184-1
ACCEPTED
MAR 13 1973

ACCEPTED WITH COMMENTS