

Box Label

NOT REVIEWED  
RECEIVED PER  
PR NOTICE 88-6

32867-2

PM 31

10F2

# TFX Medical Urolocide®

Untinted CRYSTALS  
For making untinted solutions

ACTIVE INGREDIENT:  
Benzyl (Dodecylcarbamylmethyl)  
Dimethylammonium Chloride 100%

GERMICIDE

Keep out of reach of children!

DANGER

SEE PACKET LABEL  
FOR FIRST AID AND  
ADDITIONAL  
PRECAUTIONS

Brochure with additional  
information available on  
request.

REORDER NO.  
3771-00

STORAGE AND  
DISPOSAL:

Store in a cool, dry place,  
away from excessive  
heat. Wrap empty packet  
and put in trash.

made in U.S.A. for:  
TFX MEDICAL, ALPHARETTA, GA. 30201  
EPA Reg No. 32867-2  
EPA Est No. 34848-L-01


# TFX Medical Urolocide®

12 Packets  
Net Wt. 3.8 gm (0.13 oz) EACH

REORDER NO.  
3771-00

Face

End Flap

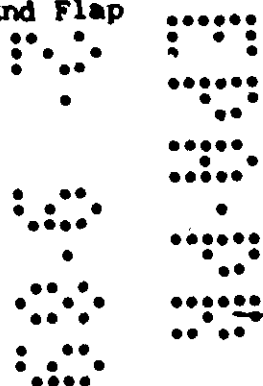

Front of Packet

Back of Packet

# TFX Medical Urolocide®

Untinted CRYSTALS  
For making untinted solutions

ACTIVE INGREDIENT:  
Benzyl (Dodecylcarbamylmethyl)  
Dimethylammonium Chloride 100%

GERMICIDE

Keep out of Reach of Children!

DANGER

### FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For the eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Call a physician immediately.

Made in U.S.A. for:  
TFX MEDICAL,  
ALPHARETTA, GA. 30201  
EPA Reg. No. 32867-2  
EPA Est. No. 34848-IL-01

SEE BACK FOR FIRST AID,  
ADDITIONAL PRECAUTIONS  
AND INSTRUCTIONS FOR  
USE.

Net Wt. 3.8 gm (0.13 oz)

### DANGER:

Keep out of reach of children. Corrosive, damage and skin irritation. Do not get in skin or on clothing. Protect eyes and skin during handling. Harmful if swallowed. Avoid contact with food.

### NOTE TO PHYSICIAN:

Probable mucosal damage may contraindicate gastric lavage. Measures against circulatory respiratory depression, and convulsion may be needed.

### Instructions for use:

To make a 1:1000 aqueous solution, dissolve 3.8 gm of Crystal UROLOCIDE in one gallon of water. The Crystals will readily dissolve in either water with brisk stirring.

For the disinfection of instruments that withstand heat sterilization, immersion for 30 minutes in 1:1000 aqueous solution UROLOCIDE is recommended.



Prior to immersion, all surfaces of the instrument should be washed and cleaned with soap and thoroughly rinsed with water. (A detergent on the instrument may neutralize the antibacterial effect of urolocide.)

### STORAGE AND DISPOSAL:

Store in a cool, dry place, away from excessive heat. Wrap empty packet and put in trash.

REOF  
377

TEAR HERE TO OPEN

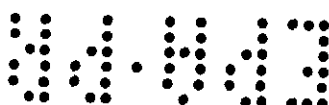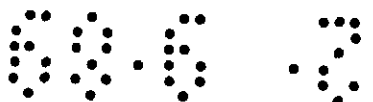

**BEST AVAILABLE COPY**