#### STRATFORD-COOKSON COMPANY



2000 SULLIVAN ROAD COLLEGE PARK GEORGIA 30337

☐ QUART (946 c.c.)

☐ GALLON (3784 c.c.)

# STRATACIDE®

**Germicidal Concentrate** 

**MUST BE DILUTED** 

#### INSTRUMENT DISINFECTANT

with rust inhibitor

This product contains Sodium Nitrite

#### **ACTIVE INGREDIENTS (16%)**

CLIVE INCKEDIENTS (10%)	
Cetyl Dimethyl Ethyl Ammonium Bromide	7%
n-Akyl (C <sub>12</sub> -50%, C <sub>14</sub> -30%, C <sub>16</sub> -17%, & C <sub>14</sub> -3%)	
Dimethyl Benzyl Ammonium Chloride	7%
Isopropyl Alcohol	2%
NERT INGREDIENTS (84%)	
Sodium Nitrite (Rust Inhibitor)	10%
Distilled Water—U.S.P.	74%
	100%

E.P.A. Registration No. 33799-1

# DANGER Keep Out Of The Reach Of Children.

See First Aid and Other Precautions on Side Panel.

FOR PROFESSIONAL USE ONLY MADE IN U.S.A.

#### DIRECTIONS FOR USE

- 1. Clean instruments thoroughly.
- 2. Rinse with clear water.
- 3. Add 16 c.c. of STRATACIDE Germicidal Concentrate to 930 c.c. (approximately one quart) of water. Use distilled water in areas known to have hard water.
- 4. Immerse instruments in the prepared solution for not less than 15 minutes prior to their use. Instruments are ready for use upon removal.

### NOTES

STRATACIDE Germicidal Concentrate has been formulated as an efficient chemical disinfectant for professional and hospital instruments and equipment. It contains a rust inhibitor and is relatively odorless and non-staining when diluted as directed above. The excellent wetting action and low surface tension of this product are advantageous for disinfection.

#### DANGER

This product is corrosive. Avoid contact with the eyes, skin, food or clothing. Do not ingest. Harmful or fatal, if swallowed. Wear goggles and rubber gloves. Rinse thoroughly this container when empty and discard same.

#### FIRST AID

In case of contact with vital tissue, flush immediately with an abundance of clear water for 15 minutes or more. Remove and launder contaminated clothing prior to reuse. If swallowed, drink large quantities of milk, egg whites, gelatin solution, or water if such are not readily available. Avoid alcohol. If swallowed or contact with the eyes is effected, call a physician immediately.

## **NOTE TO PHYSICIAN**

Probable mucosal damage may contraindicate the use of gastric lavage. Institution of measures against circulatory shock, respiratory depression, and/or convulsion may be required. This product contains 10% Sodium Nitrite.

The Stratferd-Coekson Company makes no warranty, expressed or implied, concerning the use or effectiveness of this product other than specifically indicated on this label. The buyer assumes any and all risk of use and/or handling of this material, not in strict accordance with the directions and cautions berowith given.

