

Germasonic

DISINFECTANT-ULTRASONIC CLEANER

USE DIRECTIONS:

1. Undiluted Germasonic cleans and disinfects. Undiluted Germasonic when used in conjunction with the Ultrasonic (device) removes light to medium debris such as plaster, cements, stones, and alginates from medical instruments. In the cold disinfection of surgical instruments, pre-clean the instruments to remove all adhering blood and serous exudates prior to immersion in the disinfecting solution.
2. Germasonic is used directly in stainless steel tanks or Pyrex beakers.
3. Immerse the instruments in the undiluted Germasonic solution and engage the ultrasonic (device) for 15 minutes.
4. After completion of the 15 minute disinfecting cycle, remove the instruments and rinse with water.
5. Replace Germasonic solution in the tank when the solvent becomes contaminated with debris.

GERMASONIC has been formulated to act as a medical disinfectant for professional and hospital instruments and equipment.

ACTIVE INGREDIENTS:

n-Alkyl (50% C₁₄, 40% C₁₂, 10% C₁₆)
dimethyl benzyl ammonium chloride . 5.0%
Tetrasodium salt of ethylenediamine tetra-
acetic acid 2.5%
Sodium carbonate 2.0%
INERT INGREDIENTS. 90.5%
EPA Reg. No. 35905-1

DANGER

**KEEP OUT OF REACH
OF CHILDREN. CORROSIVE.**

See side panel for first aid and additional precautions.

Container disposal: When empty, rinse container thoroughly with water and discard in a safe place.

Manufactured by -

 **POLYSCIENCES, inc**

Paul Valley Industrial Park
Warrington, Pa. 18976

NET CONTENTS

- 1 QT. (946 cc)
 1 GAL. (3784 cc)

**DANGER -
KEEP OUT OF REACH OF
CHILDREN.**

CORROSIVE. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed. Avoid contamination of food.

FIRST AID -

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, promptly drink a large quantity of milk, egg whites, or gelatin solution. If these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN -

Probable mucosal damage may contra-indicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

