

**PRECAUTIONARY
STATEMENTS**

**HAZARDS TO
HUMANS &
DOMESTIC ANIMALS**

DANGER

**KEEP OUT OF THE
REACH OF CHILDREN**

DANGER - Breathing of vapors harmful. Exposure to eyes and skin may cause severe irritation. Avoid contact with eyes, skin or clothing

IN CASE OF CONTACT - Remove all clothing and shoes - flush skin or eyes with plenty of water for at least 15 minutes - **GET MEDICAL ATTENTION IMMEDIATELY.**

**PHYSICAL OR
CHEMICAL
HAZARDS**

Contents Under Pressure:
Do not use or store near heat
or flame.

**Environmental
Hazards**

**LIQUIFIED GAS
DANGEROUSLY
REACTIVE**

**STORAGE &
DISPOSAL**

Store in cool, well ventilated
area. Avoid exposure to heat
or direct sunlight. When empty
return to manufacturer only.

ETHYLENE OXIDE

Active Ingredient
ETHYLENE OXIDE 100%

This product contains 7.25 lbs of Ethylene Oxide per gallon
(at 20 C)

**KEEP OUT OF REACH
OF CHILDREN**

DANGER

STATEMENT OF PRACTICAL TREATMENT

IF INHALED - Remove to fresh air, if breathing
has stopped administer oxygen.

IF ON SKIN - Remove contaminated clothing,
shower affected areas with water for 15 minutes.

IF IN EYES - Irrigate with copious amounts of water
for 15 minutes.

ACCEPTED

MAY 01 1987

Under the Federal Food, Drug, and Cosmetic Act, this product is recommended for use only if registered under FDA Reg. No. 36736-2

Distributed by:
ARC CHEMICAL Div. of Balchem CORPORATION
P.O. Box 180, State Hill, N.Y. 10973

EPA Registration No. 36736-2
EPA Est. 36736-NY-01

Net Contents 400 lbs

**DIRECTIONS
FOR USE**

**GENERAL
CLASSIFICATION**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

The product is limited to use by medical professionals or appropriately trained technical personnel medical and industrial use areas.

**FOR THE FOLLOWING
INTENDED USES**

1. As a sterilizing gas for industrial and/or medical uses in 100% Ethylene Oxide Commercial Gas Sterilizers Only. Follow the sterilization procedures specified in the gas sterilizers' manufacturers' manual and our booklet entitled, "Principles & Practice of Ethylene Oxide Sterilization."

2. Repacking of sterilizing gas. Refer to our technical brochure, Drummed Ethylene Oxide for chemical and physical properties.

3. Formulation of sterilizing gas mixtures. Refer to our technical brochure Drummed Ethylene Oxide for chemical and physical properties.

Use within 60 days of fill date.

PM-31, 36736-3, 1/1