

36736-2

01-29-2007

1/3

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY

JAN 29 2007

Ms. Joanne Cashin  
ARC Specialty Products  
Balchem Corporation  
P.O. Box 600  
New Hampton, NY 10958

Subject: Ethylene Oxide  
EPA Registration No.: 36736-2  
Submission Date: October 26, 2006  
EPA Receipt Date: October 31, 2006

Dear Ms. Cashin,

The following amendment submitted in connection with registration under FIFRA, as amended, is acceptable.


- Deletion of Basil
- Compliance with Tolerance Reassessment and Risk Management Decision for Ethylene Oxide

**General Comments**

A stamped copy of the accepted labeling is enclosed. Submit one (1) copy of your final printed labeling before distributing or selling the product bearing the revised labeling.

Should you have any questions concerning this letter, please contact Jacqueline McFarlane at (703) 308-6416.

Sincerely,

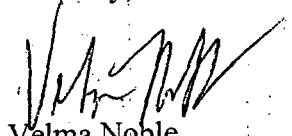

Velma Noble  
Product Manager (31)  
Regulatory Management Branch I  
Antimicrobials Division (7510P)

**CONCURRENCES**

SYMBOL							
SURNAME							
DATE							

## PRECAUTIONARY STATEMENTS

### HAZARDS TO HUMANS AND DOMESTIC ANIMALS

**DANGER!** CAUSES EYE AND SKIN BURNS. HARMFUL IF INHALED. MAY CAUSE NERVOUS SYSTEM DAMAGE.

**DANGER!** CANCER HAZARD AND REPRODUCTIVE HAZARD.

**EFFECTS OF OVEREXPOSURE:** May be fatal if inhaled in high concentrations. May cause irritation of the respiratory tract, chest tightness, headache, nausea, vomiting, diarrhea, light-headed feeling, dizziness, weakness, drowsiness, cyanosis, loss of coordination, convulsions, coma, delayed lung injury (fluid in the lungs), immediate or delayed skin irritation or blisters, allergic skin reaction.

**OTHER POSSIBLE DELAYED HEALTH EFFECTS:** May cause nervous system injury, cataracts, adverse reproductive effects, chromosomal and mutagenic changes, and cancer.

**PEL:** 1 PPM TWA (as per the Ethylene Oxide Standard 29 CFR 1910.1047).

**EL:** 5 PPM - excursion limit 15 minutes.

**ODOR:** Ether-like at high concentrations. Exposure to toxic levels may occur without warning to or detection by the user.

**PRECAUTIONS:** Do not breathe vapor. Do not swallow. Do not get in eyes, on skin, or on clothing. Store and use with adequate ventilation in accordance with the Ethylene Oxide Standard (29 CFR 1910.1047).

### PHYSICAL OR CHEMICAL HAZARDS: DANGER! - HIGHLY FLAMMABLE LIQUID AND GAS UNDER PRESSURE.

Contents under pressure. Do not use near flame, sparks, hot surfaces, or allow sources of ignition near the sterilization/fumigation area. Ethylene Oxide is extremely flammable and reactive. Ground all equipment (including this container) to prevent sparks.

**LEAK:** In case of leak evacuate area and keep personnel upwind. Shut off all sources of ignition. Use self-contained breathing apparatus and protective clothing, and shut off leak if without risk.

**FIRE:** In case of fire move container away from fire if without risk. Use water spray or fog nozzle to keep container cool.

### ENVIRONMENTAL HAZARDS:

Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans or other waters unless in accordance with the requirements of a National Pollutant Discharge Effluent System (NPDES) permit and the permitting authority has been notified in writing prior to discharge. Do not discharge effluent containing this product to sewer systems without previously notifying the local sewage treatment plant authority. For guidance contact your State Water Board or Regional Office of the EPA.

It is imperative that users of this material be familiar with ARC Specialty Products' Material Safety Data Sheet for 100% Ethylene Oxide, the label and valve tag attached to this cylinder.

EPA Registration No. 36736-2

EPA Establishment No.  36736-NY-01  
 36736-SC-01  
 36736-MO-01

# ETHYLENE OXIDE

STERILANT/FUMIGANT  
ACTIVE INGREDIENT: ETHYLENE OXIDE

(CAS NO. 75-21-8)

Keep Out of Reach of Children

## DANGER PELIGRO

PRECAUTION AL USARIO: Si usted no lee ingles, no use este producto hasta que la etiqueta le haya sido explicada ampliamente.

Users must follow requirements of the OSHA Occupational Exposure Standard for Ethylene Oxide (29 CFR 1910.1047).

### FIRST AID

IN ALL CASES OF OVEREXPOSURE GET MEDICAL ATTENTION IMMEDIATELY. TAKE PERSON TO A DOCTOR OR EMERGENCY TREATMENT FACILITY AT ONCE.

**IF INHALED:** Move person to fresh air. Keep warm. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably by mouth-to-mouth, if possible. If breathing is difficult, give oxygen. Call a poison control center or doctor for further treatment advice, even if there are no symptoms. Keep under medical observation - symptoms may be delayed.

**IF ON SKIN OR CLOTHING:** Take off contaminated clothing and shoes. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice. Aerate, wash or clean contaminated clothing and discard leather goods.

**IF SWALLOWED:** Call poison control center or doctor immediately for treatment advice. Have person sip two glasses of water if able to swallow. Do not induce vomiting. Do not give anything by mouth to an unconscious person.

**IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

**HOTLINE NUMBER:** Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-424-9300 for emergency medical treatment information.

**NOTE TO PHYSICIAN:** Skin exposure to Ethylene Oxide will commonly result in skin irritation with extensive blister formation. At high concentrations, severe conjunctivitis can occur. Irritation of the respiratory tract may occur, but without acute lung edema. Symptoms of systemic intoxication are headache, nausea, vomiting, incoordination, and cardiac irregularities. Treatment is symptomatic.

Refer to ARC Specialty Products' Ethylene Oxide MSDS. If unable to locate MSDS for this product, please call ARC Specialty Products at the telephone number below and request that one be sent immediately.

### DIRECT

It is a v  
inconsis  
follow th  
1910.10  
the requ  
1910.10  
(comme  
with 100  
persons  
Oxide S  
health c  
(comme  
clearanc

1. AS A  
A. For  
objects,  
cleaning  
closed s  
humidity  
be used  
Operato

This pro  
items, p  
201.1(d)  
ground  
180.151

### This pro

After Au  
on spic  
steriliza  
to pre-c  
aeration  
alternati  
contact  
Pesticid  
authoriz

Place s  
mixture  
chambe  
concent  
with a d  
gas from  
steam v  
(27" Hg  
chambe

# ETHYLENE OXIDE

STERILANT/FUMIGANT  
ACTIVE INGREDIENT: ETHYLENE OXIDE 100%

(CAS NO: 75-21-8)

**ACCEPTED**  
JAN 29 2007  
Under the Federal Insecticide, Fungicide, and Plant Pesticide Act, registered under EPA Reg. No. 36736-2

Net Contents: \_\_\_\_\_ lbs.

B. Sterilization/fumigation with Ethylene Oxide must be performed only in vacuum or gas tight chambers designed for use with Ethylene Oxide.

C. Ethylene Oxide cycle parameters depend on several sterilizing/fumigating variable factors: pre-conditioning (if any); exposure time; chamber air pressure; gas concentration; types and quantities of items to be sterilized/fumigated; packaging; load configuration in the chamber; microbial challenge method; desired degree of disinfection; and the desired performance of the sterilized/fumigated product and package.

D. The sterilization/fumigation cycle parameters should be those prescribed by the equipment manufacturer. If other cycle parameters are used, the safety and efficacy of the alternate cycle parameters must be validated and are the responsibility of the user.

**NEVER USE PARAMETERS WHICH ALLOW FLAMMABLE MIXTURES OF ETHYLENE OXIDE AND AIR TO ENTER THE CHAMBER.**

children

### DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. It is the employer's responsibility to follow the requirements of the Ethylene Oxide Standard (29 CFR 1910.1047). This product may be used only in facilities that meet the requirements of the Ethylene Oxide Standard (29 CFR 1910.1047). This product may be used only in non-portable (commercial) vacuum or gas-tight chambers designed for use with 100% ethylene oxide. This product may be used only by persons who have been trained in accordance with the Ethylene Oxide Standard (29 CFR 1910.1047). When used to sterilize health care items, this product must be used in non-portable (commercial) ethylene oxide gas sterilizers that have FDA clearance.

### 1. AS A STERILANT AND FUMIGANT GAS:

A. For complete use directions (including type of surfaces, objects, or items/products recommended for treatment, pre-cleaning instructions, concentration of gas per unit volume of closed space to be treated, exposure time/temperature, relative humidity, ventilation/aeration time, and method of monitoring to be used) refer to the ethylene oxide gas sterilizer manufacturers' Operators Manuals.

This product may be used only to sterilize medical or laboratory items, pharmaceuticals, and aseptic packaging, (see 21 CFR 201.1(d)(5)), or to reduce microbial load on cosmetics, whole and ground spices or other seasoning materials (see 40 CFR 180.151) and artifacts, archival material or library objects.

**This product may not be used on or in any form of basil.**

After August 1, 2008, this product may only be applied to or on spices, dried vegetables or seasonings utilizing an ETO sterilization method that uses a single sterilization chamber to pre-condition and aerate with an alternating vacuum and aeration purging procedure. If you wish to employ an alternative method to that described below, you must contact the Environmental Protection Agency Office of Pesticide Programs for instruction on how to receive authorization.

Place spices in the treatment chamber. Assure that the mixture of ethylene oxide and air is compatible with the chamber design, then, introduce into the chamber a concentration of Ethylene Oxide not to exceed 500 mg/L, with a dwell time not to exceed 6 hours. Then evacuate the gas from the chamber using a sequence of not less than 21 steam washes (injections and evacuations) between 1.5 PSIA (27" Hg) and 5.0 PSIA (20" Hg) while maintaining a minimum chamber temperature of 115° F.

### 2. INSTALLATION AND OPERATION OF THE CONTAINER

Follow the directions on the valve tag attached to this container.

### STORAGE AND DISPOSAL

Do not contaminate food, feed, or water by storage and disposal.

**PESTICIDE STORAGE:** Use in accordance with tag attached to valve. Store in cool, well-ventilated area. Avoid exposure to heat or direct sunlight as may cause polymerization.

**PESTICIDE DISPOSAL:** Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray, or mixture of rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

**CONTAINER DISPOSAL:** When empty, return to supplier only. Before returning container to supplier:

1. Pressurize container with nitrogen to 50 psig total pressure at 70° F.
2. Replace valve plug tightly in valve outlet.
3. Check container valve and plug for leaks prior to shipment.

**USE THIS PRODUCT WITHIN 120 DAYS OF ITS PURCHASE.**

**DO NOT REMOVE THIS LABEL.**

**In case of accident or emergency, call CHEMTREC at 800-424-9300.**

Distributed By:  
**ARC Specialty Products**  
Balchem Corporation  
P.O. Box 600 · New Hampton, NY 10958  
Tel: 845-326-5611  
Fax: 845-326-5706  
[www.arcspecialtyproducts.com](http://www.arcspecialtyproducts.com)

**GRO**  
Listed no. lee  
hasta que la  
mpliamente.

**SET**

r. Keep warm. If person is  
ance, then give artificial  
th, if possible. If breathing  
ontrol center or doctor for  
are no symptoms. Keep  
may be delayed.  
contaminated clothing and  
lenty of water for 15-20  
or doctor for treatment  
iated clothing and discard

ontrol center or doctor  
person sip two glasses of  
ce vomiting. Do not give  
erson.  
se slowly and gently with  
ontrol center or doctor for

st container or label with  
er or doctor, or going for  
-424-9300 for emergency

re to Ethylene Oxide will  
xtensive blister formation.  
tivities can occur. Irritation  
without acute lung edema.  
are headache, nausea,  
regularities. Treatment is

thylene Oxide MSDS. If  
please call ARC Specialty  
w and request that one be