

**PRECAUTIONARY
STATEMENTS**

**HAZARDS TO
HUMANS &
DOMESTIC ANIMALS**

DANGER

**KEEP OUT OF THE
REACH OF CHILDREN**

DANGER - Breathing of vapors harmful. Exposure to eyes and skin may cause severe irritation. Avoid contact with eyes, skin or clothing.

IN CASE OF CONTACT - Remove all clothing and shoes - flush skin or eyes with plenty of water for at least 15 minutes - **GET MEDICAL ATTENTION IMMEDIATELY.**

**PHYSICAL OR
CHEMICAL
HAZARDS**

Contents Under Pressure:
Do not use or store near heat
or flame.

**Environmental
Hazards**

**LIQUIFIED GAS
DANGEROUSLY
REACTIVE**

**STORAGE &
DISPOSAL**

Store in cool, well ventilated
area. Avoid exposure to heat
or direct sunlight. When empty
return to manufacturer only.

ETHYLENE OXIDE

Active Ingredient
ETHYLENE OXIDE 100%

This product contains 7.25 lbs of Ethylene Oxide per gallon
(at 20 C)

**KEEP OUT OF REACH
OF CHILDREN**

DANGER

STATEMENT OF PRACTICAL TREATMENT

IF INHALED - Remove to fresh air, if breathing
has stopped administer oxygen.

IF ON SKIN - Remove contaminated clothing,
shower affected areas with water for 15 minutes.

IF IN EYES - Irrigate with copious amounts of water
for 15 minutes.

ACCEPTED

OCT 11 1984

Under the Fed. Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 36736-6

Distributed by
ARC CHEMICAL Div of Balcem CORPORATION
P O Box 180, State Hill, NY 10973

EPA Registration No 36736-2
EPA Est 36736-NY-01

RECEIVED

OCT 05 1984

Net Content: 400 lbs

Manufacturer: Union Carbide (TS-262)

**DIRECTIONS
FOR USE**

**GENERAL
CLASSIFICATION**

It is a violation of Federal
Law to use this product
in a manner inconsistent
with its labeling.

The product is limited to
use by medical profes-
sionals or appropriately
trained technical person-
nel medical and industrial
use areas.

**FOR THE FOLLOWING
INTENDED USES**

1. As a sterilizing gas for
industrial and/or medical
uses in 100% Ethylene
Oxide Commercial Gas
Sterilizers Only. Follow
the sterilization pro-
cedures specified in the
gas sterilizers' man-
ufacturers manual and our
booklet entitled, "Princi-
ples & Practice of Ethylene
Oxide Sterilization."

2. Repacking of sterilizing
gas. Refer to our techni-
cal brochure, Drummed
Ethylene Oxide for chemi-
cal and physical proper-
ties.

3. Formulation of sterilizing
gas mixtures. Refer to our
technical brochure Drum-
med Ethylene Oxide for
chemical and physical
properties.

Use within 60 days of fill
date.