

38526-1 P-31  
US ENVIRONMENTAL PROTECTION AGENCY  
OFFICE OF PESTICIDES PROGRAMS  
REGISTRATION DIVISION (TS-767)  
WASHINGTON, DC 20460

EPA REGISTRATION NO.  
38526-1

DATE OF ISSUANCE  
AUG 24 1988

TERM OF ISSUANCE

NOTICE OF PESTICIDE: ☐ REGISTRATION  
☐ REREISTRATION  
(Under the Federal Insecticide, Fungicide,  
and Rodenticide Act, as amended)

NAME OF PESTICIDE PRODUCT  
Cavicide Disinfectant-Cleaner Fungicide  
Tuberculocide-Deodorant

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

Microecology Products Corporation  
116 Churchcroft Lane  
Williamsville, NY 14221

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.

Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

The product is conditionally registered in accordance with FIFRA sec. 30.10(a) provided that you:

1. Submit to EPA all data required for registration/reregistration of your product under FIFRA sec. 30.15) when the Agency requires all registrants of similar to submit such data.

2. Have the labeling changes listed below before you release the product for shipment:

a. Add the phrase "EPA Registration No. 38526-1."

b. Under "application" change "Granule mix to mix" phrase to read "mix."

3. Submit to EPA copies of your first printed labeling which you release to the public for comment. Refer to the A-7010 Sheet for details and the A-7010-1000 Labeling.

BEST AVAILABLE COPY

☐ ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL

DATE

2/3

7

-2-

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

Sincerely yours,

John H. Lee  
Product Manager 31  
Disinfectant Branch  
Registration Division (TS-767C)

1987 Label with 1988 Revisions

**CAVICIDE®** DISINFECTANT—CLEANER Ready to use  
BACTERICIDAL—FUNGICIDAL  
TUBERCULOCIDAL

KILLS *Pseudomonas aeruginosa*  
*Salmonella choleraesuis*  
*Staphylococcus aureus*  
*Trichophyton mentagrophytes*  
*Mycobacterium tuberculosis* Var *Bovis* (BCG)  
Destroys mold and mildew ~~theophylline~~

**APPLICATION**

A broad spectrum germicidal cleansing agent for hard surfaces including glass, ceramic tile, stainless steel, painted surfaces, and floors. Use in all areas of the Hospital, Nursing Homes, Medical and Dental Offices, Veterinary Clinics, Hotels, Motels and Schools. Leaves an invisible film and a sparkling clear finish. Especially effective in critical isolation sections.

**DIRECTIONS FOR USE**

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

**MEDICAL AND DENTAL OFFICES**

**CLEANING:** Submerge instruments to be cleaned and disinfected into a suitable cleaning tray of Cavicide®. Use a firm bristle brush to thoroughly loosen and remove all visible debris. Rinse and transfer to drying area.

**DISINFECTING:** Using a suitable disinfecting tray, add sufficient ready to use Cavicide® to cover such dental tools as: Saliva ejector, extraction instruments, "Cavity Prep" tools and cutting instruments, and medical instruments such as thermometers, cutting instruments, retractors, hemostats, Kelly clamps, glass syringes and similar Cavicide® cleaned instruments to be disinfected. Let soak 10 minutes or more. When ready to use, remove, rinse under tap and use immediately. Change solution daily or more often as soil becomes apparent.

or dilution

**USE IN SHARP DISPOSAL CONTAINER**

In order to reduce the level of contamination and aid in the sanitary disposal of syringes, needles, blades and other items no longer useable, fill sharps container to fill mark with Cavicide® ready to use solution. Dispose of above items by dropping into sharps container as required.

ACCEPTED  
with COMMENT  
in EPA Letter D

AUG 22 1988

38526-1

**CAVICIDE®**  
DISINFECTANT—CLEANER  
TUBERCULOCIDAL

**ACTIVE INGREDIENTS:**  
Diisobutylphenoxyethoxy  
chloride .....  
Isopropanol .....  
**INERT INGREDIENTS** .....

**CONTAINS**  
**KEEP**

See  
MICROECOLOGY PR  
1000 Golden Ring Rd  
Baltimore, Maryland  
(301) 584-1000

116 Church  
Williamson  
(716) 633-