

1987 Label with 1988 Revisions

CAVICIDE[®] DISINFECTANT—CLEANER Ready to use
BACTERICIDAL—FUNGICIDAL
TUBERCULOCIDAL

KILLS: *Pseudomonas aeruginosa*
Staphylococcus aureus
Streptococcus aureus
Escherichia coli
Mycobacterium tuberculosis var. *H3N2*
Candida albicans and *Candida tropicalis*

APPLICATION

A broad spectrum germicidal cleansing agent for hard surfaces including glass, ceramic tile, stainless steel, painted surfaces, and floors. Use in all areas of the Hospital, Nursing Homes, Medical and Dental Offices, Veterinary Clinics, Hotels, Motels and Schools. Leaves an invisible film and a sparkling clear finish. Especially effective in critical isolation sections.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.


MEDICAL AND DENTAL OFFICES

CLEANING: Submerge instruments to be cleaned and disinfected into a suitable cleaning tray of Cavicide[®]. Use a firm bristle brush to thoroughly loosen and remove all visible debris. Rinse and transfer to drying area.

DISINFECTING: Using a suitable disinfecting tray, add sufficient ready to use Cavicide[®] to cover such dental tools as: Saliva ejector, extraction instruments, Davy Prep tools, and cutting instruments, and medical instruments such as thermometers, cutting instruments, retractors, hemostats, Kelly clamps, glass syringes, and similar. Cavicide[®] cleaned instruments to be disinfected. Let soak 10 minutes or more. When ready to use, remove, rinse under tap and use immediately. Change solution daily or more often as soil becomes apparent.

USE IN SHARPS DISPOSAL CONTAINER

In order to reduce the level of contamination and aid in the sanitary disposal of syringes, needles, blades and other items no longer useable, fill sharps container to fill mark with Cavicide[®] ready to use solution. Dispose of above items by dropping into sharps container as required.

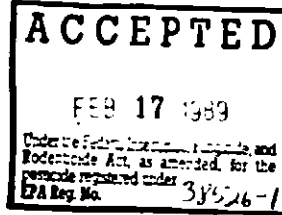

CAVICIDE[®]

DISINFECTANT—CLEANER—~~STERILIZER~~—FUNGICIDE
TUBERCULOCIDE—DEODORANT

Ready to use

ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride 0.25%
Isopropanol 15.30%
INERT INGREDIENTS 84.45%

CONTAINS BIODEGRADABLE DETERGENT

KEEP OUT OF REACH OF CHILDREN

CAUTION

See inside panel for additional precautionary statements

NET CONTENTS:

1 GALLON

EPA REG. NO. 38526-1

MICROECOLOGY PRODUCTS CORP.
1000 Greenburg Road
Summers, Maryland 21156 U.S.A.
(301) 281-0000

116 Churchcroft Lane
Williamsville, NY 14221
(716) 633-8081

GENERAL INSTITUTIONAL MAINTENANCE

To disinfect walls, floors, and other hard surfaces, mop or wipe clean with ready to use Cavicide[®], wetting the entire surface thoroughly and allow to air dry. Prepare fresh solution for each use.

Not at least 10 minutes

MOLD AND MILDEW

To control mold and mildew on clean hard surfaces, mop and wipe to wet surface thoroughly with Cavicide[®]. Allow to air dry after application. Repeat application in seven days or as necessary to maintain control.

HEAVY SOILS

Surfaces contaminated with bacterial growth supporting media, such as food stuffs, blood, exudates, vomitus, or fecal matter, should be scrubbed thoroughly with ready to use Cavicide[®] until free of all visible deposits. Cleaned surfaces should be immersed in a fresh solution of Cavicide[®] for 10 minutes. Allow to air dry.

FOOD CONTACT SURFACES

CLEANING: Contaminated food contact surfaces should be free from bacteria growth supporting media then thoroughly scrubbed and rinsed with Cavicide[®].

DISINFECTING: Pre-cleaned surfaces or food contact items such as: eating utensils, drinking glasses, beer mugs, plates, pots, pans, cutting boards, and bread boxes must be immersed or thoroughly wetted with Cavicide[®] for 10 minutes and allowed to air dry. Rinse with hot water of drinking quality before reuse.

DEODORIZING

When used as directed, ready to use Cavicide[®] removes odorous contaminants, reduces odor-causing bacteria, and provides sanitizing action and a pleasant mint aroma. See left panel for use directions.

PRECAUTIONARY STATEMENTS
Hazard to Humans and Domestic Animals
CAUTION

KEEP OUT OF REACH OF CHILDREN. MAY CAUSE EYE AND SKIN IRRITATION. AVOID CONTACT WITH EYES AND SKIN. IMMEDIATELY FLUSH EYES WITH PLenty OF WATER FOR AT LEAST 15 MINUTES.

Statement of Practical Treatment

STORAGE AND DISPOSAL

Pesticide ~~Disposal:~~ ~~When empty, rinse container thoroughly with water and dispose of unusable contents in an approved landfill.~~

~~Container Disposal:~~ ~~When empty, rinse container thoroughly with water and dispose of in regular trash.~~

*Store in a cool place. Do not reuse empty container.
Wrap container and put in trash.*

*PH 3
3/2/01
JH*