

PM31

38526-1

10/4

SQUARE BOTTLE

cavicideTM

HOSPITAL DISINFECTANT/CLEANER

Immersion Solution - Surface Cleaner/Disinfectant - Sonic Solution

BACTERICIDAL • VIRUCIDAL • FUNGICIDAL • TUBERCULOCIDAL • PSEUDOMONICIDAL

Ready to use

ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl

ammonium chloride

Isopropanol

INERT INGREDIENTS

TOTAL

0.25%

15.30%

84.45%

100.00%

CONTAINS BIODEGRADABLE DETERGENT

KEEP OUT OF REACH OF CHILDREN

CAUTION

PRECAUTIONARY STATEMENTS

Harmful to Humans and Domestic Animals if Swallowed.

CAUTION

KEEP OUT OF REACH OF CHILDREN. MAY CAUSE EYE IRRITATION. AVOID
DIRECT CONTACT WITH EYES. IN CASE OF EYE CONTACT, IMMEDIATELY FLUSH
EYES WITH PLENTY OF WATER FOR AT LEAST 15 MINUTES. HARMFUL IF
SWALLOWED. AVOID CONTAMINATION WITH FOOD.



PRODUCTS, INC
887 E. Wilmette Road
Palatine, Illinois 60067

NET CONTENTS:

1 GALLON

front panel

ACCEPTED

DEC 31 1992

Under the Federal Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 38526-1

ACCEPTED
DEC 31 1992

Under the Federal Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 38526-1

- Effective against:
- Staphylococcus aureus
 - Pseudomonas aeruginosa
 - Salmonella choleraesuis
 - Mycobacterium tuberculosis Var Bovis (BCG)
In ten minutes at 20°C
 - Trichophyton mentagrophytes
 - herpes simplex 1 and 2 viruses,
 - poliovirus type 1 and 2
 - coxsackievirus
 - HIV-1 (AIDS Virus)
 - on inanimate surfaces

DIRECTIONS FOR USE: It is a violation of U.S. Federal law to use this product in a manner inconsistent with its labeling.

DESCRIPTION:

CAVICIDE is a multipurpose, broad spectrum, highly effective cleaner and disinfectant for use on the surfaces of inanimate objects. It is especially useful in hospital operating rooms, emergency departments, isolation areas, neonatal units, and other high risk areas where the control of cross contamination is important.

Safe for use on delicate medical/dental/surgical equipment and instrumentation, can be used to clean and disinfect such items as: infant incubators and bassinets, oxygen pods, anesthesia and respiratory therapy equipment, OR tables and lights, laboratory equipment and surfaces, and to thoroughly clean and disinfect surgical and dental instruments, including plastics (such as: polycarbonate, polyvinylchloride, polypropylene and polystyrene).

APPLICATIONS:


For cleaning large equipment and other surfaces:

- Apply CAVICIDE directly to surface using a spray type bottle. Allow to remain wet for about 30 seconds. Wipe away solution and debris using a clean paper or cloth towel. Discard towel.
- Re-spray surface thoroughly wetting precleaned area. Allow to remain wet for 2 minutes. Wipe CAVICIDE from surface using a fresh, clean paper or cloth towel. (FOR MAXIMUM BACTERICIDAL ACTIVITY: Allow surface to remain wet for 10 minutes before wiping as above.)
- Cavicide completely inactivates the HIV-1 (AIDS virus) on hard, non-porous surfaces in the presence of a moderate amount of organic soil (5% blood serum) at a contact time of 2 minutes at room temperature (20-25°C)

• RIGHT PANEL

BEST AVAILABLE COPY

3044

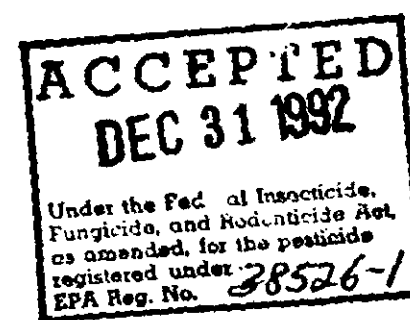

For use as an ultrasonic cleaning solution:

- Thoroughly rinse instruments under cool running water to remove gross debris.
- Using 1 part CAVICIDE to 3 parts water, submerge instruments into solution and activate ultrasonic unit for 10 minutes.
- Remove and rinse.
- Change solution as needed.

For use as an ultrasonic cleaner/disinfectant:

- Add undiluted CAVICIDE to ultrasonic unit.
- Immerse pre-cleaned instruments into solution and activate ultrasonic unit for 10 minutes.
- Remove and rinse prior to use.

For use as a hospital disinfectant for instruments, etc.:

- Instruments must be thoroughly cleaned to remove excess organic debris, rinsed and then rough dried. (Clean and rinse the lumens of hollow instruments before filling with solution or before immersion.)
- Immerse instruments into undiluted CAVICIDE solution and allow to remain submerged for 10 minutes at room temperature (20°C) for tuberculocidal kill.
- Remove and rinse prior to use.
- Change solution daily or more often as needed if the solution becomes diluted or soiled.

For use as manual cleaner for instrumentation:

- First rinse dirty, contaminated instrumentation under cool running tap water to remove gross debris.
- Using 1 part CAVICIDE to 3 parts water, fill an appropriate size container with a sufficient amount of solution as to allow for complete submersion of instruments.
- Place instruments into solution and allow to soak for about 5 minutes before scrubbing surfaces using a stiff bristle brush. Allow instruments to soak for an additional 10 minutes before removing.
- Rinse thoroughly.
- Change solution as needed.

MOLD AND MILDEW:

To control mold and mildew on clean hard surfaces, apply and wipe to wet surface thoroughly with CAVICIDE. Allow to air dry after application. Repeat application in seven days or as necessary to maintain control.

STORAGE: Store in a cool place.

PESTICIDE DISPOSAL: Dilute with water. Dispose in sanitary sewer.

CONTAINER DISPOSAL: Do not reuse empty container. Wrap container and put in trash.

LEFT PANEL

BEST AVAILABLE COPY

4044

ACCEPTED
DEC 31 1992
Under the Federal Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 38526-1

* KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACE OBJECTS
PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care
settings or other settings in which there is an expected
likelihood of soiling of inanimate surfaces/objects with
blood or body fluids, and in which the surfaces/objects
likely to be soiled with blood or body fluids can be
associated with the potential for transmission of human
immunodeficiency virus Type 1 (HIV-1) (associated with
AIDS).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION
AGAINST HIV-1 (HUMAN IMMUNODEFICIENCY VIRUS OR AIDS VIRUS)
OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUID.**

Personal Protection: Wear appropriate barrier protection
such as latex gloves, gowns, masks or eye coverings.

Cleaning Procedure: Blood and other body fluids must be
thoroughly cleaned from surfaces and objects before
application of CAVICIDE.

Contact Time: While the HIV-1 virus is inactivated in 2
minutes use the recommended contact time for disinfection of
other organisms on this label.

Infectious Materials Disposal: Cleaning materials that may
contain blood and other body fluids should be autoclaved and
disposed of in accordance with local regulations for
infectious disposal.

For product information, please contact our Customer Service
department at 1-(708)-358-6303.

Reorder No. CO4-128

Manufactured for
MICRO-ASEPTIC PRODUCTS, INC.
Palo Alto, Niles 94067 U.S.A.

EPA REG. NO. 38526-1

EPA EST. NO. 5484 IL-1

BEST AVAILABLE COPY

BACK PANEL