

OMEGA™

DETERGENT-DISINFECTANT-VIRUCIDE* DEODORANT FOR USE IN HOSPITALS


Active Ingredients

n-alkyl (60% C ₁₂ , 30% C ₁₄ , 5% C ₁₂ , 5% C ₁₄) dimethyl benzyl ammonium chlorides	6.00%
n-alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides	6.00%
Inert Ingredients	88.00%

KEEP OUT OF REACH OF CHILDREN
DANGER

See Back Panel for Additional Precautionary Statements and First Aid Instructions.

PELIGRO: SI NO PUEDE LEER EN INGLES, PREGUNTE A SU SUPERVISOR SOBRE LAS INSTRUCCIONES DE USO APROPIADAS ANTES DE TRABAJAR CON ESTE PRODUCTO.

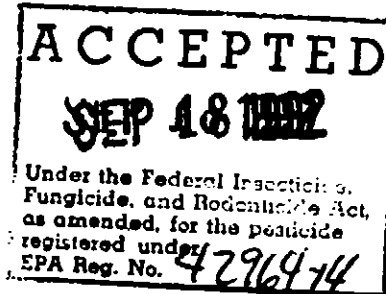

Net Contents: 5 Gallons
(18.9 Liters)

 **Airkem**® A Service of
ECOLAB
PROFESSIONAL PRODUCTS

EPA Reg. No. 42964-14
EPA Est. 1677-IL-2 (LJ) 1677-MN-1 (P)
1677-NJ-1 (W) 1677-IL-3 (F)
1677-TX-1 (D) 1677-CA-1 (S)
1677-GA-1 (M) 1677-PR-1 (B)

Superscript refers to first letter of date code.

OMEGA is a balanced formulation combining Airkem's deodorants, synthetic detergents, chelating agents and synergizing builders... for effective one-step cleaning, disinfecting, and deodorizing in the presence of protein (5% blood serum) and soap film residue.

OMEGA is intended as a cleaner where disinfection and deodorization are also indicated. It is for washing floors, walls and other hard nonporous inanimate environmental surfaces such as tables, chairs, countertops, bathroom fixtures, sinks, bed frames, shelves, racks, carts, refrigerators, coolers, glazed tile, glazed porcelain, plastic (such as polyolefins, polyesters, ABS, PVC), metal, glass, and does not damage hard, nonporous washable painted, varnished or lacquered surfaces. Because OMEGA rinses, it is particularly suitable for wood, linoleum, rubber, rubber sealed cork, vinyl, asphalt tile floors, marble and terrazzo. OMEGA is recommended for use on conductive flooring. Without spotting.

Areas of Use: Hospitals, nursing homes, other health-care facilities, schools, colleges, veterinary clinics, animal life science laboratories, USDA inspected food processing facilities and other food processing plants, dietary areas, office buildings, recreational facilities, retail and wholesale establishments.

When used as directed, this product will deodorize surfaces in toilet areas, behind and under sinks and counters, garbage cans and garbage storage areas, and other places where bacterial growth can cause malodors.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. Prepare a fresh solution daily or more often when the solution becomes diluted or soiled.

Do not mix with other chemicals or products including anionic soaps and detergents.

Remove gross filth or heavy soil prior to application of product.

To prepare an accurate solution use Airkem's metering pump to dispense 1/2 oz. of the concentrated product. Use one pump stroke (up and down) for each gallon of solution prepared. To increase detergency in heavily soiled areas use two (1:128) or three strokes (1:85) per gallon. Apply use-dilution with a cloth, mop, sponge, brush, scrubber, powered or mechanical spray device or by soaking. Wet all surfaces thoroughly and allow to remain wet for 10 minutes. No rinse necessary.

As a Detergent: OMEGA's detergents in combination with synergizing builders: (a) loosen soil through wetting and penetrating action; (b) suspend insoluble matter in solution; and (c) emulsify oil and grease. Low foaming OMEGA leaves surfaces without soil and film.

As a Disinfectant: When used as directed (1/2 oz. to 1 gallon) OMEGA provides approximately 469 ppm active quaternary. On hard, nonporous, inanimate environmental surfaces, OMEGA rapidly destroys a broad range of gram positive and gram negative organisms such as those listed below.

Germicidal vs. the following organisms when tested according to A.O.A.C. Use Dilution Procedure at 20 °C, in 5% blood serum and soap scum.

<i>Staphylococcus aureus</i>	ATCC # 6538	<i>Streptococcus pyogenes</i>	ATCC # 19615	<i>Salmonella enteritidis</i>	ATCC # 13076
<i>Salmonella choleraesuis</i>	ATCC # 10208	<i>Staphylococcus species</i>	ATCC # 12715	<i>Salmonella schottmuelleri</i>	ATCC # 10719
<i>Pseudomonas aeruginosa</i>	ATCC # 15442	<i>Vibrio cholerae</i>	ATCC # 25873	<i>Salmonella typhimurium</i>	ATCC # 13311
<i>Proteus vulgaris</i>	ATCC # 13345	<i>Corynebacterium</i>		<i>Shigella sonnei</i>	ATCC # 25831
<i>Escherichia coli</i>	ATCC # 11228	<i>Diphtheria</i>	ATCC # 11913	<i>Staphylococcus epidermidis</i>	ATCC # 12228
<i>Enterobacter aerogenes</i>	ATCC # 13048	<i>Neisseria gonorrhoeae</i>	ATCC # 19424	<i>Streptococcus mutans</i>	ATCC # 25175
<i>Micrococcus sedentarius</i>	ATCC # 27573	<i>Serratia marcescens</i>	ATCC # 9103	<i>Micrococcus luteus</i>	ATCC # 14452
<i>Pseudomonas cepacia</i>	ATCC # 25416	<i>Shigella flexneri</i>	ATCC # 25875	<i>Escherichia coli</i>	ATCC # 29181
<i>Salmonella gallinarum</i>	ATCC # 9184	<i>Klebsiella pneumoniae</i>	ATCC # 4352	<i>Campylobacter fetus</i>	ATCC # 27374
<i>Salmonella typhi</i>	ATCC # 6639	<i>Brucella abortus</i>		<i>Haemophilus influenzae</i>	ATCC # 19418
<i>Shigella dysenteriae</i>	ATCC # 29026	<i>Amoebae</i>	ATCC # 6872	<i>Staphylococcus aureus</i>	(methicillin resistant)
<i>Staphylococcus aureus</i>	ATCC # 14154	<i>Listeria monocytogenes</i>	(clinical isolate)		
<i>Streptococcus faecalis</i>	ATCC # 828	<i>Proteus mirabilis</i>	ATCC # 9240		

when tested according to A.O.A.C. Test.

*Virucidal (1:256): On hard, nonporous, inanimate environmental surfaces OMEGA kills Human Immunodeficiency Virus, HIV-1 (commonly known as the AIDS virus), Influenza A, (Aichi), Parainfluenza 1/Sandhi52, Canine distemper, Feline pneumonitis, Herpes simplex type 1 and type 2, Vaccinia/Ladaria-Chorioallantoic, Feline Herpes Virus (Feline rhinotracheitis)C-27, and Rotavirus type 3/Abney in the presence of protein (5% blood serum) and residual soap scum with a 10 minute contact time.

KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

Special instructions for cleaning and decontamination against HIV-1 on surfaces/objects soiled with blood/body fluids.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids, wear personal protection such as disposable latex gloves, gowns, masks, or eye coverings.

CLEANING PROCEDURES: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of disinfectant.

CONTACT TIME: 10 minutes.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

As a Fungistat: Apply use-dilution (1:256) to pre-cleaned, hard nonporous surface. Wet surface thoroughly. If not washed or rubbed off, it will control the growth of mold and mildew on environmental surfaces for a period of 7 days. To maintain fungistatic control, repeat application in 7 days or when new growth appears.

PRECAUTIONARY STATEMENTS—HAZARDS TO HUMANS AND DOMESTIC ANIMALS

DANGER: Corrosive. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Wear rubber gloves, chemical goggles and protective clothing. Harmful if swallowed. Remove contaminated clothing and wash before reuse. Avoid contamination of food.

STATEMENT OF PRACTICAL TREATMENT

IF IN EYES: Flush immediately with cool water. Remove contact lenses. Continue flushing for 15 minutes, holding eyelids apart. Get prompt medical attention.

IF ON SKIN: Wash with plenty of soap and water. Get medical attention if irritation persists.

IF SWALLOWED: Rinse mouth at once; then drink one or two glasses of water. DO NOT INDUCE VOMITING. Never give anything by mouth to an unconscious person.

CALL A POISON CONTROL CENTER OR PHYSICIAN IMMEDIATELY.

Note to Physicians: Probable mucosal damage may contraindicate the use of gastric lavage.

FOR EMERGENCY MEDICAL INFORMATION, CALL TOLL-FREE 1-800-328-0028

STORAGE AND DISPOSAL

Do Not Contaminate Water, Food or Feed by Storage or Disposal.

Storage: Store at ambient temperature. Keep container tightly closed when not in use. Do not freeze.

Container Disposal: Triple rinse (or equivalent). Then offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration, if allowed by state and local authorities, by burning, by burial, or by other means.

Particle Disposal: Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative in the nearest EPA Regional Office for guidance.

Airkem Professional Products Division of ECOLAB INC.
Ecolab Center, St. Paul, MN 55102
5105 Tomken Road, Mississauga, Ont. Canada L4W 2X5

ACCEPTED

SEP 18 1992

Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under 42964-14
EPA Reg No.

700880/9906/02
Made in U.S.A. © 19

BEST AVAILABLE COPY

ACCEPTED
SEP 13 1992
Under the Federal Insecticide,
Fungicide, and Rodenticide Act,
as amended, for the pesticide
registered under EPA Reg. No. *42044-14*

000000
000000
000000
000000
000000
000000
000000
000000
000000
000000

3700