

ALCIDE LD^{IM} BASE 4:1:1
for use with ALCIDE LD^{IM} ACTIVATOR 4:1:1

LD^{IM} kills bacteria, fungi and viruses on environmental surfaces. It is supplied as a concentrate for economy and convenience.


*Kills Influenza Virus, Type A2 (Hong Kong), Herpes Simplex Virus, Types I and II, Poliovirus, Type I, Coxsackievirus, Adenovirus-2, Echovirus, Type II, Vaccinia Virus and Vesicular Stomatitis Virus (VSV).

Active Ingredient: Sodium chlorite:.....	1.37%
Inert Ingredients.....	98.63%
TOTAL.....	100.00%

ALCIDE LD^{IM}
DISINFECTANT for MEDICAL ENVIRONMENTS
4:1:1

U.S. Patent #4,084,747 Contents: 1 qt. EPA Reg. No. 45631-14
Foreign Patents EPA Est. No. 45631XY01

ALCIDE Corporation
Westport, CT 06880

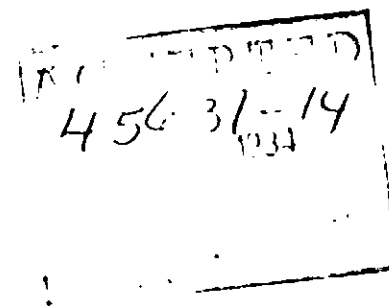

(Back panel of Base Label)

ALCIDETM LD

4:1:1

LD Disinfectant is recommended for hospitals, clinics, dental offices and other medical facilities to disinfect equipment, instruments, glassware, incubators, and tabletops.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with the labeling.

1. Mix with water as follows: Dilute 1 part Base into 4 parts tap water (20°-30°C), then add 1 Part Activator making 6 parts total. LD Base must be used with LD Activator. Example: Dilute 100 ml Base into 400 ml water, then add 100 ml Activator for 600 ml total. Disinfectant solution may appear clear or pale yellow. Once mixed, it is effective for 24 hours.

Moderate amounts of organic soil will not lower the efficiency of LD disinfectant. To ensure best results, however, equipment should be dismantled, if possible, and rinsed free of heavy soil and gross debris. Lumen of any hollow instrument must be flushed and filled.

3. Soaking is the best method of application; however, the solution may be sprayed or wiped on by any convenient means (sponge, etc.).

4. The surface, instrument, or apparatus should be thoroughly wet with solution. To inactivate bacteria, viruses, pathogenic fungi and *M. tuberculosis* apply for 1 minute.

5. Rinsing is not necessary after disinfection. Allow surfaces to air dry.

PRECAUTIONARY STATEMENT - DANGER TO HUMANS AND DOMESTIC ANIMALS

WARNING: Base concentrate is an irritant that may cause severe injury: causes skin irritation. Harmful if swallowed. Do not get in eyes, on skin, or on clothing. Wear goggles, gloves, shield, or protective mask. Wash thoroughly with copious water after handling. Avoid contact with food, drink, and wash before eating.

PRECAUTION: If swallowed, get medical attention immediately. If swallowed, do not induce vomiting. Get medical attention immediately. If swallowed, get medical attention immediately. If swallowed, get medical attention immediately. If swallowed, get medical attention immediately.

PRECAUTIONARY STATEMENT - DANGER TO HUMANS AND DOMESTIC ANIMALS

WARNING: Base concentrate is an irritant that may cause severe injury: causes skin irritation. Harmful if swallowed. Do not get in eyes, on skin, or on clothing. Wear goggles, gloves, shield, or protective mask. Wash thoroughly with copious water after handling. Avoid contact with food, drink, and wash before eating.

PRECAUTION: If swallowed, get medical attention immediately. If swallowed, do not induce vomiting. Get medical attention immediately. If swallowed, get medical attention immediately. If swallowed, get medical attention immediately.

for use with ALCID1 ID^{IM} BASE 4:1:1

Contains 7.5% Organic Acid

ALCIDE LD™
DISINFECTANT for MEDICAL ENVIRONMENTS
4:1:1

WARNING: Activator Concentrate causes substantial but temporary eye injury; causes skin irritation. Harmful if swallowed. Do not get in eyes, on skin or on clothing. Wear goggles, face shield or safety glasses. Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash before reuse.

FIRST AID: If in eyes: Flush with plenty of water. Get medical attention. If swallowed: Drink promptly a large quantity of milk, egg whites, gelatin solution, or if these are not available, drink large quantities of water. Avoid alcohol. Get medical attention. If on skin: Wash with plenty of soap and water. Get medical attention.

U.S. Patent No. 5,678,901	U.S. Pat. Reg. No. 4,064,14
Foreign Patent	JPA PSC. No. 4,064,14N01

[illegible]