

DIRECTIONS FOR USE:

For use in Hospitals, Dental Offices, Nursing Homes and Health Care Institutions for complete disinfection or sterilization of medical instruments and equipment. **thoroughly clean, rinse and rough-dry** objects before immersing in full strength solution. Flush and fill cleansed lumen of any hollow instrument. Avoid entrapment of air bubbles.

It is a Federal Law violation to use this product in a manner inconsistent with its labeling.

Disinfection: Immerse completely for a minimum of 10 minutes to destroy vegetative pathogens including *Mycobacterium tuberculosis*, *Pseudomonas aeruginosa* and viruses [Poliovirus Type 1, 2; Coxsackievirus B-1, B-3; Echovirus Type 6; Adenovirus Type 2; Herpes simplex; Influenza Type A2 (Hong Kong); Vaccinia] on inanimate surfaces.

Sterilization: Immerse completely for a minimum of 10 hours to destroy resistant pathogenic spores including *Clostridium sporogenes* and *Clostridium tetani*. Using aseptic technique, remove item from the solution and rinse thoroughly with sterile water.

Precleaning soak treatment: This solution does NOT coagulate blood. Immerse items from septic cases directly into fresh solution until final cleaning and terminal disinfection or sterilization of contaminated objects.

DIRECTIONS FOR ACTIVATING:

Add contents of the attached vial to this container of fluid. Mix well. Do not dilute. When activated, the disinfecting/sterilizing solution is green and ready for use. The shelf life of the activated, unused stock solution is 28 days. Discard daily or sooner if used solution becomes soiled or diluted.

Distributed by

Product No.

MX-2800

Glutaraldehyde 2.0%
Inert Ingredients 98.0%
Total 100.0%

ACCEPTED
with COMMENTS
In EPA Letter Dated:

OCT 07 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

46781-2

MIXING DATE

EXPIRATION DATE

EPA Est No. 46781-CO-1
EPA Reg No. 46781-2

ACCEPTED
with COMMENTS
In EPA Letter Dated:

OCT 07 1983

EPA Est No. 46781-CO-1
EPA Reg No. 46781-2

**METRICIDE 28
LONG-LIFE
ACTIVATED
DIALDEHYDE**

Activator Vial

For use with
**METRICIDE 28
LONG-LIFE
ACTIVATED
DIALDEHYDE
FLUID**

Directions: Add contents of this vial to the contents of the Metricide 28 Activator Vial. The solution will be green.

Danger: Keep out of reach of children. Contact with eyes causes irritation. In case of contact with eyes, flush with water immediately and obtain medical attention. For skin contact, flush thoroughly with water.

CONTAINER DISPOSAL

Empty container thoroughly with water. Do not reuse. Do not pour contents into water. Wash hands and clothing thoroughly.

Net Weight 15 grams
METREX RESEARCH CORP.

EPA Est No. 46781-2

Sterilizing and Disinfecting Solution

• Medical Equipment • Surgical Appliances

Rapid Acting...Corrosive Inhibitor...Non Staining

METRICIDETM

LONG—LIFE
ACTIVATED DIALDEHYDE

28

Sporicidal
Virucidal*

Fungicidal
Bactericidal

Tuberculocidal
Pseudomonacidal

DANGER: See right side panel for precautions.

*See Director's Order

ACCEPTED
WITH COMMENTS
EPA 700/1-10-11

METRICIDETM

STORAGE AND DISPOSAL — STORE IN A COOL PLACE

DANGER: Keep out of reach of children.

PROHIBITIONS:

Do not contaminate water, food or feed by storage or disposal. Open dumping is prohibited. Do not reuse empty container.

PESTICIDE DISPOSAL

Pesticide spray mixture or rinse water that cannot be used according to label instructions must be disposed of according to Federal or approved state procedures under Subtitle C of the Resource Conservation and Recovery Act.

CONTAINER DISPOSAL

Triple rinse (or equivalent) then offer for recycling or reconditioning and disposal of material in accordance with regulations allowed by State or local authority.

GENERAL

Use only Federal or State approved equipment and procedures for application of this product.

**DANGER: KEEP OUT OF REACH OF CHILDREN.
FOLLOW DIRECTIONS FOR USE
HARMFUL IF SWALLOWED**

Eyes - Causes severe eye irritation and may cause permanent eye damage. If eye contact occurs, flush immediately and thoroughly with water. Get medical attention.
Skin - Avoid contact with skin and clothing as possibility of sensitization exists. If skin contact occurs, wash thoroughly with water. Wash clothing.
Inhalation - Use in a ventilated area.
Food - Avoid contamination of food.
Instruments - Do not immerse carbon steel objects over 24 hours. Solution contains rust inhibitor but will not prevent or remove rust caused by poor or improper plating. Caution should be used to avoid immersion of and contact between dissimilar metals.

Wear protective glasses and rubber gloves when mixing and for all subsequent handling of the product.

STORAGE AND DISPOSAL:

Store this container and any of its unused contents in a cool, safe area. This product must be kept in a secured storage sufficient to make it inaccessible to children or persons unfamiliar with its proper use. Do not reuse empty container. Rinse empty container thoroughly with water and discard it.

**CONTENT ONE GAL. (128 FLUID OZ.)
MADE IN U.S.A.
CONTROL NUMBER ON BOTTOM**

Manufactured by

W. H. ...
...

07 OCT 1983

Letco Research Corp.
P.O. Box 272
Littleton, CO 80120

Attention: David Sturj
President

Gentlemen:

Subject: Long-Life Activated Maledylde (Liquid Activator)
EPA Registration No. 40741-2
Application dated August 24, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable, provided that you:

1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

a. We have noted the product rate change and have adjusted our records accordingly.

Old: Long-Life Activated Maledylde (Liquid Activator)
New: Insecticide ZK Long-Life Activated Maledylde

b. Revise the statement: "Manufactured by" to read "Manufactured by" since you are the basic registrant for this registered product.

Rev. our letter dated February 25, 1983, Item 2a.

c. Add the statement: Active Ingredient to appear directly above the statement: Maledylde.

Rev. our letter dated February 25, 1983.

d. Revise the entire label registration, using the

label for use on the product.

to read:

| | | | | | | | |
|--------------------|--|--|--|--|--|--|--|
| SYMBOL | | | | | | | |
| NAME | | | | | | | |
| DATE | | | | | | | |
| EPA Form 1300-4B | | | | | | | |
| OFFICIAL FILE COPY | | | | | | | |

