

PM31

46781-4

1084

US ENVIRONMENTAL PROTECTION AGENCY OFFICE OF PESTICIDES PROGRAMS REGISTRATION DIVISION (75-767) WASHINGTON, DC 20480	EPA REGISTRATION NO. 46781-4	DATE OF ISSUANCE DEC 20 1988
	TERM OF ISSUANCE	
	NAME OF PESTICIDE PRODUCT Metricide Plus 30	

NOTICE OF PESTICIDE: REGISTRATION
 REREGERISTRATION
 (Under the Federal Insecticide, Fungicide,
 and Rodenticide Act, as amended)

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

...
 ...
 ...

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.

Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

1. This registration is issued in accordance with FIFRA section 3(a)(2) and is subject to the conditions set forth in this notice.

2. The registrant is required to submit to the Administrator a copy of the final printed label for the product, and to file a copy of the same with the Administrator. The final printed label must be identical to the label submitted to the Administrator and must be printed in accordance with the label submitted to the Administrator.

3. The registrant is required to submit to the Administrator a copy of the final printed label for the product, and to file a copy of the same with the Administrator. The final printed label must be identical to the label submitted to the Administrator and must be printed in accordance with the label submitted to the Administrator.

4. The registrant is required to submit to the Administrator a copy of the final printed label for the product, and to file a copy of the same with the Administrator. The final printed label must be identical to the label submitted to the Administrator and must be printed in accordance with the label submitted to the Administrator.

5. If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6(b). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

[Signature]
 Director
 Registration Division (75-767C)

BEST AVAILABLE COPY

ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL	DATE
---------------------------------	------

REUSABLE
Sterilizing and Disinfecting Solution

Rapid-acting • Non-staining • Non-rusting

METRICIDE
PLUS 30

30-Day Long-Life Activated Dialdehyde Solution

Sporicidal
Virucidal*

Fungicidal
Tuberculocidal

Bactericidal
Pseudomonas

DANGER: Keep out of reach of children.

See right side panel for precautions.

Note: Contents of attached vial *must* be added to fluid before this product is used.
See directions for activating. See directions for use.

3 of 4

PRECAUTIONARY STATEMENTS

DANGER: Hazard to Humans

Follow All Label Directions

Eyes—Safety glasses or goggles are recommended. Contact with eyes causes damage. In case of contact, flush with water immediately and get medical attention.

Skin—Avoid skin contact as possibility of sensitization exists. May cause skin irritation. Use gloves.

For skin contact, flush immediately and thoroughly with water.

Ingestion—Avoid contamination of food. Harmful if swallowed.

Inhalation—Use in ventilated area.

STORAGE AND DISPOSAL

Keep in a cool place. Do not reuse empty container. Wrap container and put in trash.

Especially recommended for sterilization or disinfection of:

Anesthesia equipment

Respiratory therapy equipment

Metallic equipment or instruments

Rubber objects

Plastic objects

Thermometers

Contents One Gal. (128 fluid oz.)

Made in U.S.A.

Control Number on Bottom

ACCEPTED
with COMMENTS
in EPA Letter Dated:

DEC 20 1988

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.

46781-4

Manufactured by:



BEST AVAILABLE COPY

**METREX RESEARCH
CORPORATION**

Parker, Colorado 80134

F 112

37

6

4034

For complete sterilization or disinfection of medical instruments and equipment, thoroughly clean, rinse and rough-dry objects before immersing in full strength solution. Cleanse and rinse the lumens of any hollow instruments before filling with solution.

Avoid entrapment of air bubbles.

Disinfection: Immerse instruments or equipment completely for a minimum of 10 minutes at 20°C. to destroy vegetative organisms including *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Salmonella choleraesuis*, and pathogenic fungi as represented by *Trichophyton mentagrophytes*. For 100% kill of *Mycobacterium tuberculosis*, immerse instruments or equipment completely for a minimum of 20 minutes at 20°C. or 10 minutes at 25°C. **VIRUSES:**

The following viruses are completely inactivated in 3 minutes at 20°C.:

Cytomegalovirus; Respiratory Syncytial Virus; Rhinovirus; Rotavirus SA-11; Herpes simplex virus 1 & 2; Adenovirus 2; Coxsackievirus B5a; Echovirus 8; Poliovirus 1 & 2;

After immersion, remove instruments or equipment from METRICIDE PLUS 30 Solution and rinse thoroughly. All times and temperatures are based on EPA-approved testing on inanimate surfaces.

Sterilization: Immerse instruments or equipment completely for a minimum of 6 hours at 20°C. to destroy resistant pathogenic spores as represented by *Clostridium sporogenes* and *Bacillus subtilis*. Remove instruments or equipment from METRICIDE PLUS 30 Solution using sterile technique and rinse thoroughly with sterile water.

Add contents of attached vial to this container of METRICIDE PLUS 30 Solution. Shake. Solution is ready to use. When activated, solution changes color to bluish-green. This solution may be used and reused for up to 30 days after activation. Do not use activated stock solution beyond 30 days.

General Classification - For use in Hospitals, Dental Offices, Nursing Homes and Health Care Institutions. For repeated use in manual (bucket and tray) systems. This product is not formulated as a "holding" solution.

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Product No.

MX-3200

Active ingredient

Glutaraldehyde . . .	3.2%
Inert ingredients	96.8%
Total	100.0%

Contains Sodium Nitrite as a corrosion inhibitor.

MIXING DATE

EXPIRATION DATE

ACCEPTED
with COMMENTS
in EPA Letter Dated:

DEC 20 1988

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.

46781-1

E.P.A. Reg. No. 46781
E.P.A. Est. No. 46781-CO-1

BEST AVAILABLE COPY