

CAVIWIPES

[Front Panel – The following verbiage will only appear on canister containers.]

DISINFECTING TOWELETTES

[Front Panel – The following verbiage will only appear on individually wrapped packets.]

DISINFECTING TOWELETTE

TUBERCULOCIDAL • BACTERICIDAL • VIRUCIDAL* • FUNGICIDAL


ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride	0.233%
Isopropanol	14.300%
INERT INGREDIENTS	85.467%
TOTAL	100.000%

KEEP OUT OF REACH OF CHILDREN

CAUTION

See back panel for additional precautionary statements.

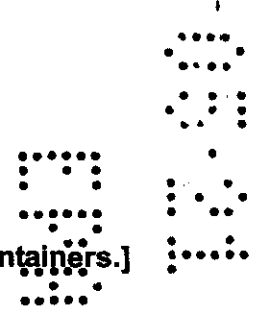


[Front Panel – The following verbiage will only appear on canister containers.]

CaviWipes Disinfecting Towelettes are non-woven disposable towelettes pre-saturated with *CaviCide*. *CaviWipes* are intended for use in health care settings such as hospitals, laboratories, clinics, dental offices, ophthalmic offices and veterinary facilities. *CaviWipes* are effective against the following microorganisms on hard, non-porous surfaces when used as directed:

[Front Panel – The following verbiage will only appear on individually wrapped packets.]

CaviWipes Disinfecting Towelette is a non-woven disposable towelette pre-saturated with *CaviCide*. *CaviWipes* is intended for use in health care settings such as hospitals, laboratories, clinics, dental offices, ophthalmic offices and veterinary facilities. *CaviWipes* is effective against the following microorganisms on hard, non-porous surfaces when used as directed:

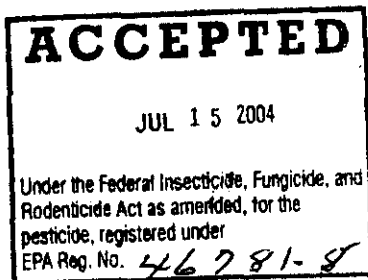

286

- Mycobacterium tuberculosis var: bovis (BCG)
- Staphylococcus aureus
- Pseudomonas aeruginosa
- Salmonella choleraesuis
- Trichophyton mentagrophytes
- Methicillin Resistant Staphylococcus aureus (MRSA)
- Vancomycin Resistant Enterococcus faecalis (VRE)
- Hepatitis B Virus (HBV)*
- Herpes simplex virus types 1 and 2*
- Human Immunodeficiency Virus (HIV-1)*

EPA Reg. No. 46781-8

EPA Est. No. 56952-WI-001 [and/or] 71979-SC-001

REORDER No.

Net Contents: # Pre-Saturated Towelettes # x # in.

Packed For: Metrex Research Corporation
28210 Wick Road
Romulus, Michigan 48174

ACCEPTED

JUL 15 2004

Under the Federal Insecticide, Fungicide, and
Rodenticide Act as amended, for the
pesticide, registered under
EPA Reg. No. 46781-8

[Side Panel – The following verbiage will only appear on canister containers.]

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

CaviWipesCaviWipes Disinfecting Towelettes are non-woven disposable towelettes pre-saturated with **CaviCide**. **CaviWipesCaviWipes** are intended for use on hard non-porous surfaces in health care settings such as hospitals, emergency medical settings, neonatal units, surgical centers, laboratories, clinics, nursing homes, dental offices, ophthalmic offices, veterinary facilities, salon settings, emergency vehicles, correctional facilities, schools, health club facilities, non-food contact surfaces and other critical care areas where control of cross contamination is required. **CaviWipesCaviWipes** will clean and disinfect, when used as directed, such items as tables, chairs, counter tops, carts, baskets, cabinets, lights, telephones, dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, exterior surfaces of infant incubators, exterior surfaces of anesthesia machines and respiratory therapy equipment which are made of stainless steel, glass, Plexiglas®, and plastics (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene). **DO NOT USE AS A BABY WIPE.**


[Note: Listing of any of the above areas, equipment, instruments, hard non-porous surfaces in production labels is optional.]

Dispenser Directions:

To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

Cleaning and Disinfection Instructions:

Use one **CaviWipesCaviWipes** towelette to completely preclean surface of all gross debris. To disinfect the precleaned surface, use a second **CaviWipesCaviWipes** towelette to thoroughly wet the surface and allow to remain visibly wet for 3 minutes at 20°C. For tuberculocidal disinfection, allow the surface to remain visibly wet for 5 minutes at 20°C.

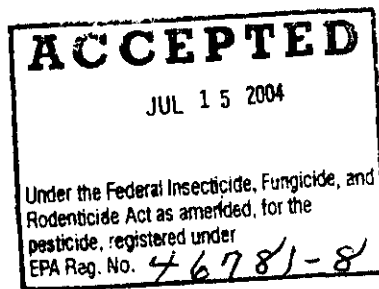

[Side Panel – The following verbiage will only appear on individually wrapped packets.]

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

CaviWipes Disinfecting Towelette is a non-woven disposable towelette pre-saturated with **CaviCide**. **CaviWipes** is intended for use on hard non-porous surfaces in health care settings such as hospitals, emergency medical settings, neonatal units, surgical centers, laboratories, clinics, nursing homes, dental offices, ophthalmic offices, veterinary facilities, salon settings, emergency vehicles, correctional facilities, schools, health club facilities, non-food contact surfaces and other critical care areas where control of cross contamination is required.

CaviWipes will clean and disinfect, when used as directed, such items as tables, chairs, counter tops, carts, baskets, cabinets, lights, telephones, dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, exterior surfaces of infant incubators, exterior surfaces of anesthesia machines and respiratory therapy equipment which are made of stainless steel, glass, Plexiglas®, and plastics (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene). **DO NOT USE AS A BABY WIPE.**


[Note: Listing of any of the above areas, equipment, instruments, hard non-porous surfaces in production labels is optional.]

Packet Directions:

Open packet, remove towelette and unfold.

Cleaning and Disinfection Instructions:

Use one **CaviWipes** towelette to completely preclean surface of all gross debris. To disinfect the precleaned surface, open and use a second **CaviWipes** towelette to thoroughly wet the surface and allow to remain visibly wet for 3 minutes at 20°C. For tuberculocidal disinfection, allow the surface to remain visibly wet for 5 minutes at 20°C.

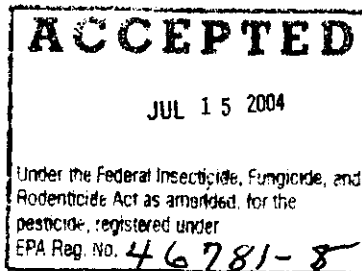

CaviWipes effectively kills HIV and HBV on precleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) and Hepatitis B Virus (HBV).

Special Instructions for Cleaning and Decontamination Against HIV-1 (Human Immunodeficiency Virus) and HBV of Surfaces or Objects Soiled With Blood or Body Fluids:

Personal Protection: When using this disinfecting towelette, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A **CaviWipes** towelette can be used for this purpose.

Contact Time: Thoroughly wet surface with a new **CaviWipes** towelette. Allow to remain visibly wet for 3 minutes.

Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids should be autoclaved and disposed of in accordance with Federal, State and local regulations for infectious materials disposal.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION

Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for further advice.


If swallowed: Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything to an unconscious person.

If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment.

If inhaled: Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth if possible. Call a poison control center or doctor for further treatment advice.

For emergency information on product, call Metrex at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time. After 4 p.m., call Chemtrec 24-Hour Emergency Service at 1-800-424-9300.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

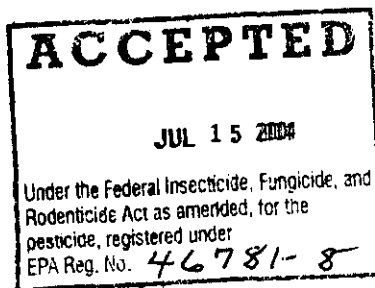

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

[Side Panel – The following verbiage will only appear on canister containers.]

STORAGE AND DISPOSAL

Towelette - Do not reuse towelette. Dispose of used towelette in trash.

Dispenser - Do not reuse empty container. Wrap container and discard in trash.

For product or technical information, contact Metrex at **800-841-1428** or visit our web site at **www.metrex.com**.

[Side Panel – The following verbiage will only appear on individually wrapped packets.]

STORAGE AND DISPOSAL:

Towelette - Do not reuse towelette. Dispose of used towelette and empty packet in trash.

For product or technical information, contact Metrex at **800-841-1428** or visit our web site at **www.metrex.com**.

[The following graphic is optional for canister containers only. It will be affixed to the label/canister so that it does not interfere with any label text. Due to size constraints, it will not appear on individually wrapped packets.]

